AUF 1, AUF 2

de	Gebrauchsanweisung Aufnahme für Übertragungsinstrumente
da	Brugsanvisning Holder til hånd- og vinkelstykker
es	Instrucciones de manejo Alojamiento para instrumental de transmisión
fi	Käyttöohje Pidike kapeita ja onttoja instrumentteja varten
fr	Mode d'emploi Support pour instruments rotatifs
en	Operating instructions Holder for transmission instruments
hr	Upute za uporabu Držać za prijenosne instrumente
it	Istruzioni d'uso Supporto per strumenti di trasmissione
nl	Gebruiksaanwijzing Houder voor hand- en hoekstukken
no	Bruksanvisning holder for roterende instrumenter
pt	Instruções de utilização Encaixe para instrumentos de transmissão
sv	Bruksanvisning Hållare för roterande instrument
sl	Navodila za uporabo nastavka za nasadne instrumente

de	4
da	15
es	26
fi	37
fr	48
en	59
hr	70
it	81
nl	92
no	103
pt	114
sv	125
sl	136

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en - Contents

Notes about these instructions	59
Queries and technical problems.....	59
Intended use	60
Items supplied	61
Scope of delivery AUF 1.....	61
Optional accessories.....	62
Disposal of the packing material	62
Warning and Safety instructions	63
Areas of application	64
Filters.....	64
Installation	65
Injector manifolds with screw thread	65
Injector manifolds with silicone adapters	65
Reprocessing transmission instruments	67
Cleaning or replacing filters.....	68
Replacing filter plates.....	68

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

■ Select an option using the arrow buttons and save your choice with *OK*.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

Queries and technical problems

In the event of queries or technical problems, please contact Miele. Contact details can be found at the end of the operating instructions for your cleaning machine or at www.miele.com/professional.

This holder can be used when reprocessing transmission instruments used in dental applications (dental handpieces, for example) in special Miele washer-disinfectors. Follow the operating instructions for your washer-disinfector as well as the instructions of the medical device manufacturer on how to reprocess their items by machine.

The wash water used for reprocessing transmission instruments needs to be filtered.

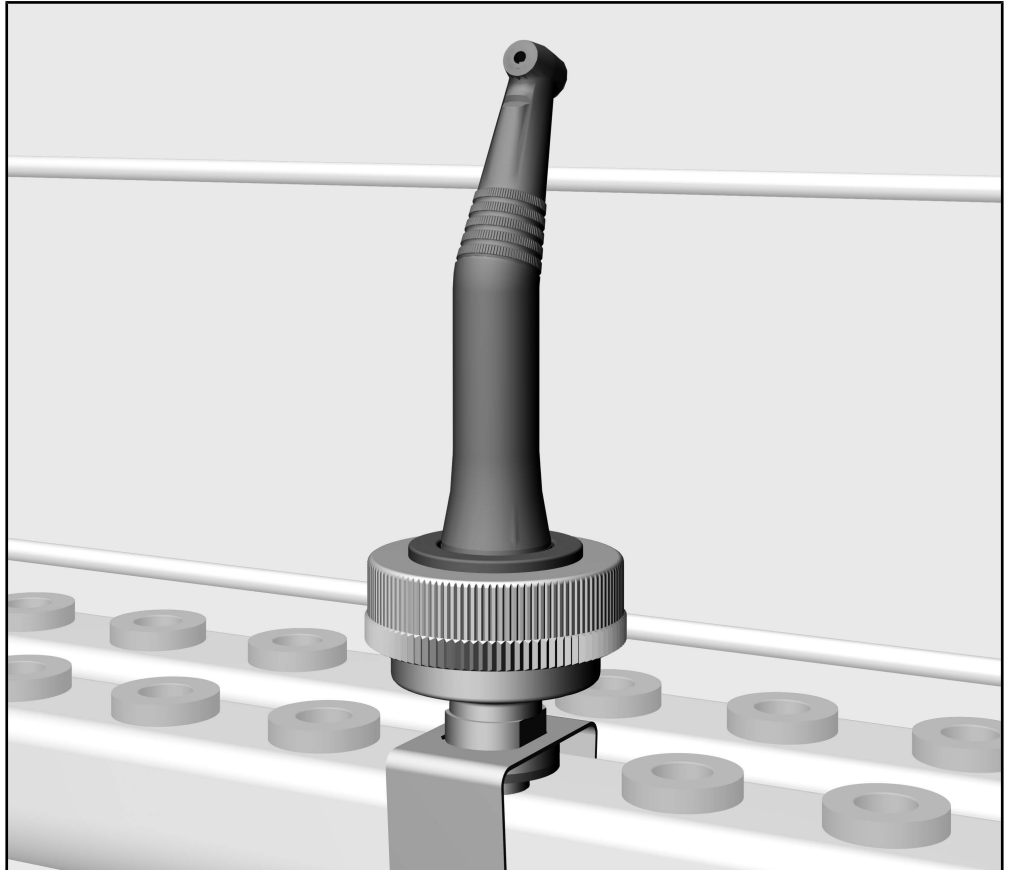
The clamp supplied with the AUF 1 holder can be used for mounting on injector manifolds with silicone adapters.

The clamp supplied with the AUF 2 holder is designed solely for mounting on the injector manifolds of the O 801/2 upper basket.

You will need to use the A 801 threaded adapter to mount the holder on injector manifolds with a screw thread.

The washer-disinfector is generally referred to as “the machine” in these operating instructions. Reprocessable medical devices are referred to as “load items” if they are not more closely defined.

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- Two-piece adapter, comprising an upper part and a lower part
 - Height approx. 48 mm, Ø 45 mm

Items supplied

- 1 x sealing ring
- 1 x threaded bush
- 1 x clamp
- 5 x filter plates, porosity 2, Ø 30 mm

Optional accessories

- A 801, threaded adapter
- ADS 1, silicone adapter, white, for holding transmission instruments with a $\varnothing$ of approx. 20 mm
- ADS 2, silicone adapter, green, for holding transmission instruments with a $\varnothing$ of approx. 16 mm
- ADS 3, silicone adapter, red, for holding transmission instruments with a $\varnothing$ of approx. 22 mm
- FP 20, 20 filter plates, porosity 2, $\varnothing$ 30 mm

Other optional accessories are available to order from Miele.

Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

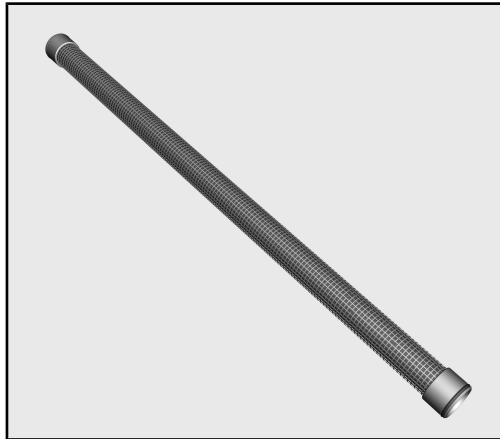
Read the operating instructions carefully before using this adapter. Keep these operating instructions in a safe place.

 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ The adapter has been approved for the applications stated in these operating instructions. Applications that go beyond those listed and use of the adapter with additional nozzles, irrigation sleeves and rinse fittings from the Miele range of accessories are described in the operating instructions for the accessories or are to be agreed with Miele. Conversions, modifications and any other use are not permitted.
- ▶ New load carriers must be cleaned in the washer-disinfector without a load prior to first use.
- ▶ Inspect all mobile units, baskets, modules and inserts as described in the “Maintenance” section of the operating instructions for your machine.
- ▶ Only items which have been declared by their manufacturer as suitable for machine reprocessing may be processed. The manufacturer's specific reprocessing instructions must be observed. Disposable instruments must not be put into the machine for processing.
- ▶ To clean and disinfect lumened instruments thoroughly from the inside, they must be attached to the relevant holders or nozzles on the injector manifolds. The connectors must not come loose during a programme sequence.
At the end of every programme sequence, check that the connectors have not come loose.
- ▶ Any hollow items that have become disconnected from their injector nozzles, adapters or holders during processing must be re-processed.
- ▶ The last rinse, as a minimum, must be carried out with DI water.

Filters

The wash water used for reprocessing loads with narrow lumens needs to be filtered.

- The injector manifold must be fitted with an A 800 tubular filter.

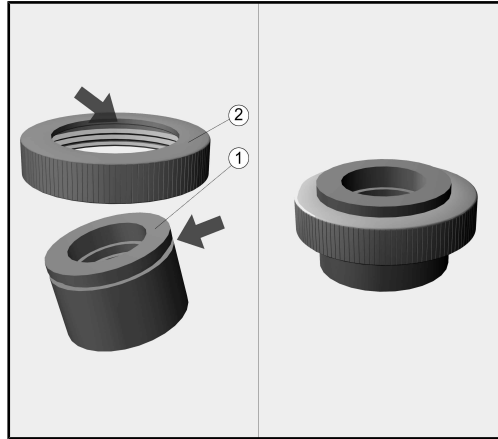
The tubular filter is supplied with its own operating instructions. Follow the instructions on how to perform regular cleaning on the tubular filter.

An FP 20 filter plate can be inserted into the holder instead of the tubular filter.

Do not insert a filter plate into the holder if the injector manifold has been fitted with an A 800 tubular filter.

Some injector manifolds (those with silicone adapters, for example) cannot be fitted with a tubular filter. In such cases the holder must always be fitted with an FP 20 filter plate.

Installation

- Unscrew the upper part ② from the holder.
- Fit a silicone adapter (ADS) ① in its place which has the same diameter as the transmission instrument being reprocessed.

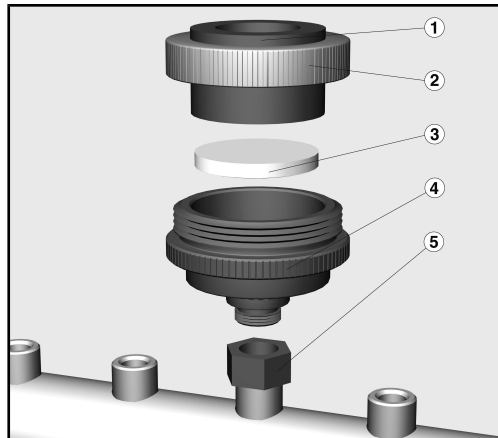
⚠ Damage to filter plates

Fitting a transmission instrument into a silicone adapter that is too big can cause damage to the filter plate. Damage to filter plates may affect their ability to effectively filter out dirt particles from the wash water, leading to blockages in the channels of the transmission instruments.

Only use silicone adapters (ADS) with the same diameter as the transmission instrument being reprocessed.

Injector manifolds with screw thread

An A 801 adapter is required.

- Remove an injector nozzle from the injector manifold and replace it with the A 801 ⑤ adapter.
- Screw the lower part of the ④ holder into the adapter.
- Insert a filter plate ③ into the lower part.

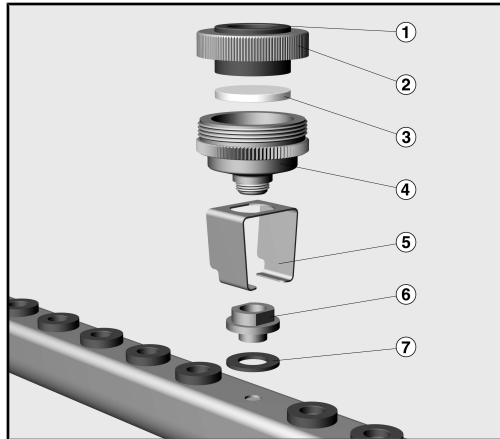
⚠ Do not insert a filter plate into the holder if the injector manifold has been fitted with an A 800 tubular filter.

- Screw the upper part into the lower part.

en - Areas of application

Injector manifolds with silicone adapters

- Remove a nozzle and the silicone adapter from the injector manifold.
- Place the sealing ring ⑦ around the injector port that is now free.
- Insert the threaded bush ⑥ into the injector port.
- Secure the threaded bush with the clamp ⑤. The clamp must reach around the injector manifold.


- Screw the lower part of the adapter ④ into the threaded bush.
- Insert a filter plate ③ into the lower part.
- Screw the upper part into the lower part.

Reprocessing transmission instruments

Before loading check that the silicone adapters are present and correctly fitted.

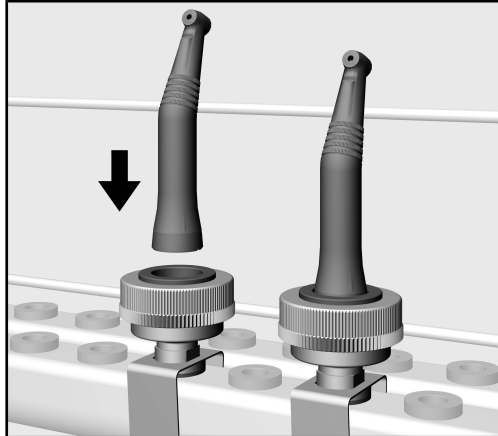- Wipe away any deposits from the area around the silicone adapters that may be present depending on local water conditions using a damp cloth.
- Check that internal sections of transmission instruments are free of blockages.

- Fit the instruments into their holders using the respective silicone adapters.
- After reprocessing, the insides of the transmission instruments must be dried with sterile compressed air before being cleaned and sterilised as appropriate in accordance with the manufacturer's instructions. Please observe national regulations.

Before using transmission instruments again following reprocessing, a function check must be carried out, e.g. by spraying into a basin, to ensure they are clear.

Cleaning or replacing filters

Cleaning the A 800 tubular filter

The tubular filter must be cleaned **at least once a week**.

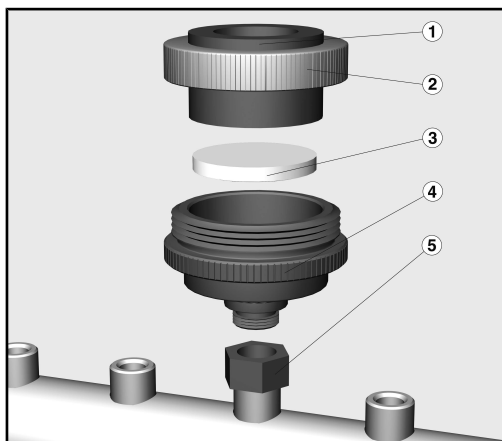
We recommend cleaning the tubular filter after every 10 programme sequences.

- Follow the cleaning instructions in the operating instructions for the tubular filter.

Replacing filter plates

⚠ Filter plates must be replaced **approx. every 2 weeks** or after **20 reprocessing cycles** have been completed.

- Unscrew the holder completely.

- Dismantle the holder and dispose of the filter plate ③.
- Rinse out the lower part ④ thoroughly.
- Insert a new filter plate.

⚠ Damaged or used filter plates must not be inserted under any circumstances.

- Screw the upper and lower parts of the holder back together and fit the holder back onto the injector manifold.

Do not insert a filter plate into the holder if the injector manifold has been fitted with an A 800 tubular filter.

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