ATT 86

- de** Betriebsanleitung Transferwagen
- en** Operating instructions Transfer trolley
- da** Betjeningsvejledning Transportvogn
- fr** Mode d'emploi Chariot de transfert
- it** Istruzioni d'uso Carrello di trasporto
- no** Bruksanvisning Transporttralle

de	5
en	35
da	65
fr	95
it	127
no	158

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Notes about these instructions	35
Queries and technical problems.....	35
Warnings and safety notes	36
Appropriate use.....	36
Please pay attention to the following notes to avoid injury	37
Safety notes and warnings for the rechargeable battery and charger	38
Basic requirements.....	39
Using accessories	39
Cleaning instructions.....	39
Symbols on the transfer trolley.....	40
Functional description	42
Product description.....	43
Appliance overview	43
Commissioning	44
Notes on installation.....	44
Adjusting the height	46
Transfer trolley control elements	47
Areas of application	48
Safe shutdown	48
Interrupting the transport procedure	49
Locking of the transfer trolley blocked	49
Operation.....	50
Docking the transfer trolley	50
Undocking the transfer trolley	51
Drain connection in the drip tray	52
Rechargeable battery information	52
General.....	52
Before use.....	53
Discharge	53
Charging.....	53
Service life.....	53
Replacing the rechargeable battery	54
Maintenance	55
Cleaning and disinfecting the transfer trolley	55
Cleaning the drip tray	55
Checking the locking system	56
Maintenance	57
Technical data.....	58
Caring for the environment	59
Disposal of your old machine.....	59
Enclosure	60
General	60
Training record	60
Appliance details	60
Special (optional) specifications for user training	61

en - Contents

Scope of training	61
Participants	61

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property.

Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow buttons and save your choice with OK.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

Queries and technical problems

In the event of queries or technical problems, please contact Miele. Contact details can be found at the end of the operating instructions for your cleaning machine or at www.miele.com/professional.

Original operating instructions

 Risk to health and risk of damage if the operating instructions are not followed.

Failure to follow the operating instructions – and in particular the safety notes and warnings they contain – may result in persons being seriously injured or property being seriously damaged.

Read these operating instructions carefully before using Miele accessories. It is also essential that you pay attention to the operating instructions for the cleaning machine. This will prevent both personal injury and damage to accessories.

Keep the operating instructions in a safe place.

Conversions, modifications and use for any other purpose are not permitted and could be dangerous.

The washer-disinfector is generally referred to as “the machine” in these operating instructions.

Appropriate use

- The conveyor system is only approved for the range of applications specifically mentioned in these operating instructions. All other uses, conversions and modifications made by unauthorised persons are not permitted and are considered dangerous.
- The conveyor system is intended for indoor use only.
- The conveyor system is only intended to load and unload load carriers approved by Miele for use with the conveyor system.
- Transfer trolleys may only be stopped in level areas.
- The operator must not modify the construction or the facility without first consulting with Miele.
- The system operator must check the conveyor system disinfection process, especially when retrofitting conveyor systems.
- The conveyor system must not be used as a resting place for objects.
- The height of the variable transfer trolley must never be adjusted while the transfer trolley is docked onto the conveyor system or to the cleaning machine.
- Only items approved by Miele may be used for the conveyor system and the wider system.
- This conveyor system should be commissioned, serviced and maintained by the Miele Customer Service Department or by a Miele authorised service dealer/agent only. A Miele service/maintenance contract is recommended to ensure continued safe functioning of the system. Unauthorised repairs and other work could be dangerous to users and to patients.

Please pay attention to the following notes to avoid injury

- A damaged machine is a safety risk. In the event of damage, disconnect the machine from the mains electricity supply immediately and contact Miele Service.
- The facility must not be operated if any safety devices have been removed.
- During cleaning and maintenance work the facility must be disconnected from the electricity and prevented from inadvertent switching on.
- Maintenance and repairs must not be carried out while the facility is in operation.
- Maintenance and inspection points must be kept clear of any obstructions.
- Ensure the working area of the operator is sufficiently illuminated.

Removing the load

The mobile units, loads and any liquids or steam can reach temperatures up to 135 °C, creating a risk of burning or scalding if they come into contact with bare hands.

Suitable gloves and long-sleeved clothing must be worn when loading and unloading to prevent the risk of burning and scalding.

Transporting hot loads

The temperature of the load carriers may still be as much as 55 °C when the transfer trolley is being transported or has stopped. Touching them with bare hands presents a risk of injury.

Special care must be taken when transporting hot load carriers. Suitable protective clothing must be worn during transport.

- For your own safety, wear gloves and, if necessary, a surgical mask and protective goggles when sorting the instruments and unloading the machine. To avoid recontamination, wear clean gloves when unloading the machine.
- Only use the conveyor system if it is functioning correctly. Otherwise you could endanger yourself and others.
- The loading accessories for the variable transfer trolley may only be used when in a fault-free condition.
- It is the duty of the supervisor to inform machine operatives of the dangers of incorrectly loading and unloading the large steam steriliser.
- Riding, stepping on or climbing over the conveyor system while it is in use is prohibited.

en - Warnings and safety notes

- ▶ It is the duty of the supervisor to inform the machine operatives about the prevention and consequences of overloading.
- ▶ Jewellery, long hair or loose clothing could present a safety risk. Appropriate safety measures must be taken to avoid injury.
- ▶ Access to the infeed and outfeed areas by personal who have not received proper training and instruction is not permitted.
- ▶ Loading and unloading areas, work areas and thoroughfares must be kept clear.
- ▶ The operator must ensure that emergency exits are kept clear of any obstructions.
- ▶ The transfer trolley must only be coupled to the cleaning machine or to the conveyor system when in the correct position. Vertical adjustment is considered gross misuse.
- ▶ Never adjust the trolley height when there people in the vicinity of the height-adjustable transfer trolley. Always stand behind the transfer trolley handle when adjusting the trolley height.
- ▶ Only move the height-adjustable transfer trolley when the lifting device is at its lowest position.
- ▶ Be sure to keep your hands away from the height-adjustable transfer trolley when raising and lowering it.
- ▶ The conveyor system must be checked every day for the correct functioning of the safety related locking mechanisms.
- ▶ Transporting people or riding on the conveyor system is prohibited.

Safety notes and warnings for the rechargeable battery and charger

- ▶ The rechargeable battery and charger must be used for the variable transfer trolley. For safety reasons, the transfer trolley must not be used with a rechargeable battery or charger produced by another manufacturer.
- ▶ If you are using a mains plug adapter, make sure that it is approved for continuous operation.
- ▶ Stop using the rechargeable battery immediately if you notice an unusual smell from it, if liquid is leaking from it, if it is significantly overheating or if it is discoloured or deformed. Dispose of the rechargeable battery immediately.
- ▶ Batteries will deplete over time. They can then leak corrosive fluids. Avoid contact with the eyes and skin. In the event of contact, rinse thoroughly with water and seek medical advice.

- Disposing of the rechargeable battery: remove the rechargeable battery from the transfer trolley. Insulate the metal contacts with adhesive tape to prevent short circuits. Dispose of the rechargeable battery in a municipal recycling facility. Do not dispose of the rechargeable battery with your household waste.
- Do not allow the rechargeable battery to come into contact with liquids.
- Do not cause a short circuit by bridging the contacts of the rechargeable battery, either unintentionally or deliberately.
- Do not dismantle the rechargeable battery.
- Do not drop or throw the rechargeable battery. For safety reasons, do not continue to use a rechargeable battery which has been damaged.
- Keep the rechargeable battery away from naked flames or other heat sources. Do not heat the rechargeable battery. Do not expose the rechargeable battery to direct sunlight.

Basic requirements

- Operators of this conveyor system must be trained either by the Miele Professional Service Department or by a Miele authorised person. Unauthorised and untrained personnel must not be allowed access to the conveyor system or its controls.
- The supervisor of the conveyor system is responsible for personnel being inducted into the operational procedures of the system.
- The manufacturer cannot be held liable for damage caused by non-compliance with these Warning and Safety Instructions.

There is a training record template in the operating instructions of the large steam steriliser.

- Non-compliance with these warning and safety instructions will void the warranty for the conveyor system.

Using accessories

- Only genuine Miele accessories or Miele recommended additional accessories may be combined with the conveyor system. The model designations of the specific additional accessories are available from the Miele Customer Service Department. Making modifications to Miele accessories or using additional accessories which are not authorised can cause unsatisfactory results. Any resultant damage would also invalidate the manufacturer's warranty.

Cleaning instructions

- The conveyor system must not be cleaned in a washer-disinfector or sterilised in a steriliser.

en - Warnings and safety notes

- ▶ When handling cleaning agents please follow the manufacturer's instructions and observe the relevant safety data sheets.
- ▶ Ensure that solutions or steam containing hydrochloric acid do not come into contact with the outer casing of the conveyor system in order to avoid any damage through corrosion.
- ▶ Any labels on the equipment are wipe-resistant, but should nevertheless be handled with care.

Symbols on the transfer trolley

Symbols on the transfer trolley:

	Unlock or transfer position
	Lock or loading position
	General warning, caution or fault
	Caution: risk of crushing

en - Applicable directives and standards

The variable transfer trolley complies with all relevant European directives and standards and carries the CE mark.

Directives

Machinery Directive

2006/42/EC relating to machinery. Current amendment.

Standards

EN 619:2002 + A1:2010

Continuous handling equipment and systems –
Safety and EMC requirements for equipment for mechanical handling
of unit loads.

en - Functional description

The ATT 86 variable transfer trolley is used for transporting load carriers from preparation areas to washer-disinfectors, and from washer-disinfectors to checking and packing tables.


In cases where automatic conveyor systems are being used for loading washer-disinfectors, the ATT 86 transfer trolley is used to place load carriers on them and transport load carriers away from them.

The transfer trolley must only be operated by persons who have been properly instructed and trained in its use. There is a training record template enclosed with these instructions.

The transfer trolley is designed exclusively for transporting load carriers used with the following washer-disinfector types:

- PLW 8615
- PLW 8616
- PLW 8617
- PWD 8626
- PWD 8628

Appliance overview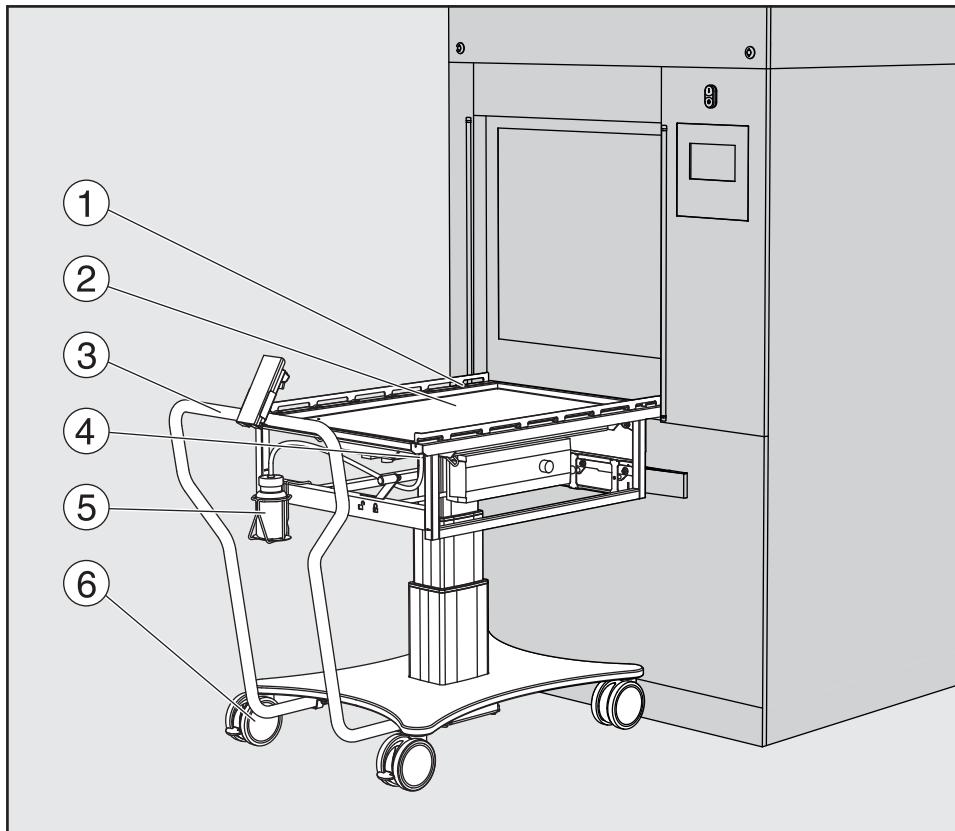

- ① Conveyor rollers
- ② Drip tray
- ③ Handle
- ④ Drain connection
- ⑤ Drip bottle with holder (accessory sold separately)
- ⑥ Wheels with parking brakes and height adjustment

Accessories sold separately

Additional accessories are available from Miele:

- LDTT86, bottle holder, 1 l bottle and connection hose

Notes on installation

 The transfer trolley must only be installed by the Miele Customer Service Department or by a Miele authorised service dealer.

The all-pole power disconnection of the variable transfer trolley is achieved by disconnecting the power supply.

The outer casing of the device must not be used as a deposit or storage area for utensils or other objects.

The transfer trolley can only be coupled in the correct position. The vertical position must not be changed after coupling.


The height of the transfer trolley is adjusted when the cleaning machine or conveyor system is commissioned.

Successful commissioning must be documented in a training certificate. A copy of this certificate should then be filed with the supervisor. The training certificate must also document the fact that instructions on using the equipment were provided to users, and the names of the users present.

The conveyor system is intended for indoor use only.


The height of the variable transfer trolley must never be adjusted while the transfer trolley is docked onto the conveyor system or to the cleaning machine.

Escape routes must be kept clear at all times. This is particularly important when using several transfer trolleys.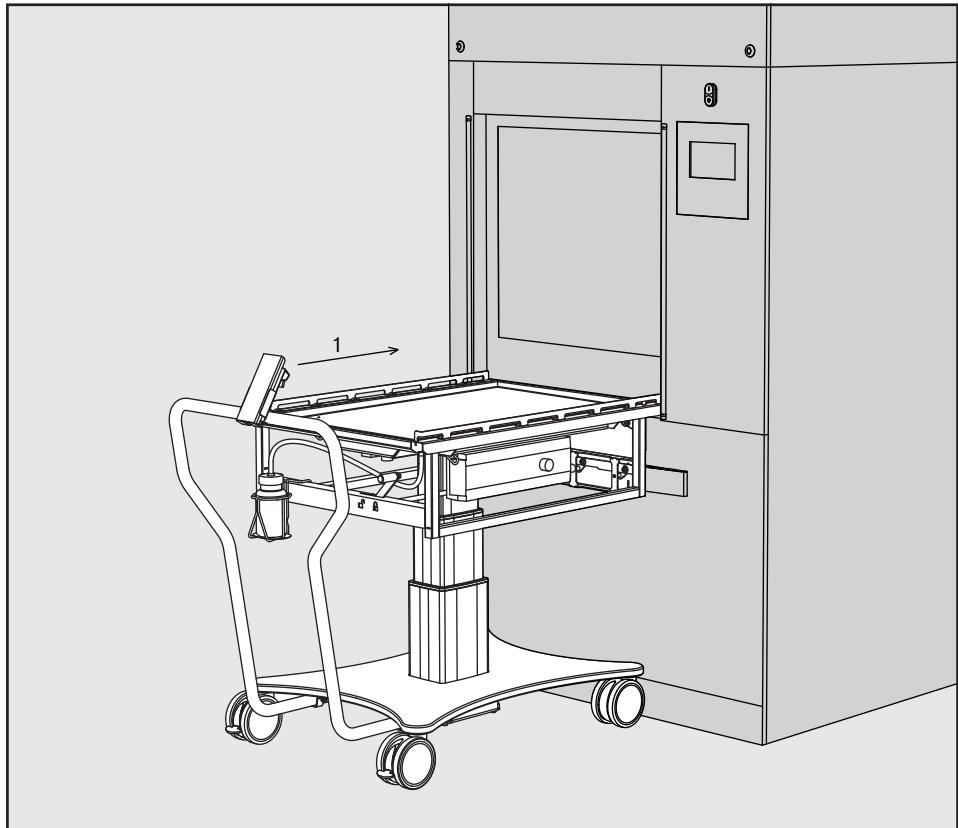


1 Vertical position

Adjusting the height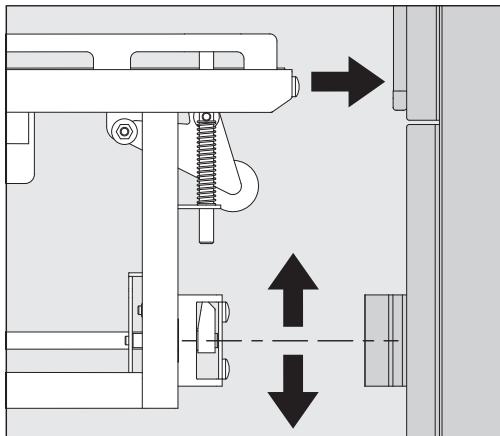

The height of the transfer trolley can be adjusted using the lift cylinder. The height must be set so that the docking mechanisms of the transfer trolley and the cleaning machine engage. This is performed by the Miele Customer Service Department or a specialist company authorised by Miele.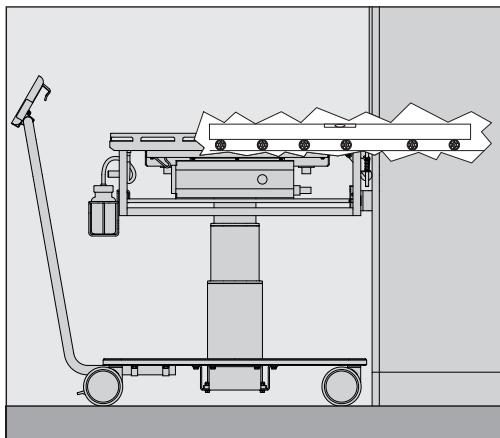

The conveyor rollers on the transfer trolley and in the cleaning machine must be at the same height so that the load carriers can be moved easily.

It is possible to set 3 different height positions using a control element. The control element is on the handle of the transfer trolley.

- Press the “S” function button until a signal sounds (this will take around 5 to 7 seconds).
- Press the selected number button (within 2 seconds) until another signal sounds.

The position is saved. When the number button is pressed, the height-adjustable transfer trolley moves into the required position.

Transfer trolley control elements

Operating buttons	
↑	Lift
↓	Lower
1, 2, 3	Buttons for moving into saved positions
S	Function button for saving a position

Safe shutdown

The transfer trolley is shut down safely by disconnecting the power supply. This is required for cleaning and maintenance work, for example.

The rechargeable battery on the transfer trolley is removed:

- Make sure the rechargeable battery pack is correctly connected.
- Press the snap lock on the rechargeable battery.
- The rechargeable battery can be detached from the holder and removed.

Interrupting the transport procedure

If the transfer trolley cannot be raised or lowered during the transport procedure, the rechargeable battery may be discharged.

- Check the battery capacity.
- Connect the battery to the charger if necessary.

 If the fault occurs again, contact the Miele Customer Service Department immediately.

Locking of the transfer trolley blocked

The transfer trolley cannot be locked after docking onto the conveyor system or the cleaning machine.


- Check whether the transfer trolley and the cleaning machine or the conveyor system have a common central axis.
- Correct the setting if required and activate the locking mechanism once again.

 If the fault occurs again, contact the Miele Customer Service Department immediately.

Docking the transfer trolley


If the cleaning machine is equipped with a conveyor belt system, the transfer trolley can be docked onto the loading and unloading belts.

The service panel on the cleaning machine must be securely locked.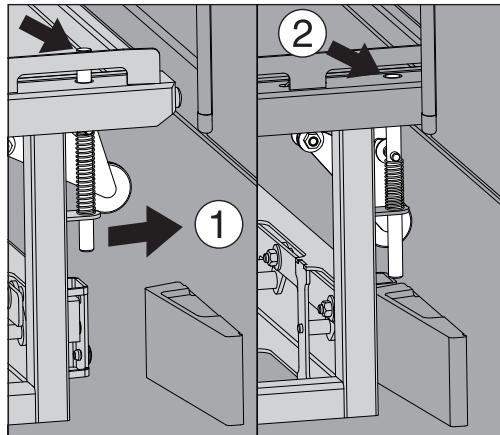

- Move the transfer trolley towards the cleaning machine and dock it onto the docking mechanism on the machine ①.


The safety device on the transfer trolley triggers automatically ②.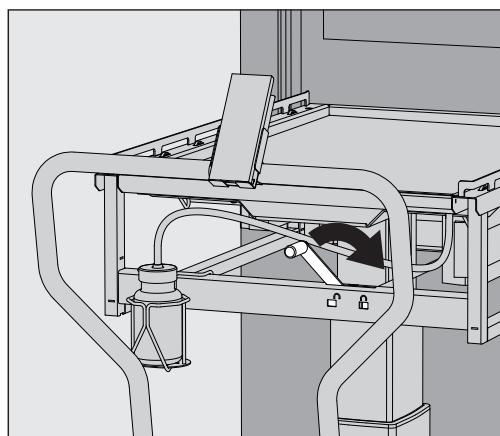

- Turn the locking lever on the transfer trolley to the right .

The transfer trolley is now locked in the docking mechanism on the cleaning machine.

Check that the connection between the cleaning machine and the transfer trolley is secure.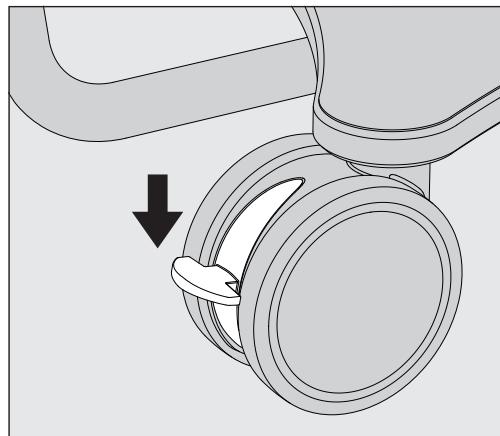

- Secure the wheels on the transfer trolley.

The load carrier can be pushed into the cleaning machine wash cabinet.

 **Incorrect weight distribution**

If the weight of the load is not correctly distributed, the transfer trolley may become unstable or a fault may occur during the transport procedure.

The load carriers must be distributed evenly over the loading surface.


Undocking the transfer trolley

- Turn the locking lever on the transfer trolley to the left .
- Release the wheels on the transfer trolley.

The transfer trolley can now be pulled away from the machine.


The transport lock for the load carrier is activated automatically as soon as the transfer trolley is pulled away from the machine.

Drain connection in the drip tray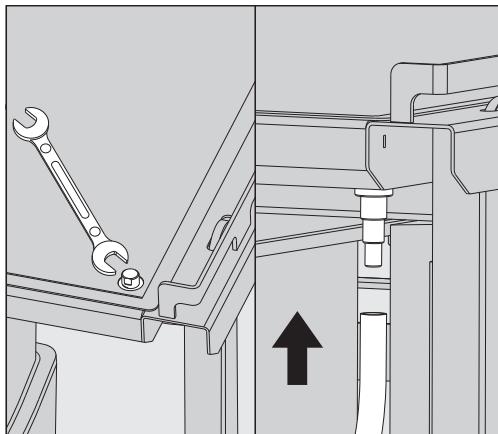

A drain connection is located on the right-hand side of the drip tray. There is a blind stopper closing off the drain connection when the equipment is in its as-delivered state.

The drain connection can be connected to a drip bottle using a hose.

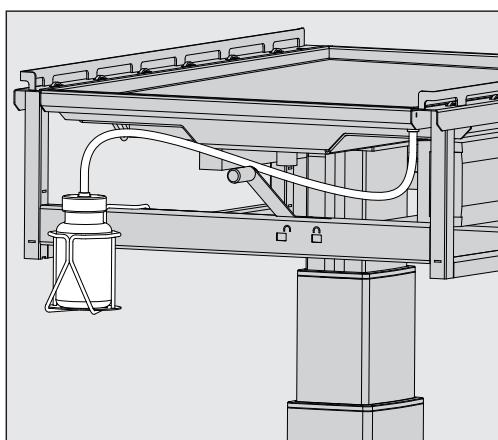It is possible to attach a holder for the drip bottle (accessory sold separately) to the strut under the drain connection.

Liquid from the drip tray can be conveyed from the drain connection into the drip bottle via the hose connection.

Rechargeable battery information

The height-adjustable transfer trolley is powered by a rechargeable battery which is charged by means of a charger. The system operator must ensure that the battery is charged before the transport procedure.

Charging it frequently will not harm it. Normally, fully charged batteries permit operation over several weeks. However, in order to prevent complete discharge of the battery, it is advisable to charge it on a regular basis. To guarantee full battery charging, the charger should remain connected for at least 12 hours. If necessary, the battery can also be charged on a daily basis.

Once charging is complete, the battery must be used correctly in the transfer trolley.

General

Rechargeable batteries are sealed lead-acid batteries similar to starter batteries in motor vehicles. In batteries of this kind, the electrolyte is linked to a fibreglass fleece. This allows the battery to be placed in any position. The battery has a limited service life. When it is spent, it must be disposed of at a battery disposal site in accordance with environmental protection laws.

 The battery compartment is equipped with ventilation that ensures the correct amount of air is supplied as required. The ventilation must not be blocked or covered.

Risk of explosion from excess pressure.

If the product has been exposed to excess mechanical load (from falling on the floor, a collision/pressure in the application or forceful lifting), it must be sent to Miele or a specialist company authorised by Miele so that the hermetic separation between the battery compartment and electronics compartment can be checked.

See the manufacturer's installation instructions for further information. (Linak: <https://www.linak.co.uk/products/lifting-columns/bl1/#/brochuresmanuals>)

Before use

To ensure the rechargeable battery works correctly and to extend its service life, it must be charged for at least 24 hours before it is used for the first time.

If a charged battery is not used for an extended period, it will gradually discharge. It is recommended that the battery is charged for at least a few hours before use.

Discharge

The battery capacity (Ah – ampere hours) is derived from the product of the current strength and discharge capacity at a specified voltage. The capacity largely depends on the strength of the discharge current. When the battery capacity has dropped to 50 %, the integrated electronics emit a signal. In order to extend the service life of the battery, it should always be recharged before this level is reached.

Charging

Only use the charger provided in the accessories to charge the battery:

- CHJ2
- COBO

Service life

The service life of rechargeable batteries is difficult to predict as it depends on the following parameters:

- Discharge depth

If the battery is deeply discharged between charge cycles, its service life will be reduced. It is therefore recommended that the battery is charged as often as possible.

- Strength of discharge current

The current strength influences the service life. This means that a battery with a maximum charge has a shorter service life than one that is less charged.

The battery should be replaced after no more than 4 years. To optimise its service life, the battery must be connected to the mains voltage as often as possible.

We recommend charging the battery every 6 months. Otherwise, it will discharge itself and have a lower capacity.

We recommend testing the battery function at least once a year.

Replacing the rechargeable battery

The rechargeable battery must be replaced with an original rechargeable battery from the manufacturer. This is available as a spare part.

The battery must be new or must be maintained by charging it at least once every 6 months.

 Regularly check that the ventilation opening is undamaged and intact.

The design of the ventilation line means that battery gases may escape. However, water cannot get into it.

- Make sure the rechargeable battery pack is correctly connected.

 The battery compartment is hermetically separated from the electronics compartment.

The material separating it must not be damaged or altered while the battery is being replaced. Battery gas may enter the electronics compartment, creating a risk of explosion.

- Press the snap lock on the rechargeable battery.

The rechargeable battery can be detached from the holder and the replaceable battery can be used.

- With the snap lock pressed down, insert the replaceable battery at a slight angle until it engages in the holder.

You can now resume operation.

Dispose of the rechargeable battery in a municipal recycling facility. Do not dispose of the rechargeable battery with your household waste.

 After the transfer trolley is shut down, the cleaning machine chamber walls may still be hot.

We recommend allowing the cleaning machine to cool down.

When the system is shut down, the maintenance and repair areas are not located in a hazard area. Never carry out maintenance and cleaning work while the system is in operation.

Cleaning and disinfecting the transfer trolley

 According to the cleaning and disinfection plan in the CSSD, contaminated areas of the transfer trolley must undergo cleaning and disinfection using a suitable cleaning and disinfecting agent. National recommendations relating to surface disinfectant listings and approvals, for example, must be observed.

 Take care when using process chemicals. Some agents may be corrosive and irritant.

The relevant safety regulations and safety data sheets issued by the process chemical manufacturers must be observed.

Wear gloves and protective goggles.

The variable transfer trolley must not be reprocessed by machine. Cleaning must be performed manually using a suitable detergent and disinfectant.

Any markings and warning labels are generally durable and resistant to cleaning processes, but should nevertheless be handled with care.


- Clean the transfer trolley using a moist cotton cloth.
- Use a detergent and disinfectant for more stubborn soiling. Follow national recommendations when doing this.

The use of high pressure cleaners or similar is not permitted.

Cleaning the drip tray

The drip tray can be removed to prevent liquid from collecting in it. It can be cleaned and disinfected if required.

- Remove the hose from the drain connection in the drip tray (if present).
- Remove the drip bottle (if present) from the holder and dispose of its contents carefully.

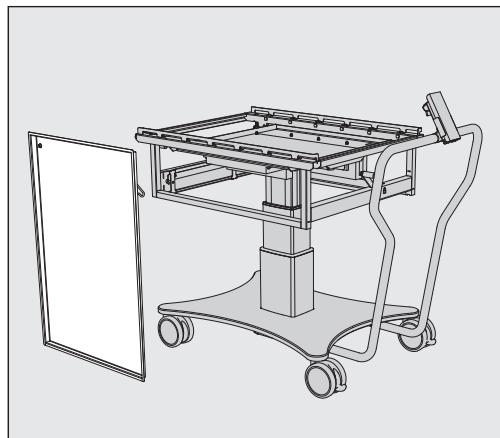- Remove the drip tray.

The drip tray can be reprocessed by machine in a suitable load carrier.

Checking the locking system

The transfer trolley has a locking system to prevent mobile units from accidentally tipping over. The system is not released until a transfer trolley is correctly docked onto the conveyor system or cleaning machine and locked in place.

The locking mechanism is at the front of the transfer trolley. If the transfer trolley is not docked, the conveyor system will not be able to push the mobile unit.

The locking system must be checked regularly:

- Push the transfer trolley against the cleaning machine. Check that the trolley and chamber rail have a common axis.

 Notify the Miele Customer Service Department if the locking function is not present. If this is the case, do not use the transfer trolley.

Maintenance

Maintenance of the system must only be carried out by the Miele Customer Service Department or by a Miele authorised service dealer.

⚠ Before beginning maintenance work, ensure that the transfer trolley is disconnected from the electricity supply and that it cannot be inadvertently switched on. Accident prevention regulations must be adhered to during all maintenance work.

An electrical safety check must be conducted at least every three years or in accordance with national test intervals.

The transfer trolley is designed to be low-maintenance. Neither the drive elements nor the bearings need to be lubricated or greased. Maintenance can be performed on the transfer trolley at the same time as the cleaning machine.

Scope of the maintenance:

- Visual check of the transfer trolley, including checking any accessories for damage
- Subjective check of ease of operation and operating noises
- Visual and functional check of the transfer trolley locking mechanism
- Alignment of the transfer trolley and check to ensure it is level with the disinfecter or conveyor system
- Daily check to ensure proper functioning of safety related locking mechanisms
- Check for fault-free operation when raising and lowering the height-adjustable transfer trolley
- Check to ensure stability of the height-adjustable transfer trolley in the lowest and highest setting

en - Technical data

Electrical data	
Max. current consumption	5.3 A
Voltage	24 V DC
Switch-on duration	10 % (2 minutes of continuous operation followed by an 18-minute pause)
Electrical data of battery module	
Voltage	24 V DC
Max. rated charge	2.9 Ah
Protection rating	IPX5
Ambient conditions	
Permissible (operating) temperature	5 °C to 40 °C
Permissible (operating) humidity	20 % to 80 % rH (non-condensing)
Permissible (storage) temperature	-10 °C to 50 °C
Permissible (storage) humidity	20 % to 80 % rH (non-condensing)
Maximum height above sea level	2000 m
Other data	
Weight	75 kg
Maximum load	180 kg
Dimensions	Length 1160 mm, width 665 mm, height 925 mm (without manual operation)
Emissions	
Sound pressure level	< 70 dB(A)

Disposing of old batteries

In many cases, electrical and electronic appliances contain batteries which must not be disposed of with household waste after use. You are legally obliged to remove any old batteries which are not securely enclosed by the transfer trolley and take them to a suitable collection point (e.g. retailer) where they can be handed in free of charge. Insulate the metal contacts with adhesive tape to prevent short circuits. Batteries may contain substances which can be hazardous to human health and the environment.

The labelling on the battery will provide further information. The bin symbol with a cross through it means that the batteries must not be disposed of in household waste. A bin with a cross through it which also has one or more chemical symbols on it indicates that the battery contains lead (Pb), cadmium (Cd) and/or mercury (Hg).

Old batteries contain valuable raw materials which can be recycled. Disposing of batteries separately makes them easier to handle and recycle.

Disposal of your old machine

Please contact Miele to arrange for the technical decommissioning of the conveyor system before disposing of it in accordance with regulations.

Electrical and electronic appliances often contain valuable materials. They also contain materials which, if handled or disposed of incorrectly, could be potentially hazardous to human health and to the environment. They are, however, essential for the correct functioning of your appliance. Please do not therefore dispose of it with your household waste.

Please dispose of it at your local community waste collection / recycling centre or contact your Dealer for advice.

Please ensure that your old appliance poses no risk to children while being stored prior to disposal.

en - Enclosure

General

Filing location of training certificate:

Filing period: _____ years

Training record

Operator	
Training location	
Scope of training	<input type="checkbox"/> Initial training <input type="checkbox"/> Repeat of training due to _____
Training instructor (medical product advisor, applies only to DE and AT)	Name, date and signature
Date, time and duration of training	
Date and time of first application by user (planned)	

Appliance details

Manufacturer	Miele Bürmoos plant	Serial no.	
Type		Year of manufacture	
Accessories		Software version	
Process		Cycle counter	
Location		Inventory no.	
Operating instructions and other documents	Mat. no.:	Appliance status (safety checks, condition of accessories, process validation, etc.)	

Special (optional) specifications for user training

Process documents created by supervisor, including description of internal quality standards as a basis for user training. Appliance integrated into customer's hygiene process.	
--	--

General information:

Training is given to users and technicians together. Following the user training, a special section is conducted for technicians. Technicians are asked not to put forward any technical questions during the user training.

Scope of training

- Guidance on areas of application of the appliances (functional description)
- Safety notes and warnings according to operating instructions
- Docking system
- Operation of user interface (control elements)
- Appliance operation
- Shutdown
- Emergency stop button (how it works and risk avoidance)
- Checks, cleaning and routine checks
- Service and maintenance
- Machine log book
- Procedure in the event of faults, safety procedures and first aid measures
- Information on correct loading and unloading
- Information on using unsuitable load carriers
- Information on the risk of overloading
- Information on escape routes (keeping clear and labelling/describing)
- Transfer of documentation

Participants

The following personnel confirm by their signature that they have received and understood the above training with all its subject matter and content. In addition to appropriate handling, the dangers associated with operational activities and with Miele products to people, the environment and material goods, the relevant safety and protective equipment and necessary behaviour in the event of danger were also addressed.

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Miele Werk Bürmoos GmbH