APWD 646

de	Gebrauchsanweisung Injektormodul
fr	Mode d'emploi Module à injection
it	Istruzioni d'uso Modulo iniettore
da	Brugsanvisning Injektormodul
fi	Käyttöohje Suorasuihkumoduuli
nl	Gebruiksaanwijzing injectormodule
no	Bruksanvisning injektormodul
en	Operating instructions Injector module
sv	Bruksanvisning injektormodul

de	4
fr	16
it	28
da	40
fi	52
nl	64
no	76
en	88
sv	100

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Notes about these instructions	88
Queries and technical problems	88
Definition of terms	88
Intended use	89
Items supplied	90
Scope of delivery APWD 646	90
Disposal of the packing material	90
Optional components	91
Warnings and safety notes	92
Areas of application	93
Reprocessing hollow instruments.....	93
Screwing in nozzles.....	93
Securing injector nozzles	93
Preparing the load	93
To check before loading the machine and before starting a programme	94
A 803 holder for transmission instruments	94
Installation	94
Reprocessing transmission instruments.....	95
AUF 1 holder for transmission instruments	95
Optional accessories.....	95
Installation	95
Reprocessing transmission instruments.....	96
Replacing filter plates.....	96
A 814 holder for the tips of air scalers and piezo scalers	97
Fitting E 499 for suction and rinsing cannulae	98
Irrigation sleeves	98

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property.

Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow buttons and save your choice with *OK*.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

Queries and technical problems

In the event of queries or technical problems, please contact Miele. Contact details can be found at the end of the operating instructions for your cleaning machine or at www.miele.com/professional.

Definition of terms

The washer-disinfector is generally referred to as "the machine" in these operating instructions. Reprocessable medical devices are referred to as "load items" if they are not more closely defined.

This module can be used to reprocess machine-reprocessable medical devices in special Miele washer-disinfectors. Follow the operating instructions for your washer-disinfector as well as the instructions provided by the medical device manufacturer on how to reprocess their items by machine.

The APWD 646 injector module is suitable for reprocessing dental and ENT instruments by machine.

The module has 2 injector rails with a total of 22 holders for nozzles and other irrigation connectors for reprocessing lumened instruments.

The module can be used in an APWD 217 mobile unit.

Disposable items must not be reprocessed.

Scope of delivery APWD 646

- APWD 646 injector module with 2 nozzle manifolds and a total of 22 connections for irrigation connectors
- Height 84 mm, width 125 mm, depth 523 mm
- Injector manifold vertical clearance in combination with the APWD 217 mobile unit: 195 mm
- A 838, tool set, consisting of 1 x WAF 9 spanner and 1 x T20 offset screwdriver

Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in land-fill sites.

Optional components

- A 800, reusable tubular filter with a filtration grade of ≥ 0.07 mm, 405 mm long, \varnothing 22.5 mm
- A 801, threaded adapter
- A 803, holder for dental transmission instruments, height 76.5 mm, width 40 mm, depth 32 mm
- A 813, two-piece adapter with 4 screw threads for various irrigation connectors, height approx. 48 mm, \varnothing 45 mm, 4 nozzles with silicone hoses and 3 blind stoppers
- A 814, holder with screw cap for reprocessing the tips of air scalers and piezo scalers, height approx. 41 mm, \varnothing 16 mm
- A 836, nozzle set for e.g. dental applications, consisting of 22 x A 833, injector nozzle with aperture on the side, length 110 mm, \varnothing 4 mm
- A 837, irrigation sleeve set for e.g. ENT applications, consisting of 22 x A 817, irrigation sleeve for instruments \varnothing 4–8 mm, length 67 mm, \varnothing 11 mm, cap with aperture \varnothing 6 mm
- AUF 1, two-piece adapter for transmission instruments (dental hand-pieces, for example), height approx. 48 mm, \varnothing 45 mm; silicone adapter (ADS) required for reprocessing
- E 336, irrigation sleeve, length 121 mm, \varnothing 11 mm
- E 442, irrigation sleeves for instruments \varnothing 4 - 8 mm, 121 mm long, \varnothing 11 mm, cap with aperture \varnothing 6 mm
- E 443, irrigation sleeves for instruments \varnothing 8 - 8.5 mm, 121 mm long, \varnothing 11 mm, cap with aperture \varnothing 10 mm
- E 445, 12 caps for irrigation sleeves, aperture \varnothing 6 mm
- E 446, 12 caps for irrigation sleeves, aperture \varnothing 10 mm
- E 447, Luer Lock Adapter ♀ for Luer Lock ♂
- E 499 for reprocessing suction and rinsing cannulae with a variety of connector types as well as Veress cannulae.
- FP 20, 20 filter plates, \varnothing 30 mm

Further components are available from Miele as optional extras.

To avoid the risk of accidents and damage to this module please read these instructions carefully before using it for the first time. Keep these instructions in a safe place where they are accessible to users at all times.

 Please also read the operating instructions for your dishwasher or the washer-disinfector and pay particular attention to the Warning and Safety instructions.

- The module is approved solely for the applications specified in the "Areas of application" chapter of these operating instructions. Components such as nozzles may only be replaced with Miele components or genuine original spare parts.
- New load carriers must be cleaned in the washer-disinfector without a load prior to first use.
- Inspect all mobile units, baskets, modules and inserts daily as described in the "Maintenance" section in the Operating instructions for the washer-disinfector.
- For your own safety, wear gloves and, if necessary, a surgical mask and protective goggles when sorting the instruments and unloading the machine. To avoid recontamination, wear clean gloves when unloading the machine.
- To clean and disinfect lumened instruments thoroughly from the inside, they must be attached to the relevant holders or nozzles on the injector manifolds. The connectors must not come loose during a programme sequence.
At the end of every programme sequence, check that the connectors have not come loose.
- Any hollow items that have become disconnected from their injector nozzles, adapters or holders during processing must be re-processed.
- The last rinse, as a minimum, must be carried out with DI water.
- Only items which have been declared by their manufacturer as suitable for machine reprocessing may be processed. The manufacturer's specific reprocessing instructions must be observed.
Disposable instruments must not be put into the machine for processing.
- The reprocessing programme used should include thermal disinfection, and the chemical agents used must be suitable for the material quality of the wash load.

Miele cannot be held liable for damage or injury caused by non-compliance with the warning and safety notes.

Reprocessing hollow instruments

Before inserting lumened instruments, please make sure that all hollow sections are free of blockages. Otherwise the suds solution will not be able to access all areas properly.

- Attach the lumened instruments to the nozzles.
- At the end of the programme, the insides of the instruments must be dried with sterile compressed air before being cleaned and sterilised as appropriate in accordance with the manufacturer's instructions. Please observe national regulations.


Where necessary, the nozzles supplied can be replaced with different fittings, e.g. holders for dental transmission instruments, irrigation sleeves, etc. Further examples are given on the following pages.

⚠ The wash water used for reprocessing narrow lumen instruments needs to be filtered. The injector manifolds can each be fitted with an A 800 tubular filter for this purpose. The tubular filter is supplied with its own operating instructions.

The operating instructions for the loading carriage describe how to insert and remove the modules.

Screwing in nozzles

Securing injector nozzles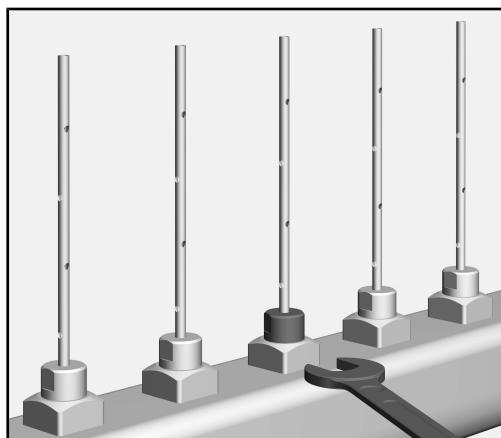

- Screw the injector nozzles into the required positions.
- Use the 9 mm spanner (WAF 9) to tighten the injector nozzles.

Preparing the load

For validation purposes please make sure that you follow the loading instructions given on the template.

⚠ Risk of injury caused by load.

There is a risk of injury when loading and unloading due to possible sharp edges, rims or pointed ends.

To minimise the risk of injury, loading should take place from the back forwards and unloading should take place in the reverse order.

en - Areas of application

 Risk of infection due to insufficient disinfection.

Load items that become detached from the irrigation connector during reprocessing will not be sufficiently disinfected on the inside. Any items that become detached from the irrigation connector during reprocessing must be reprocessed again.

To check before loading the machine and before starting a programme


- Check that fittings such as irrigation sleeves and injector nozzles are securely screwed into position.

 Make sure that all screw connectors are fitted with injector nozzles, adapters, irrigation sleeves or blind stoppers to ensure that all fittings in use are supplied with sufficient standardised pressure. Damaged fittings such as injector nozzles, adapters and irrigation sleeves must not be used.

Fittings not equipped with wash items do not have to be replaced blind stoppers.

- Check that spray arms are rotating freely.
- Check spray arm jets for blockages.

Cleaning the A 800 tubular filter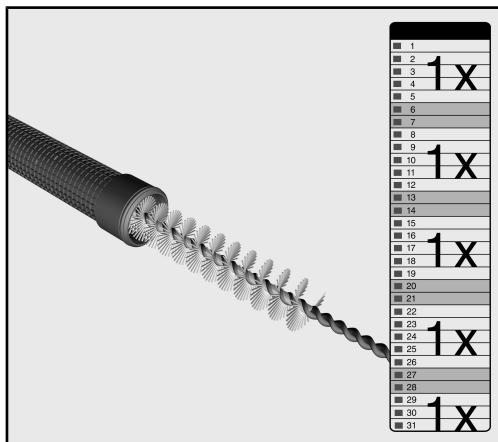

The tubular filter must be cleaned **at least once a week**.

We recommend cleaning the tubular filter after every 10 programme sequences.

- Follow the cleaning instructions in the operating instructions for the tubular filter.

A 803 holder for transmission instruments

A 803 holders must be retro-fitted for reprocessing dental transmission instruments. The holders are suitable for almost all standard transmission instruments.


Installation

- Remove a nozzle from the injector manifold and replace it with an A 803 holder.

Reprocessing transmission instruments

When the A 803 holders are used, the wash water must be filtered. The injector manifold must be fitted with an A 800 tubular filter. The tubular filter is supplied with its own operating instructions. Follow in particular the instructions on how to perform regular cleaning on the tubular filter.

- Check that internal sections of transmission instruments are free of blockages.

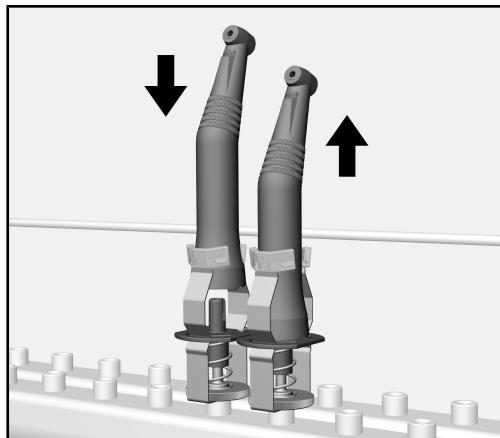- Insert the transmission instrument into the holder. The instrument is then held in position by the spring clamp.
- After reprocessing pull the transmission instrument out of its holder. The spring clamp will automatically spring back to its starting position.
- After reprocessing, the insides of the transmission instruments must be dried with sterile compressed air before being cleaned and sterilised as appropriate in accordance with the manufacturer's instructions. Please observe national regulations.

Before using transmission instruments again following reprocessing, a function check must be carried out, e.g. by spraying into a basin, to ensure they are clear.

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AUF 1 holders are designed for reprocessing dental transmission instruments.


Optional accessories

- A 801, threaded adapter
- ADS 1, silicone adapter, white, for holding transmission instruments with a Ø of approx. 20 mm
- ADS 2, silicone adapter, green, for holding transmission instruments with a Ø of approx. 16 mm
- ADS 3, silicone adapter, red, for holding transmission instruments with a Ø of approx. 22 mm
- FP 20, 20 filter plates, porosity 2, Ø 30 mm

Installation

An A 801 adapter is required.

en - Areas of application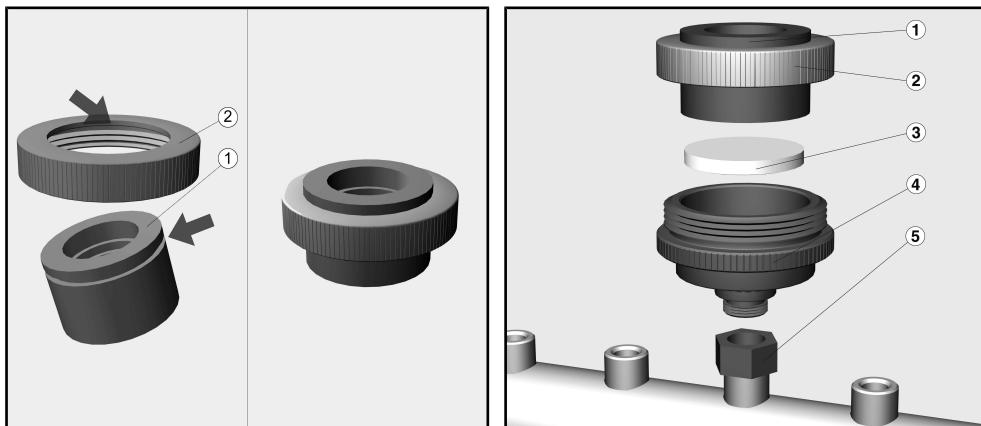

- Unscrew the upper part (2) from the AUF 1 holder.
- Fit a silicone adapter (ADS) (1) in its place which has the same diameter as the transmission instrument being reprocessed.

⚠ Fitting a transmission instrument into a silicone adapter that is too big can cause damage to the filter plate.

- Remove an injector nozzle from the injector manifold and replace it with the A 801 adapter (5).
 - Screw the lower part of the holder (4) into the adapter.
 - Insert a filter plate (3) into the lower part.
- ⚠** Do not insert a filter plate into the holder if the injector manifold has been fitted with an A 800 tubular filter.
- Screw the upper part into the lower part.

Reprocessing transmission instruments

Before loading check that the silicone adapters are present and correctly fitted.


- Wipe away any deposits from the area around the silicone adapters that may be present depending on local water conditions using a damp cloth.
- Check that internal sections of transmission instruments are free of blockages.
- Fit the instruments into their holders using the respective silicone adapters.
- After reprocessing, the insides of the transmission instruments must be dried with sterile compressed air before being cleaned and sterilised as appropriate in accordance with the manufacturer's instructions. Please observe national regulations.

Before using transmission instruments again following reprocessing, a function check must be carried out, e.g. by spraying into a basin, to ensure they are clear.

Replacing filter plates

⚠ Filter plates must be replaced **approx. every 2 weeks** or after **20 reprocessing cycles** have been completed.

- Unscrew the holder completely.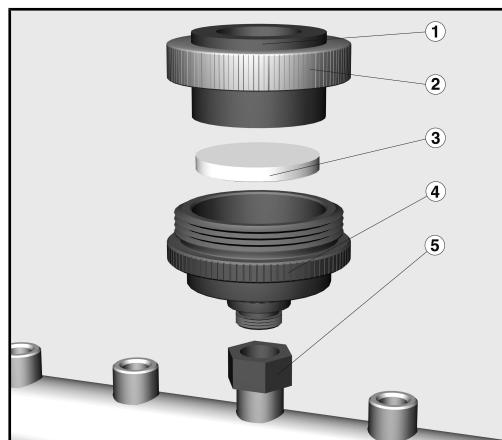


- Dismantle the holder and dispose of the filter plate ③.
- Rinse out the lower part ④ thoroughly.
- Insert a new filter plate.


⚠ Damaged or used filter plates must not be inserted under any circumstances.

- Screw the upper and lower parts of the holder back together and fit the holder back onto the injector manifold.

A 814 holder for the tips of air scalers and piezo scalers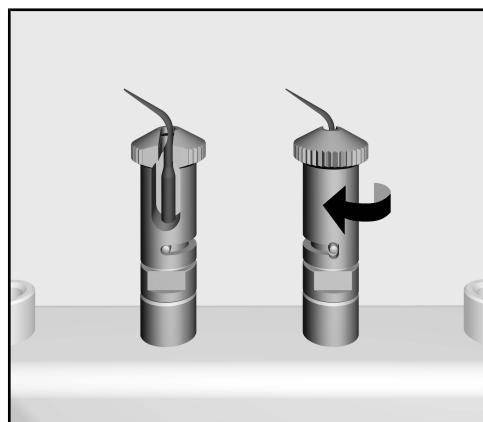

A 814 holder

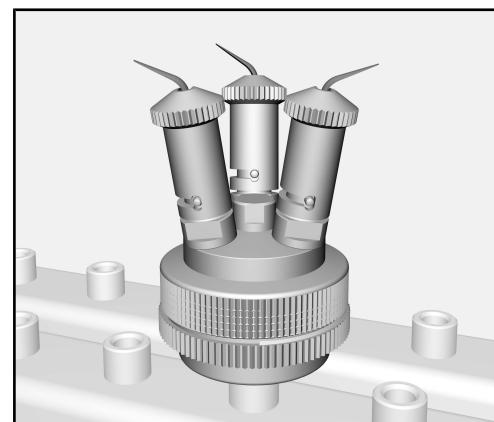A 814 holder with A 813 adapter

The A 814 holder enables the tips of air scalers and piezo scalers to be reprocessed. To increase capacity, the holder can be combined with the A 813 adapter. The holder and the adapter each come with their own operating instructions.

Fitting E 499 for suction and rinsing cannulae

The E 499 is for reprocessing rinsing and suction cannulae, including suction breakers. It comes with its own instructions.

Irrigation sleeves

Irrigation sleeves are suitable for reprocessing hollow instruments and instruments with the following external diameters:

- A 817, irrigation sleeve, length 65 mm, Ø 11 mm, cap with opening Ø 6 mm
- E 336, irrigation sleeve, length 121 mm, Ø 11 mm
- E 442, irrigation sleeves for instruments Ø 4 - 8 mm, 121 mm long, Ø 11 mm, cap with aperture Ø 6 mm
- E 443, irrigation sleeves for instruments Ø 8 - 8.5 mm, 121 mm long, Ø 11 mm, cap with aperture Ø 10 mm

The aperture can be varied using optional replacement caps:

- A 818, 12 caps for irrigation sleeves, opening Ø 2 mm
- E 445, 12 caps for irrigation sleeves, aperture Ø 6 mm
- E 446, 12 caps for irrigation sleeves, aperture Ø 10 mm

The diameter of the instruments and the irrigation sleeve apertures or caps must be compatible with each other, e.g. caps for irrigation sleeves with an aperture diameter of Ø 6 mm for dental surgical suction cannulae.

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