A 500

de	Gebrauchsanweisung Wagen
cs	Návod k obsluze vozíku
da	Brugsanvisning Vogn
el	Οδηγίες χρήσης ειδικό συρόμενο βαγονέτο με εγχυτήρες
en	Operating instructions: Mobile unit
es	Instrucciones de manejo Carros
fi	Käyttöohje Vaunu
fr	Mode d'emploi Chariot
it	Istruzioni d'uso Carrelli
ja	取扱説明書 モバイルユニット
ko	사용 설명서: 모바일 장치
nl	Gebruiksaanwijzing Wagen
no	Bruksanvisning vogn
pl	Instrukcja użytkowania Wózek
pt	Instruções de utilização Carro
ru	Инструкция по эксплуатации и гарантия качества Тележка
sk	Návod na použitie vozíka
sv	Bruksanvisning Vagn
tr	Kullanım kılavuzu Mobil ünite
uk	Інструкція з експлуатації Візок
zh	操作说明书：移动装置

de	4
cs	13
da	22
el	31
en	40
es	49
fi	58
fr	67
it	76
ja	85
ko	94
nl	103
no	112
pl	121
pt	130
ru	139
sk	149
sv	158
tr	167
uk	176
zh	185

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Notes about these instructions	40
Intended use	41
Queries and technical problems	41
Items supplied	42
Load carriers.....	42
Disposal of the packing material	42
Optional accessories.....	43
Warnings and safety notes	44
Medical devices.....	44
Laboratory glassware and utensils.....	44
Areas of application.....	45
To check before loading the machine and before starting a programme	45
Programme selection.....	45
Installation.....	46
Screwing the A 603 level onto the mobile unit	46

Important warnings

⚠ Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property.
Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow buttons and save your choice with *OK*.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings | menu.

This mobile unit can be used to reprocess machine-reprocessible medical devices, laboratory glassware and utensils in a Miele washer-disinfector. Follow the operating instructions for your washer-disinfector as well as the instructions of the medical device or glassware and utensil manufacturer on how to reprocess their items by machine.

The A 500 mobile unit can be loaded with inserts for baby bottles, inserts for supply and disposal containers or a variety of mesh trays and inserts for reprocessible medical devices, laboratory glassware and utensils.

The mobile unit can be used in the following washer-disinfectors:


- PLW 8615
- PLW 8616
- PLW 8617
- PWD 8626
- PWD 8628

The washer-disinfector is generally referred to as “the machine” in these operating instructions. Reprocessible medical devices, laboratory glassware and utensils are referred to as “items” if they are not more closely defined.

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your dishwasher or the washer-disinfector or go to www.miele-professional.com.

Load carriers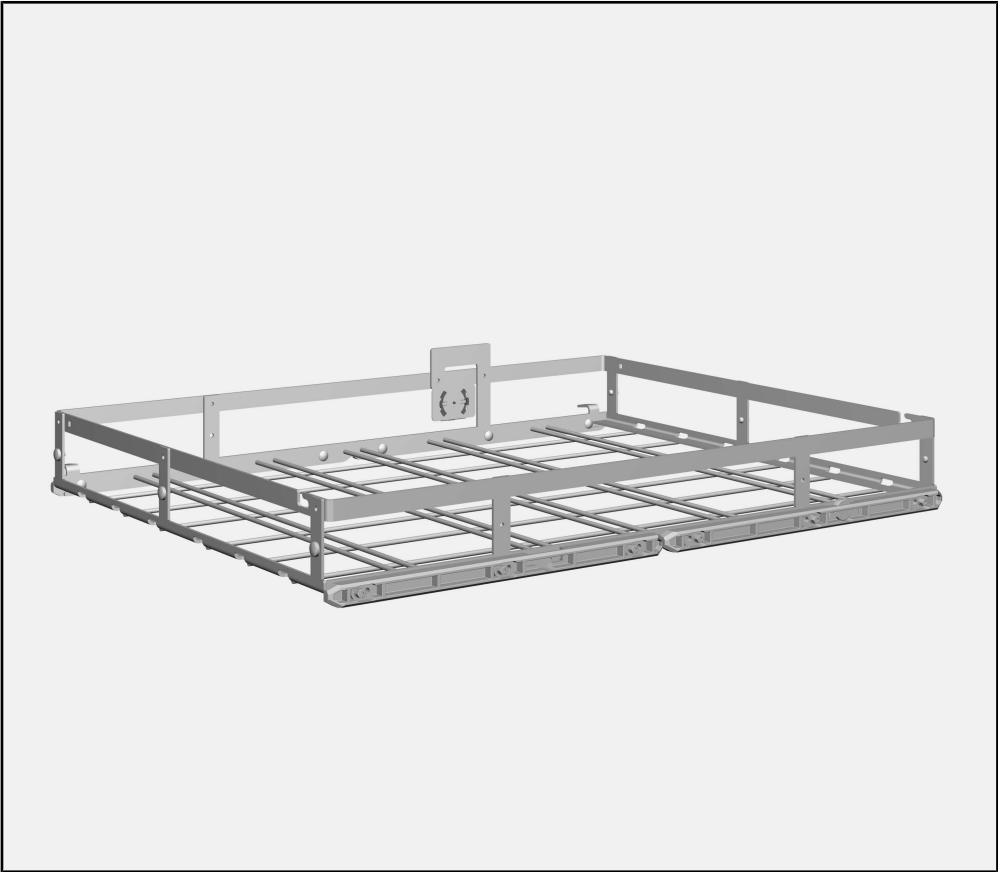

Mobile unit A 500, height 153 mm, width 650 mm, depth 790 mm
Mobile unit A 500 plus A 603 level, height 426 mm, width 650 mm, depth 790 mm

A 500 *	Loading dimensions [height x width x depth in mm]
Level 1	670 x 600 x 775
+ A 603: level 1	305 x 600 x 775
Level 2	302 x 595 x 770

* Levels are numbered from the bottom up

Disposal of the
packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled. Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in land-fill sites.

Optional accessories

The following and other accessories are available to order from Miele:

- Module A 603, attachable level with spray arm, height 413 mm, width 784 mm, depth 611 mm
- Module A 615 for 6 shallow DIN containers and their lids, height 441 mm, width 597 mm, depth 771 mm
- Module A 616 for 4 tall DIN containers and their lids, height 441 mm, width 597 mm, depth 771 mm
- Module A 617 for 4 ISO containers and their lids, height 436 mm, width 597 mm, depth 793 mm

en - Warnings and safety notes

To avoid the risk of accidents and damage to this mobile unit please read these instructions carefully before using it for the first time. Keep these instructions in a safe place where they are accessible to users at all times.

 Please also read the operating instructions for your dishwasher or the washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ The mobile unit is approved solely for the applications specified in the “Areas of application” chapter of these operating instructions. Applications that go beyond those listed and use of the mobile unit with additional components from the Miele product range are described in the operating instructions for the components or are to be agreed with Miele.
- ▶ New mobile units and baskets must be washed in the washer-disinfector without a wash load prior to first use.
- ▶ Inspect all mobile units, baskets, modules and inserts daily as described in the “Maintenance” section in the Operating instructions for the washer-disinfector.
- ▶ Broken glass and ceramics can result in serious injury when loading or unloading. Damaged glass or ceramic load items must not be reprocessed in the machine.

Medical devices

- ▶ The reprocessing programme used should include thermal disinfection, and the process chemicals used must be suitable for the material quality of the load items.
- ▶ The last rinse, as a minimum, must be carried out with DI water.
- ▶ Visual checks should be undertaken to check the cleaning result. Additional protein analysis checks should also be carried out e.g. weekly, on a random sampling basis.

Laboratory glassware and utensils

- ▶ The reprocessing result must be checked as necessary using a suitable test method and not just by a visual check.

Miele cannot be held liable for damage or injury caused by non-compliance with the warning and safety notes.

For validation purposes please make sure that you follow the loading instructions given on the template.

To check before loading the machine and before starting a programme

- Check that the mobile unit is correctly docked to the water path in the chamber.

Programme selection

When selecting a programme manually on the machine, the relevant mobile unit must be selected first. Pictograms representing the mobile units are shown on the display.

The pictogram of the mobile unit without inserted modules can be found on a bar on the front of the mobile unit.


Depending on the modules inserted, several pictograms are displayed for the A 500 mobile unit.

Once you have selected a pictogram, the list of assigned reprocessing programmes is displayed.


A 500 + module	Pictogram
A 500	
A 500 + A 603	
A 500 + A 615 A 500 + A 616 A 500 + A 617	

Screwing the A 603 level onto the mobile unit


Using the A 603 module, the A 500 mobile unit can be equipped with a level with spray arm.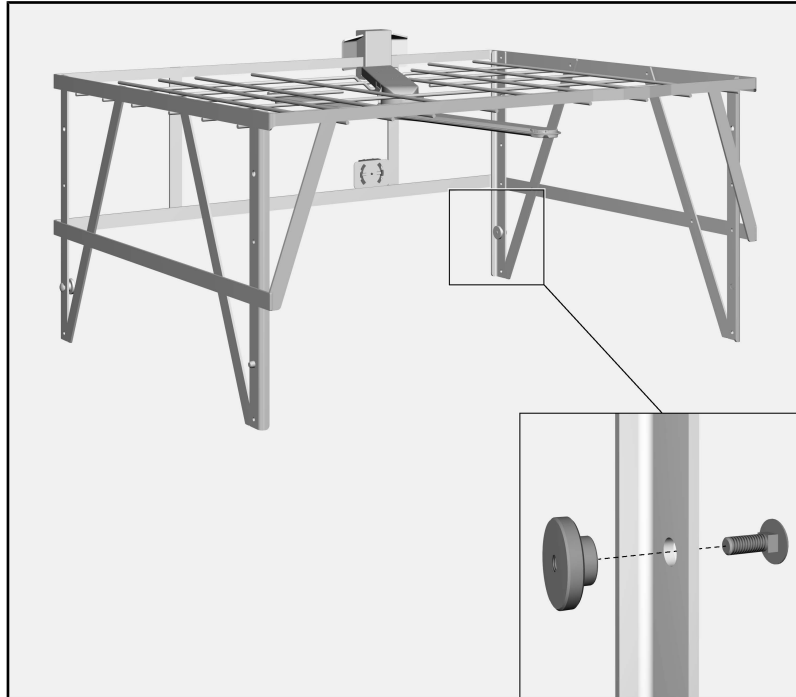

- Unscrew the right and left knurled nuts on the module and remove the screws.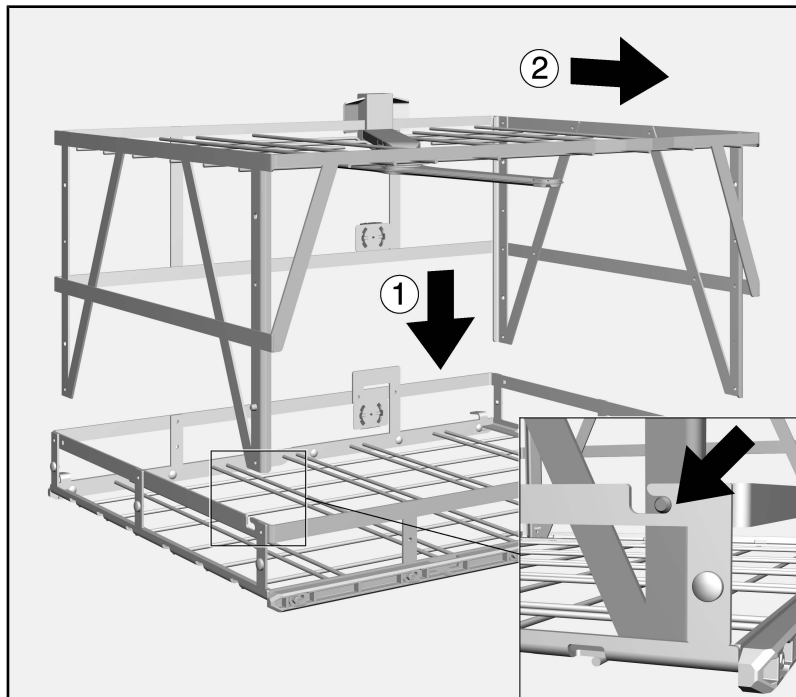

- Place the module on the mobile unit ① so that the pins on the module's front bars engage in the recesses of the mobile unit.
- Pull the module forward to secure the pins in the mobile unit guide ②.

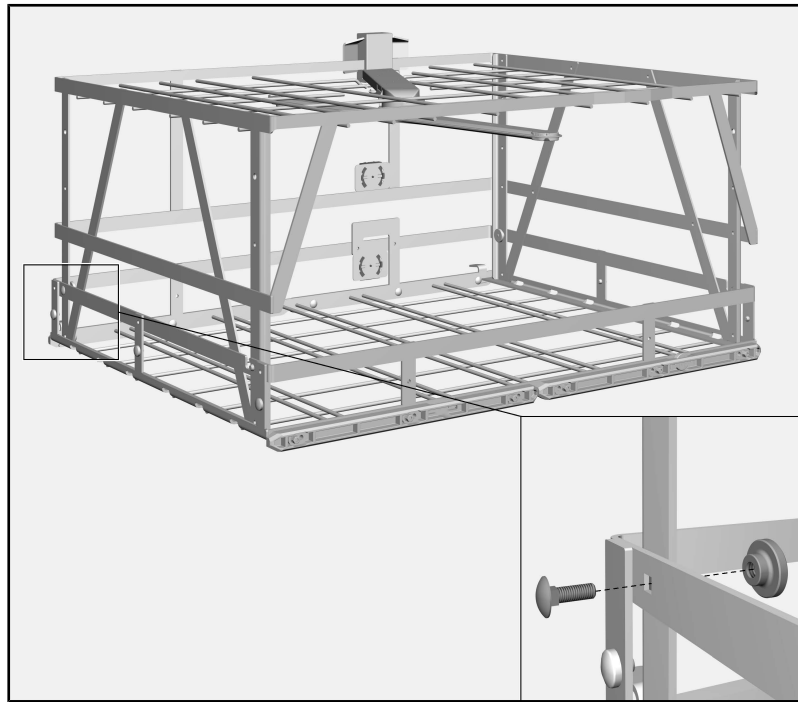- From the outside, insert the right and left screws through the bars of the mobile unit and the module and tighten them with the knurled nuts.

To unscrew the module from the mobile unit, follow these instructions in reverse order.

⚠ Risk of injury due to falling mobile unit.

The screw connection between the module and mobile unit may come loose.

Always hold the bars on the mobile unit when carrying the mobile unit and the module together.

Always use one of the bars on the mobile unit to pull or push the mobile unit and module together.

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