E 337/1 & E 802/1

de	Gebrauchsanweisung Einsätze für Dentalinstrumente
da	Brugsanvisning Indsatser til tandlægeinstrumenter
es	Instrucciones de manejo Complementos para instrumental odontológico
fi	Käyttöohje Telineet hammaslääkärin instrumenteille
fr	Mode d'emploi Compléments pour instruments dentaires
en	Operating instructions Inserts for dental instruments
hr	Upute za uporabu Umetci za stomatološke instrumente
it	Istruzioni d'uso Inserti per strumenti odontoiatrici
nl	Gebruiksaanwijzing Inzetten voor tandheelkundige instrumenten
no	Bruksanvisning innsatser til dentalinstrumenter
pt	Instruções de utilização Complementos para instrumentos da área dental
sv	Bruksanvisning Insatser för dentalinstrument
sl	Navodila za uporabo vložkov za zobozdravstvene instrumente

de	4
da	13
es	22
fi	31
fr	40
en	49
hr	58
it	67
nl	76
no	85
pt	94
sv	103
sl	112

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en - Contents

Notes about these instructions	49
Queries and technical problems	49
Intended use	50
Items supplied	51
Scope of delivery E 337/1.....	51
Optional accessories	52
Warnings and safety notes	53
Areas of application.....	54
Pre-treating of instruments.....	54
Sorting of instruments.....	54
Mouth specula	55
Reprocessing.....	56
To check before loading the machine and before starting a programme	56

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow buttons and save your choice with *OK*.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

Queries and technical problems

In the event of queries or technical problems, please contact Miele. Contact details can be found at the end of the operating instructions for your cleaning machine or at www.miele.com/professional.

The E 337/1 and E 802/1 inserts can be used for reprocessing reprocessible medical instruments used in various applications (e.g. dentistry, podiatry) in special Miele washer-disinfectors. Follow the operating instructions for your washer-disinfector as well as the instructions issued by the medical device manufacturer on how to reprocess their items using a machine.

Before reprocessing can commence, the instruments must be sorted and placed inside the receptacles and compartments of the inserts and the inserts must be placed into an upper or lower basket.

The E 337/1 features the following receptacles:

- 4 plastic receptacles for instruments that are sharp at both ends
- 8 compartments, each approx. 47 x 28 mm
- 4 compartments, each approx. 30 x 28 mm
- 8 compartments, each approx. 22 x 28 mm
- 48 compartments, each approx. 13 x 13 mm, for e.g. mouth mirrors

The E 802/1 has the following receptacles:


- 3 plastic receptacles for instruments that are sharp at both ends
- 8 compartments, each approx. 30 x 28 mm
- 4 compartments, each approx. 16 x 28 mm
- 15 compartments, each approx. 14 x 20 mm, for e.g. mouth mirrors

The E 337/1 and E 802/1 inserts are not suitable for reprocessing transmission instruments (dental handpieces, for example). Only irrigation connectors specifically designed for the reprocessing of transmission instruments should be used for this purpose, e.g. the A 803 or the AUF 1.


Disposable items must not be reprocessed.

The washer-disinfector is generally referred to as “the machine” in these operating instructions. Reprocessible medical devices are referred to as “items” if they are not more closely defined.

Scope of delivery E 337/1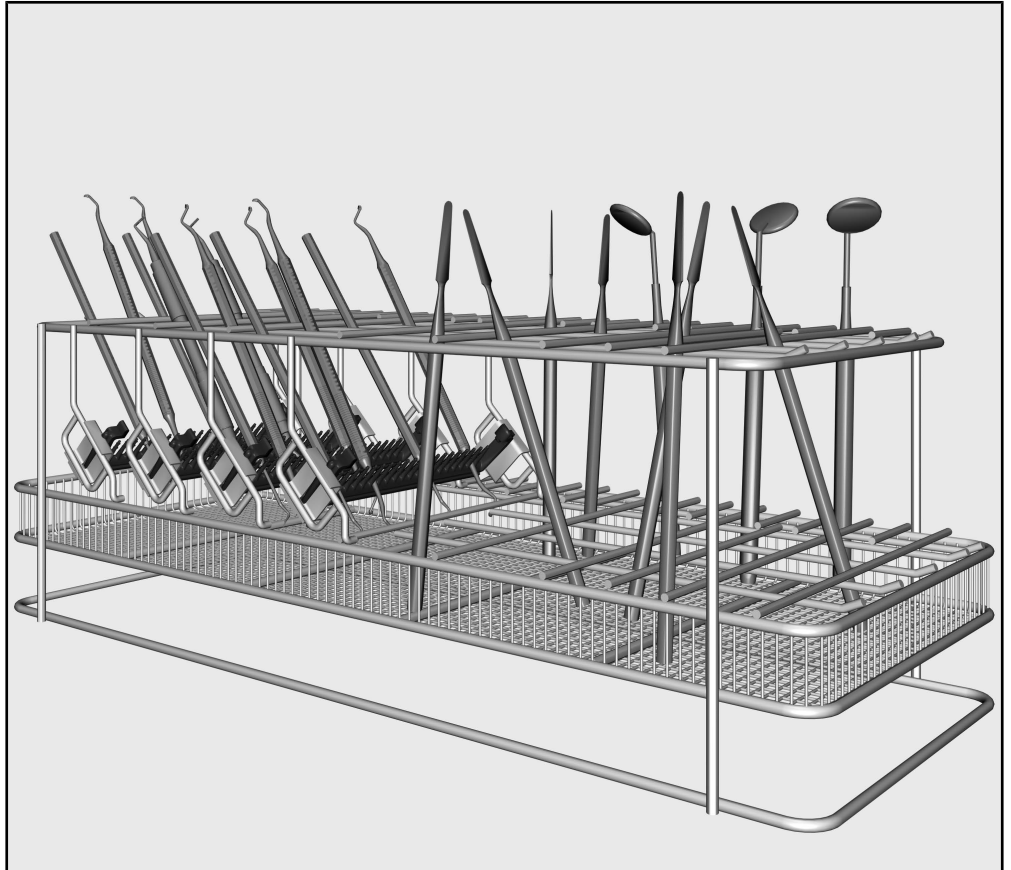

- E 337/1 insert, height 149 mm, width 173 mm, depth 445 mm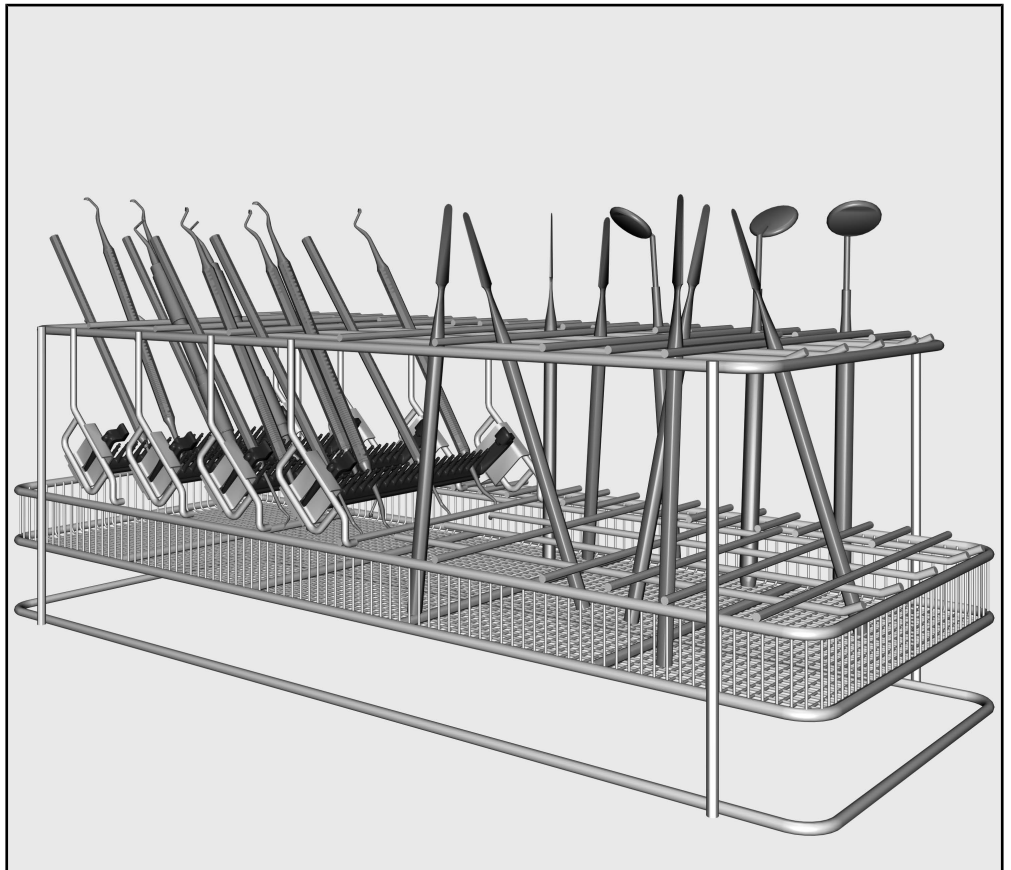

en - Items supplied

Illustrated: E 337/1

- E 337/1 insert, height 149 mm, width 173 mm, depth 445 mm
- E 802/1 insert, height 149 mm, width 163 mm, depth 295 mm

Optional accessories

- A 830, insert for working ends of separable mouth mirrors, 3 compartments Ø 8 mm, 14 compartments 10 x 19 mm, 2 compartments 8 x 19 mm, can only be used in conjunction with the E 337/1 insert

Other optional accessories are available to order from Miele.

Read the operating instructions carefully before using this insert.
Keep these operating instructions in a safe place.


 Please also read the operating instructions for your dishwasher or the washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ The inserts are only approved for use in the applications stated in the operating instructions. Applications that go beyond those listed and use of the inserts with additional components from the Miele range of components are described in the operating instructions of the components or are to be agreed with Miele. Conversions, modifications and any other use are not permitted.
- ▶ New load carriers must be cleaned in the washer-disinfector without a load prior to first use.
- ▶ Inspect all mobile units, baskets, modules and inserts as described in the “Maintenance” section of the operating instructions for your machine.
- ▶ For your own safety, wear gloves and, if necessary, a surgical mask and protective goggles when sorting the instruments and unloading the machine. To avoid recontamination, wear clean gloves when unloading the machine.
- ▶ Load from back to front and unload from front to back.
- ▶ The last rinse, as a minimum, must be carried out with DI water.

Pre-treating of instruments

- Any significant deposits such as dental cement, composite, polishing paste or similar left behind after treatment must be removed immediately after examination of the patient so that little trace of them remains.
- Instruments with complex functional ends or very stubborn deposits may require ultrasonic pre-treatment.

Sorting of instruments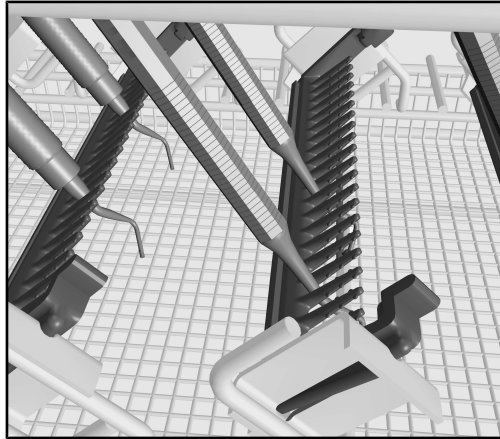


- Instruments that are sharp at both ends should be placed into the plastic receptacles. Instruments that are sharp at one end can also be placed into the receptacles.
- Make sure that the functional ends of the instruments are not resting on the frame of the insert.
- Spread out the rest of the instruments in the empty compartments of the insert. Make sure that the functional ends of the instruments do not touch each other during reprocessing inside the washer-disinfector.

Mouth specula

⚠ Not all mouth specula with a glass mirror in them are suitable for machine reprocessing. Follow the manufacturer's instructions.


Rhodium-coated mouth specula, because of their delicate surface, must be loaded in such a way that the mirror surfaces cannot sustain mechanical damage during reprocessing, e.g. by knocking against other instruments.

- Place the mouth specula individually into the 13 x 13 mm (E 337/1) or 14 x 20 mm (E 802/1) compartments (one speculum in each compartment).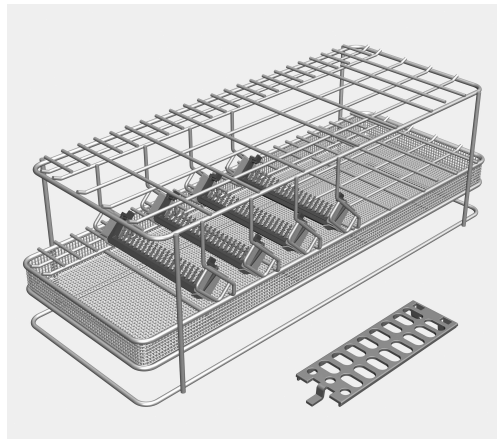


E 337/1 insert and A 830/1 insert


The A 830/1 insert (optional accessory) must be used when reprocessing separable mouth mirrors. The insert is supplied with its own set of operating instructions.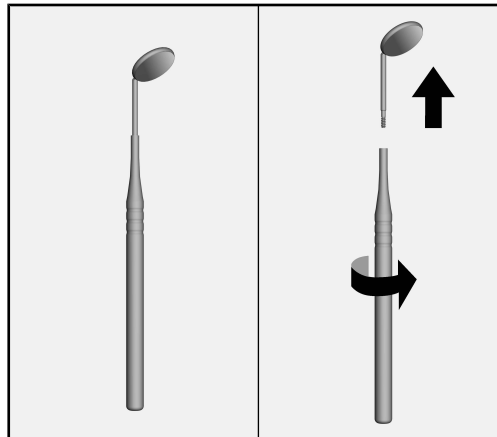

- Unscrew the work ends with the mirror surfaces before reprocessing.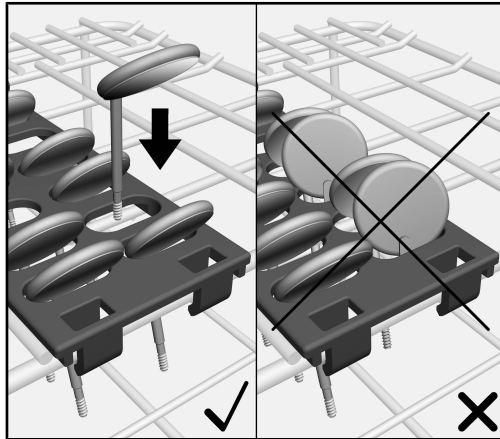

- Place these inside the receptacles of the A 830/1. Make sure that the mirror surfaces do not touch, otherwise they may sustain damage during reprocessing.

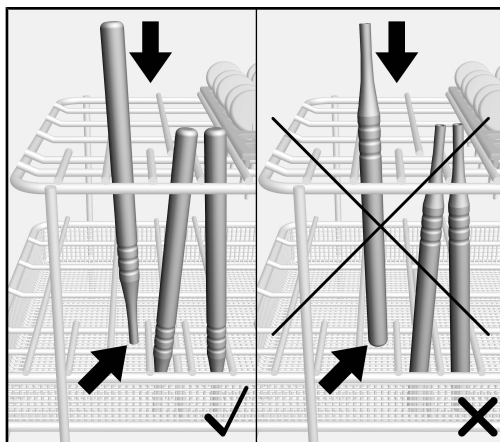- Handles should be placed inside the compartments of the E 337/1 insert with the screw threads pointing down.

Reprocessing

- Once the insert has been loaded, place it into an upper or lower basket where space allows.
Do not stack load carriers such as mesh trays or inserts one above the other.

To check before loading the machine and before starting a programme

- Are instruments that are sharp at one or both ends positioned securely enough inside the plastic receptacles?
- Have the instruments been arranged so that the functional ends are not resting on the wire frame?

Napomene uz upute	58
Pitanja i tehnički problemi	58
Opis namjene	59
Sadržaj isporuke	60
Sadržaj isporuke E 337/1	60
Dodatni pribor	61
Sigurnosne napomene i upozorenja	62
Tehnika primjene	63
Prethodna obrada instrumenata	63
Razvrstavanje instrumenata	63
Zubna ogledala	64
Obrada	65
Prilikom punjenja i prije svakog početka programa provjerite	65

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