A 104

de	Gebrauchsanweisung Injektoroberkorb
da	Brugsanvisning Injektoroverkurv
en	Operating instructions Upper injector basket
es	Instrucciones de manejo Cesto superior del inyector
fi	Käyttöohje Suorasuihkuyläkori
fr	Mode d'emploi Panier à injection
hr	Upute za uporabu Injektorska gornja košara
it	Istruzioni d'uso cesto superiore a iniezione
nl	Gebruiksaanwijzing Injectorbovenrek
no	Bruksanvisning Overkurv/injektor
pt	Instruções de utilização Cesto superior injector
sl	Navodila za uporabo Injektorska košara
sv	Bruksanvisning Injektoröverkorg

de	4
da	18
en	32
es	46
fi	60
fr	74
hr	88
it	102
nl	116
no	130
pt	144
sl	158
sv	172

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Notes about these instructions	32
Intended use	33
Queries and technical problems.....	33
Items supplied.....	34
Optional accessories	35
Disposal of the packing material	35
Warning and Safety instructions	36
Areas of application	37
Before loading the machine and before starting a programme	37
Examples of use	38
Trocars sleeves Ø 10 to 20 mm	38
Trocars without lumens	39
Handles	40
Irrigation sleeves	40
Shafts	41
Rigid fibre optics	41
Veress cannulae	42
Small items	43
Test point for measuring water pressure	44

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property.

Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow button and save your choice with OK.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

Re-usable medical devices can be reprocessed using this upper basket in a Miele washer-disinfector. Follow the instructions for your washer-disinfector as well as the instructions of the medical device manufacturer on how to reprocess their items by machine.

Internal cleaning and disinfection to an appropriate standard is only possible with instruments which can be sufficiently dismantled and with adequate use of injector nozzles, irrigation sleeves and adapters. The reprocessing result of the inside of instruments must be checked as necessary using a test other than a visual check.

The A 104 upper injector basket is designed for the machine reprocessing of surgical MIS, arthroscopy and urology instruments.

It is divided into two sections:

- The right-hand side is fitted with an injector manifold with 11 silicone holders for injector nozzles and irrigation sleeves for reprocessing hollow instruments such as trocars, cannulae and the shafts of MIS instruments.
- The left-hand side is free and can be equipped with inserts and mesh trays for reprocessing instruments without lumen.

The water intake pipe also has several connections for hose adapters, e.g. for E 448 hoses. Connectors not in use must be fitted with E 362 blind stoppers..


This upper basket can be used in the following washer-disinfectors:

- PG 8582
- PG 8582 CD
- PG 8592

Medical devices for reprocessing are referred to as the wash load if they are not more closely defined.

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your Washer-Disinfector or go to www.miele-professional.com.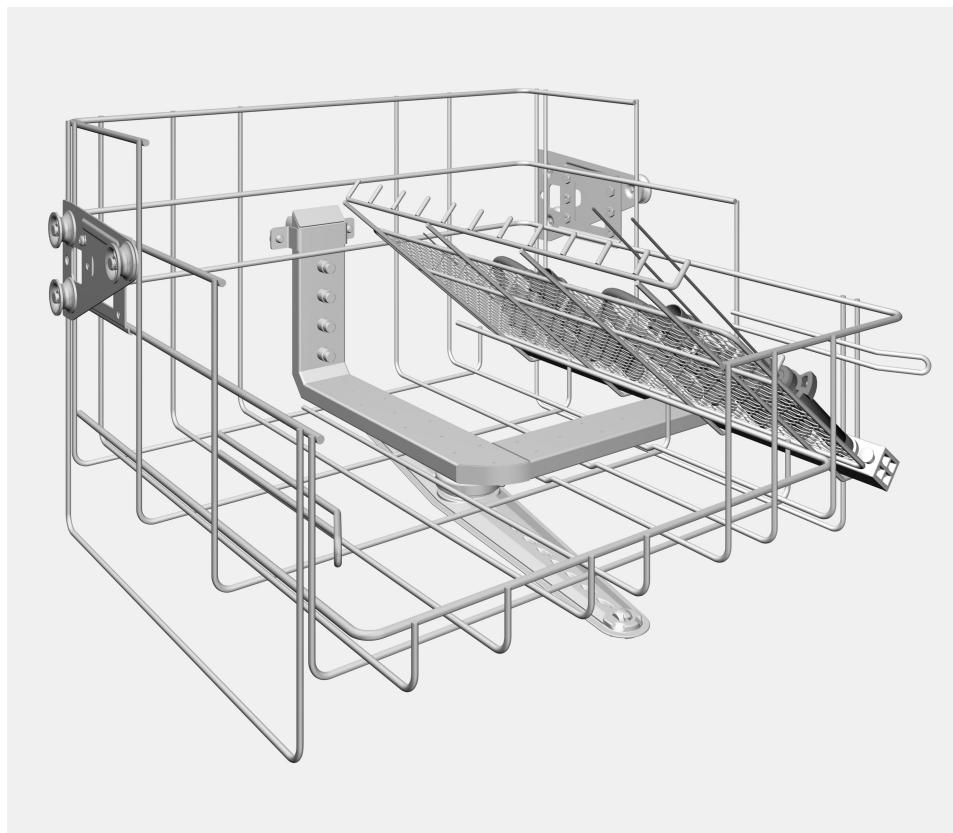

- Upper injector basket A 104, height 332 mm, width 528 mm, depth 526 mm,
loading height: 275 mm
- with spray arm magnets for spray arm monitoring

Items supplied

- 4 x E 362 blind stoppers
- 2 x E 442 irrigation sleeves for instruments Ø 4 - 8 mm, 121 mm long, Ø 11 mm, cap with aperture Ø 6 mm
- 2 x E 448 silicone tubes with Luer Lock adapter ♂ for Luer-Lock ♀, 300 mm long
- 1 x E 452 injector nozzle, 60 mm long, Ø 2.5 mm
- 1 x E 453 injector nozzle with retaining clamp, 100 mm long, Ø 4 mm
- 1 x E 454 injector nozzle for hollow instruments Ø 10 to 15 mm, e.g. trocar sleeves, with height adjustment clip, 150 mm long, Ø 8 mm
- 2 x injector nozzles, Ø 2.5 mm
- 4 x injector nozzles, 30 mm long, Ø 4 mm
- 4 x funnels

Optional accessories

The following and other accessories are available to order from Miele:

- E 441/1 insert for micro-instruments, height 60 mm, width 183 mm, depth 284 mm
- E 449 Luer-Lock-adapter ♂, without ridge, for Luer-Lock ♀
- E 451 mesh tray with lid for small items, height 55 mm, width 150 mm, depth 255 mm
- E 460 insert for rigid fibre optics of different lengths, height 53 mm, width 100 mm, depth 430 mm
- E 464 holder for injector nozzle E 454 for hollow instruments Ø 20 mm, with height adjustment spring, length 65 mm, Ø 13 mm
- E 472 spring clamp for injector nozzles Ø 4 mm
- E 473/1 mesh tray with lid for small items, height 85 mm, width 60 mm, depth 60 mm
- E 499 for reprocessing suction and rinsing cannulae with a variety of connector types as well as Veress cannulae.

Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites. Ensure that any plastic wrappings, bags etc. are disposed of safely and kept out of the reach of babies and young children. Danger of suffocation.

en - Warning and Safety instructions

To avoid the risk of accidents and damage to this upper basket please read these instructions carefully before using it for the first time.

Keep these instructions in a safe place where they are accessible to users at all times.

 Please also read the Operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- This upper basket is only for use in the areas of application as defined in the Operating instructions. Components such as injector nozzles must only be replaced with Miele accessories or original parts. Any other application, modification or change is unauthorised and could be dangerous.
Miele cannot be held liable for damage caused by unauthorised use or incorrect operation.
- Inspect all mobile units, baskets, modules and inserts daily as described in the "Maintenance" section in the Operating instructions for the washer-disinfector.
- New mobile units and baskets must be washed in the washer-disinfector without a wash load prior to first use.
- Only items which have been declared by their manufacturer as suitable for machine reprocessing may be processed. The manufacturer's specific reprocessing instructions must be observed.
Disposable instruments must not be put into the machine for processing.
- The processing programme used should include thermal disinfection, and must be suitable for the material quality of the wash load.
- To clean and disinfect hollow instruments thoroughly from the inside they must be attached to the relevant holders or injector nozzles on the injector manifold. The connectors must not come loose during a programme!
Hollow instruments which have come away from their connectors during the programme must be processed again.
- The last rinse should be carried out with demineralised water wherever possible.

Miele cannot be held liable for damage caused by non-compliance with these Warning and Safety Instructions.

For validation purposes please make sure that you follow the loading instructions given on the template.

⚠ To minimise the risk of injury, the mobile unit should be loaded from the back forwards and unloaded in the reverse order.

⚠ All MIS instruments must be dismantled for reprocessing according to the manufacturer's instructions.

Shafted instruments with narrow lumens which cannot be dismantled must be pre-treated in accordance with their manufacturer's instructions.

Before connecting hollow instruments, please make sure that all hollow sections are free of blockages. Otherwise the cleaning solution will not be able to access all areas properly.

⚠ Any hollow items that have become disconnected from their injector nozzles, adapters or holders during processing must be re-processed.

Before loading the machine and before starting a programme

- Check that the silicone holders in the injector manifold are all present and undamaged.
- Check that fittings, such as irrigation sleeves, injector nozzles and funnels are firmly pushed onto the silicone holders.

⚠ Missing or water-damaged silicone holders must be replaced with new ones. Only then may the mobile unit (basket or module) be used again. If these conditions are present silicone holders standardise the pressure themselves.

Damaged injector nozzles, adapters or irrigation sleeves etc. must not be used.

- Check that the basket is correctly connected to the water supply in the washer-disinfector.
- Check that spray arms are rotating freely.
- Check spray arm jets for blockages.

Examples of use


Trocars sleeves
 \varnothing 10 to 20 mm


- Dismantle the trocar sleeves.

Process small parts such as seals or removable taps in a mesh tray with a lid.

- Place trocar sleeves \varnothing 10 to 15 mm onto an E 454 injector nozzle.
For trocar sleeves \varnothing 15 to 20 mm use an E 464 adapter together with an E 454 injector nozzle.
- Position trocar sleeves with the spring clamps so that the cleaning solution can reach critical areas such as the valve seating through the side apertures.

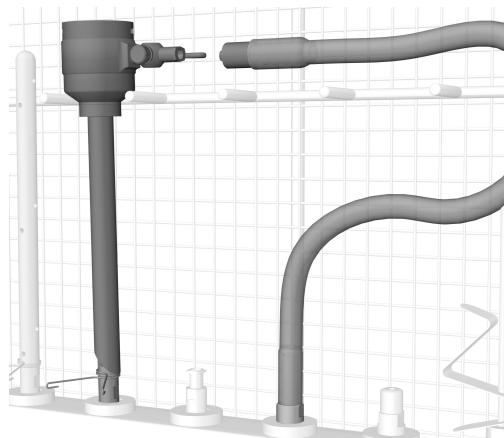
- Connect the side connector of the trocar sleeve to an E 448 hose adapter.
- Push the hose adapter into the injector manifold or into a connector in the water inlet tube.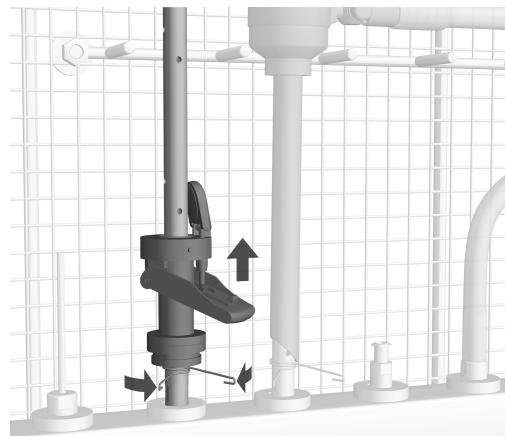

- Place the valve onto an E 454 injector nozzle for trocar sleeves or E 453 with holding clamps.
- Position the valve with the spring clamp or holding clamp so that the cleaning solution can reach critical areas through the side apertures.

Trocars without lumens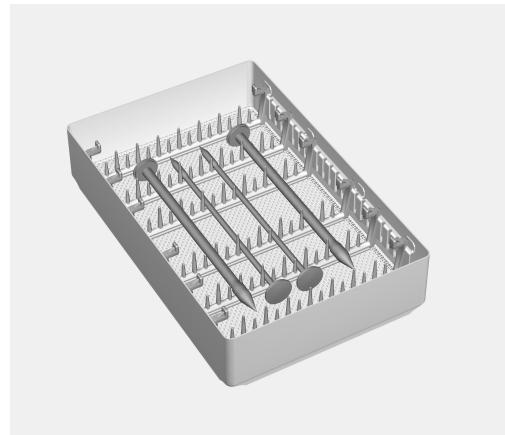


Trocars without lumens or micro instruments can be reprocessed in the E 441/1 insert.

- Push the trocar into the retainers.
- Put the insert in a free place in one of the mobile units or baskets.

en - Areas of application

Handles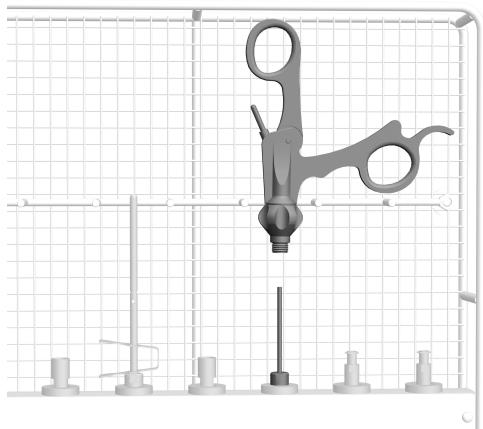

- Fit contaminated handles onto an injector nozzle, e.g. E 452.

Irrigation sleeves

Shafts and tool inserts

E 442 irrigation sleeves are suitable for reprocessing hollow instruments and instruments with an external diameter less than 10 mm.

- Place the shafts of MIS instruments in irrigation sleeves with caps.
- Place HF forceps and electrodes in irrigation sleeves with caps, with the functional ends pointing downwards.


Shafted instruments which cannot be dismantled

- Place instruments which cannot be dismantled in an irrigation sleeve. The functional ends must be open; if necessary use an E 472 spring clamp to hold them open.
Attach the side connectors for instruments with an E 448 hose adapter. Secure the hose adapter to the injector manifold using a silicone holder or with a screw fitting.

Trocars with lumens

- Push the trocar with lumen into the irrigation sleeve.
- Push trocar sleeves of less than 10 mm Ø into irrigation sleeves. Attach the side connector using an E 448 hose adapter. Secure the hose adapter to the injector manifold using a silicone holder or with a screw fitting.

Shafts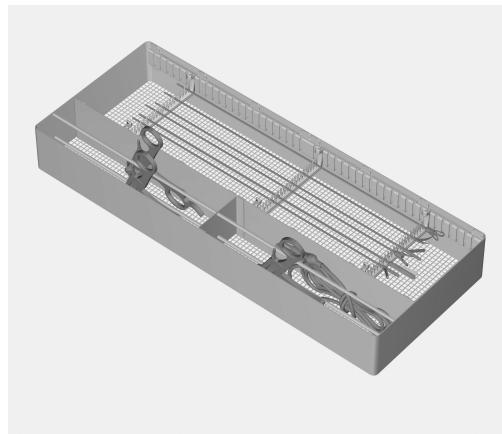

Shafts, HF cables and lightly soiled handles can be processed in the E 457 insert. The insert comes with its own Operating instructions.

Rigid fibre optics


⚠ To avoid mechanical damage only process rigid fibre optics in inserts made by the rigid fibre optic manufacturer or in the special insert E 460.

- Put the insert in a free place in one of the baskets or inserts.

en - Areas of application


Veress cannulae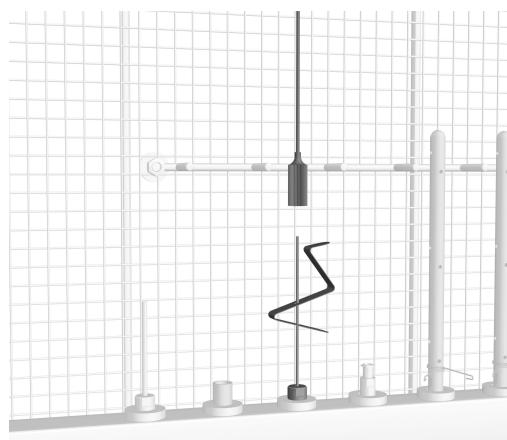

- Dismantling Veress cannulae
- Secure the exterior of the Veress cannulae onto an E 453 injector nozzle with E 472 spring clamps.

The top end of the Veress cannula is held in place by the wire support.


Alternatively an E 499 fitting can be used.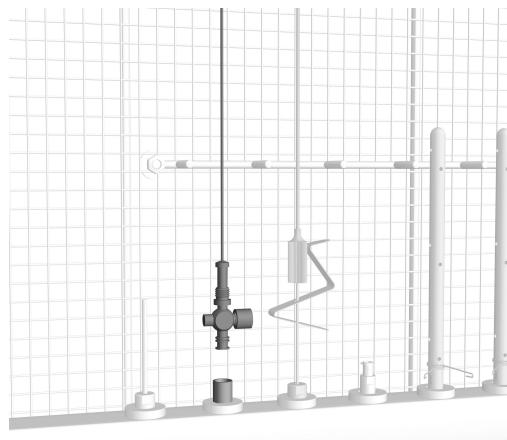


- Screw the interior of the Veress cannula into a Luer Lock adapter ♂.
The tap must be open.

The top end of the Veress cannula is held in place by the wire support.

Small items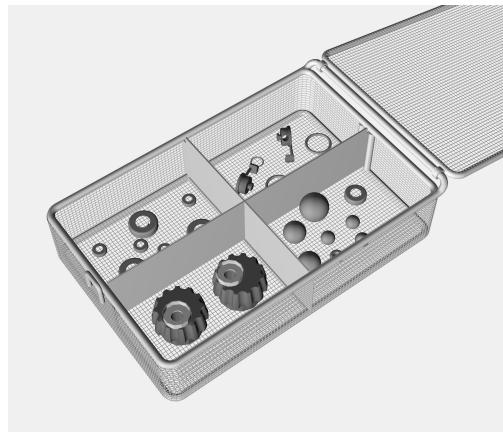


- Process small items, such as seals, taps etc in mesh trays with lids such as E 451.
- Put the mesh tray with the lid closed in a free place in one of the baskets or inserts.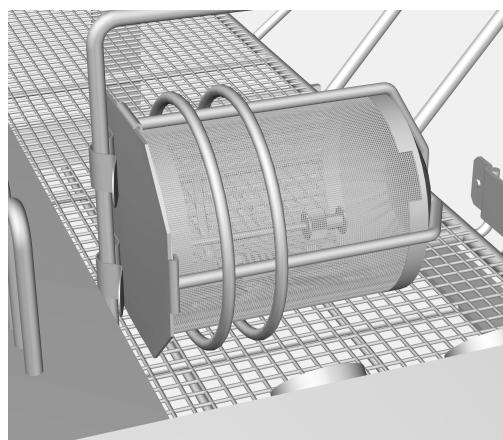

Make sure that extremely small items which could escape through the mesh or through the gap between the mesh tray and the lid are secured, e.g. in insert E 473/1.

- Secure insert E 473/1 to a strut either in the basket or in the mobile unit.

Test point for measuring water pressure

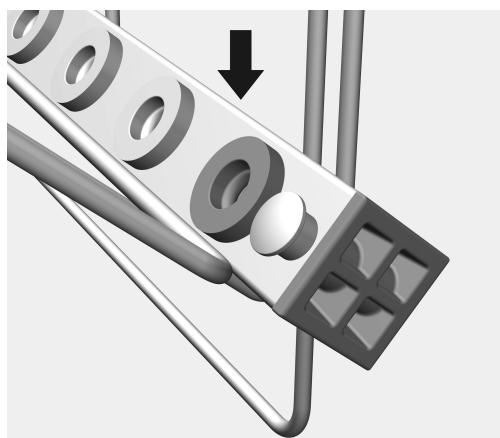The silicone holder at the very front of the injector manifold can be used as the test point for measuring water pressure for performance checks and for validation purposes in accordance with EN ISO 15883.

- To measure water pressure replace the existing fixture with a Luer Lock adapter, e.g. E 447.

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