APWD 217

de	Gebruuchsanweisung Wagen
fr	Mode d'emploi Chariot
it	Istruzioni d'uso Carrelli
da	Brugsanvisning Vogn
fi	Käyttöohje Vaunu
nl	Gebruiksaanwijzing Wagen
no	Bruksanvisning vogn
en	Operating instructions: Mobile unit
sv	Bruksanvisning Vagn

de	4
fr	12
it	20
da	28
fi	36
nl	44
no	52
en	60
sv	68

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Notes about these instructions	60
Intended use	61
Queries and technical problems	61
Items supplied	62
Load carriers.....	62
Disposal of the packing material	62
Optional components	63
Warnings and safety notes	64
Medical devices.....	64
Laboratory glassware and utensils.....	64
Areas of application.....	65
To check before loading the machine and before starting a programme	65
Sample load: APWD 217 mobile unit	65
Medical devices.....	66
Loading APWD 646 modules.....	66

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow buttons and save your choice with *OK*.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

This mobile unit can be used to reprocess machine-reprocessible medical devices in a Miele washer-disinfector. Follow the operating instructions for your washer-disinfector as well as the instructions issued by the medical device manufacturer on how to reprocess their items using a machine.

The mobile unit can be used in the following washer-disinfectors:

- PWD 7121
- PWD 7122


The washer-disinfector is generally referred to as “the machine” in these operating instructions. Reprocessible medical devices are referred to as “load items” if they are not more closely defined.

Disposable items must not be reprocessed.

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your dishwasher or the washer-disinfector or go to www.miele-professional.com.

Load carriers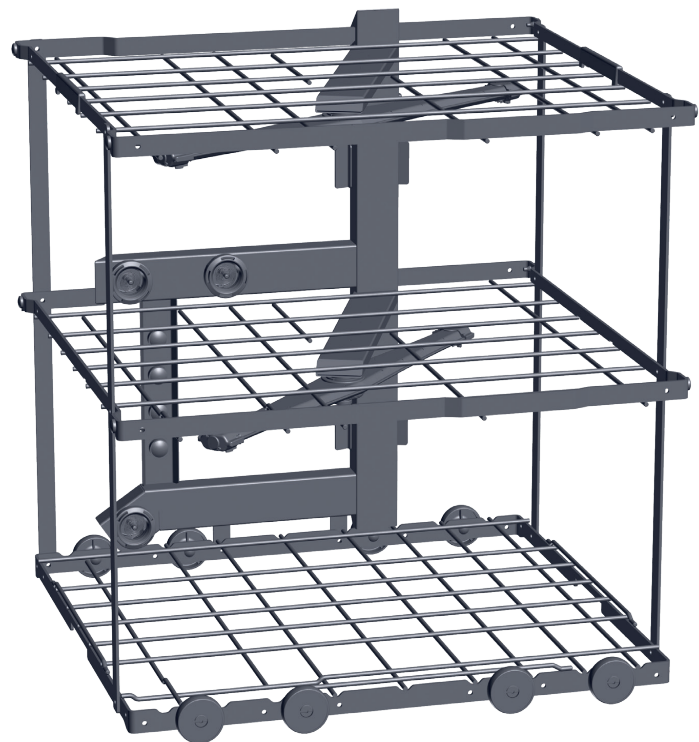

APWD 217 mobile unit, height 599 mm, width 553 mm, depth 637 mm

APWD 217 *	Clearances [height x width x depth in mm]
Level 1	245** x 517 x 570
Level 2	245** x 517 x 570
Level 3	70 x 517 x 570

* Levels are numbered from the bottom up
** Vertical clearance 195 mm when using the injector manifold APWD 646

Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled. Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in land-fill sites.

Optional components

- Medical devices
- A 314, mesh tray for reprocessing lumened instruments, with lockable lid, holding lugs and 1 injector manifold with Luer Lock ♀ connector for Luer Lock ♂, height 47 mm, width 151 mm, depth 225 mm
 - A 317, insert for holding up to 6 dental trays, length 465 mm, width 154 mm, height 143 mm
 - A 865, adapter for 3F and MF syringes
 - APWD 646, module with injector manifolds for reprocessing lumened instruments, height 83 mm, width 124 mm, depth 523 mm
 - E 337/1, insert for instruments, upright sorting and loading, height 113 mm, width 173 mm, depth 445 mm
 - E 430/1, mesh tray, height 40 mm, width 150 mm, depth 445 mm
 - E 473/2 mesh tray with lid for small items, height 85 mm, width 60 mm, depth 60 mm
 - E 521/2, insert for holding 7 extraction forceps, orthodontic or orthopaedic molar forceps, compartment size 21 x 80 mm, height 135 mm, width 100 mm, depth 189 mm
 - E 522/1, insert with 9 holders for dental impression plates, height 140 mm, width 100 mm, depth 190 mm
 - A 813, two-piece adapter with 4 screw threads for various irrigation connectors, height approx. 48 mm, Ø 45 mm, 4 nozzles with silicone hoses and 3 blind stoppers
 - A 814/1, holder with screw cap for reprocessing the tips of air scalers and piezo scalers, height approx. 41 mm, Ø 16 mm
 - A 833, injector nozzle with perforations on the side, length 110 mm, Ø 4 mm

en - Warnings and safety notes

To avoid the risk of accidents and damage to this mobile unit please read these instructions carefully before using it for the first time. Keep these instructions in a safe place where they are accessible to users at all times.

 Please also read the operating instructions for your dishwasher or the washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ The mobile unit is approved solely for the applications specified in the “Areas of application” chapter of these operating instructions. Applications that go beyond those listed and use of the mobile unit with additional components from the Miele product range are described in the operating instructions for the components or are to be agreed with Miele.
- ▶ New mobile units and baskets must be washed in the washer-disinfector without a wash load prior to first use.
- ▶ Inspect all mobile units, baskets, modules and inserts daily as described in the “Maintenance” section in the Operating instructions for the washer-disinfector.
- ▶ Broken glass and ceramics can result in serious injury when loading or unloading. Damaged glass or ceramic load items must not be reprocessed in the machine.

Medical devices

- ▶ The reprocessing programme used should include thermal disinfection, and the process chemicals used must be suitable for the material quality of the load items.
- ▶ The last rinse, as a minimum, must be carried out with DI water.
- ▶ Visual checks should be undertaken to check the cleaning result. Additional protein analysis checks should also be carried out e.g. weekly, on a random sampling basis.

Laboratory glassware and utensils

- ▶ The reprocessing result must be checked as necessary using a suitable test method and not just by a visual check.


Miele cannot be held liable for damage or injury caused by non-compliance with the warning and safety notes.

To check before loading the machine and before starting a programme

- Are the spray arms able to rotate freely?
- Check spray arm jets for blockages.
- Check that the mobile unit is correctly docked to the water path in the chamber.
- Are the inserted modules correctly connected to the water supply of the mobile unit?

For validation purposes please make sure that you follow the loading instructions given on the template.

Sample load: APWD 217 mobile unit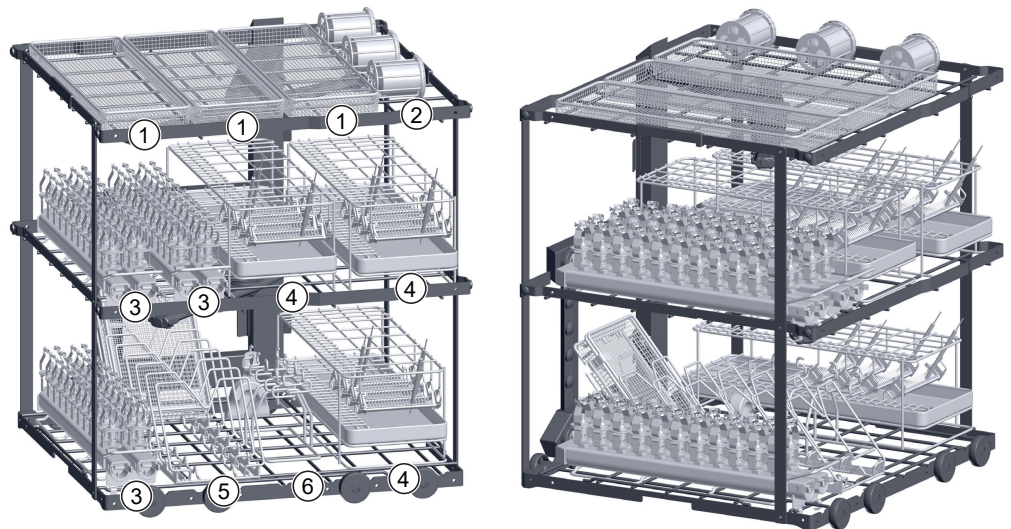

Level 3:

- ① 3x E 430/1 mesh tray
- ② 3x E 473/2 mesh tray for small parts

Level 2:


- ③ 2x APWD 646 module
- ④ 2x E 337/1 insert

Level 1:

- ③ 1x APWD 646 module
- ⑤ 1x A 317 insert with A 314 mesh tray
- ⑥ 1x E 522/1 insert with dental impression plate
- ④ 1x E 337/1 insert


Loading APWD 646 mod- ules

Medical devices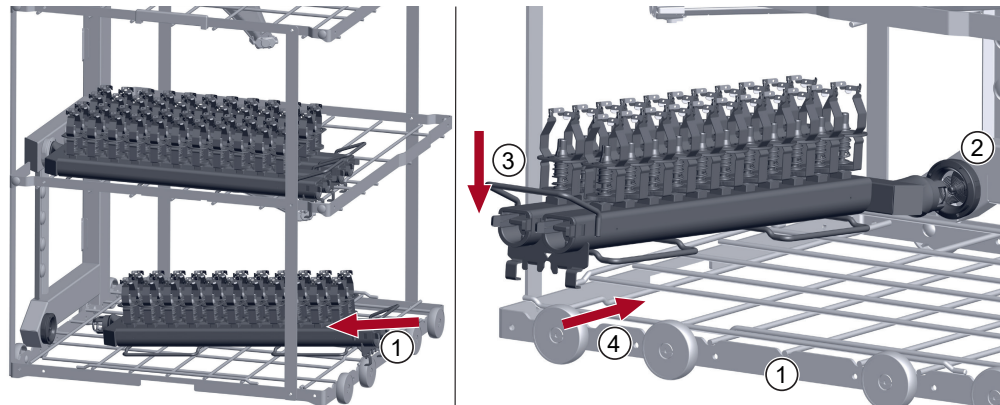

Up to three APWD 646 modules can be accommodated in the mobile unit: one module on the lowest level and two on the middle level.

- Insert the module into the mobile unit from the infeed side ①.
- Position the module at a slight angle and slide it up to the water couplings ②.
- Then press down on the retaining bracket ③ so that the module is lying flat.
- Push the module connector firmly into the water coupling of the mobile unit ④ until the module locking mechanisms click into place on the struts of the mobile unit.
- Ensure that the module is lying flat ⑤ and is firmly inserted in the water coupling. The arrows on the module and the strut must line up ⑥.

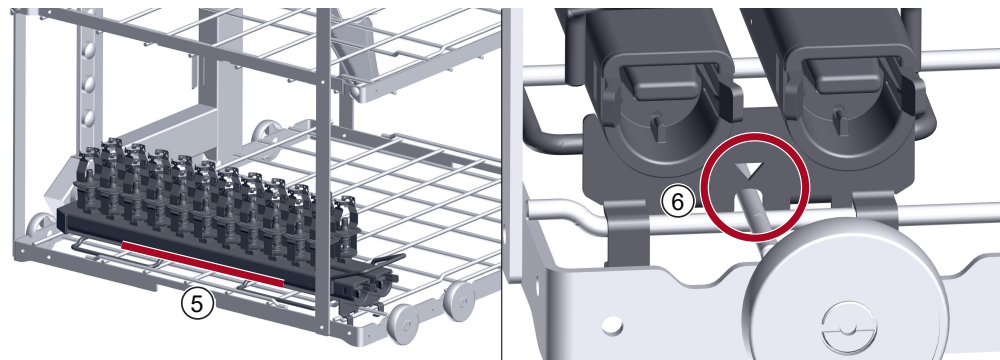Note for longer instruments, e.g. surgical suction cannulae:
Remove the module from the mobile unit.
Load the module outside the mobile unit and then insert it.

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