A 812, A 813

de	Gebrauchsanweisung Adapter
da	Brugsanvisning Adapter
en	Operating instructions Adapters
es	Instrucciones de manejo Adaptador
fi	Käyttöohje Sovitin
fr	Mode d'emploi Adaptateur
hr	Upute za uporabu Adapter
it	Istruzioni d'uso Adattatore
nl	Gebruiksaanwijzing adapter
no	Bruksanvisning adapter
pt	Instruções de utilização Adaptador
sl	Navodila za uporabo Adapter
sv	Bruksanvisning Adapter

de	4
da	15
en	26
es	37
fi	48
fr	59
hr	70
it	81
nl	92
no	103
pt	114
sl	125
sv	136

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Notes about these instructions	26
Intended use	27
Queries and technical problems.....	27
Items supplied	28
Optional accessories.....	29
Disposal of the packing material	29
Warning and Safety instructions	30
Areas of application	31
Installation	31
Fitting the A 812	31
Fitting the A 813	32
Reprocessing	33
Reprocessing transmission instruments with spray channels	33
After reprocessing	34
Cleaning or replacing filters.....	35

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property.

Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow buttons and save your choice with OK.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

The A 812 adapter can be fitted with nozzles, irrigation sleeves, and rinse fittings from the Miele range of accessories in order to reprocess medical products from various disciplines in a Miele washer-disinfector. The application scenarios are described in detail in the operating instructions of the accessories or are to be agreed with Miele. Follow the operating instructions for your washer-disinfector as well as the instructions of the medical product manufacturer on how to reprocess their items by machine.


The A 813 adapter and supplied accessories enable dental transmission instruments with external spray channels to be reprocessed in a Miele washer-disinfector. Follow the operating instructions for your washer-disinfector as well as the instructions of the medical product manufacturer on how to reprocess their items by machine.

In addition, the A 813 adapter can be fitted with additional nozzles, irrigation sleeves, and rinse fittings from the Miele range of accessories. The application scenarios are described in detail in the operating instructions of the accessories or are to be agreed with Miele.

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your washer-disinfector or go to www.miele-professional.com.

en - Items supplied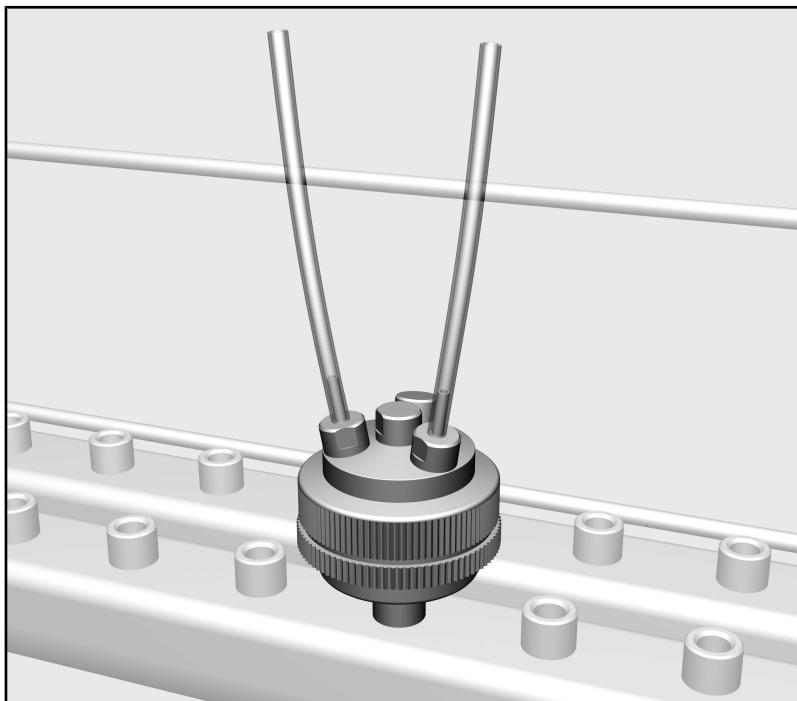

- Two-piece adapter, comprising a top part and a bottom part, height approx. 48 mm, Ø 45 mm

Items supplied

A 812

- 4 x E 362, blind stoppers
- 1 x sealing ring
- 1 x threaded bush
- 1 x clamp
- 1 x silicone adapter for filter plates

A 813

- 1 x A 815, 4 nozzles with silicone hoses, length approx. 114 mm
- 3 x E 362 blind stoppers

Optional accessories

The following and other accessories are available to order from Miele:

- A 801, threaded adapter
- A 815, 4 nozzles with silicone hoses, length approx. 114 mm
- E 362, blind stopper
- FP 20, 20 filter plates, Ø 30 mm

Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

en - Warning and Safety instructions

Read the operating instructions carefully before using this adapter. This prevents both personal injury and damage to the adapter. Keep these operating instructions in a safe place.

 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- The adapter is approved for the applications specified in the "Application technology" chapter of these operating instructions. Applications that go beyond those listed and use of the adapter with additional nozzles, irrigation sleeves, and rinse fittings from the Miele range of accessories are described in the operating instructions of the accessories or are to be agreed with Miele.
All other uses, conversions, and modifications are not permitted.
- Inspect all mobile units, baskets, modules and inserts daily as described in the "Maintenance" section in the Operating instructions for the washer-disinfector.
- New load carriers must be cleaned in the washer-disinfector without a load prior to first use.
- To clean and disinfect hollow instruments thoroughly from the inside they must be attached to the relevant holders or injector nozzles on the injector manifold. The connectors must not come loose during a programme!
Hollow instruments which have come away from their connectors during the programme must be processed again.
- Any hollow items that have become disconnected from their injector nozzles, adapters or holders during processing must be re-processed.
- The last rinse should be carried out with demineralised water wherever possible.


Miele cannot be held liable for damage caused by non-compliance with these Warning and Safety Instructions.

Installation

Fitting the A 812

The A 812 adapter is designed for use on injector manifolds with silicone adapters.

- Remove a nozzle and the silicone adapter from the injector manifold.
- Place the sealing ring around the injector port that is now free.
- Insert the threaded bush into the injector port.
- Secure the threaded bush with the clamp. The clamp must reach around the injector manifold.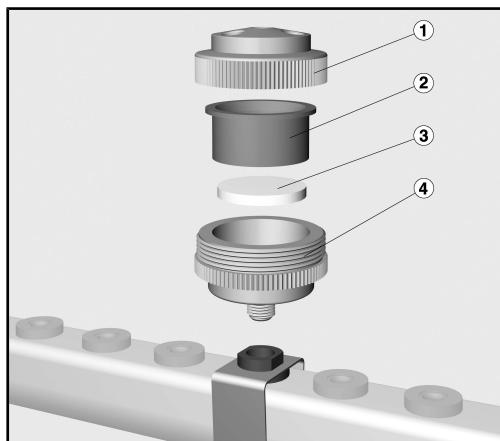

- Screw the bottom part of the adapter (4) into the threaded bush.

The wash water used for reprocessing narrow lumen instruments such as transmission instruments needs to be filtered.

- To do this, insert an FP 20 filter plate (3) into the bottom part.
- Insert the silicone adapter (2) into the bottom part.
- Screw the top part (1) and bottom part (4) together.

You can then connect the nozzles, e.g. nozzles with A 815 silicone hoses, or other irrigation sleeves and rinse fittings of your choice to the adapter.


To fit the A 812 on an injector manifold with a screw thread, you have to replace the clamp, sealing ring, and threaded bush with the A 801 threaded adapter.

en - Areas of application

Fitting the A 813


The A 813 adapter is intended for use on injector manifolds with screw connections.

- Remove a nozzle from the injector manifold of your upper basket or mobile unit.
- Unscrew the top part of the A 813 adapter.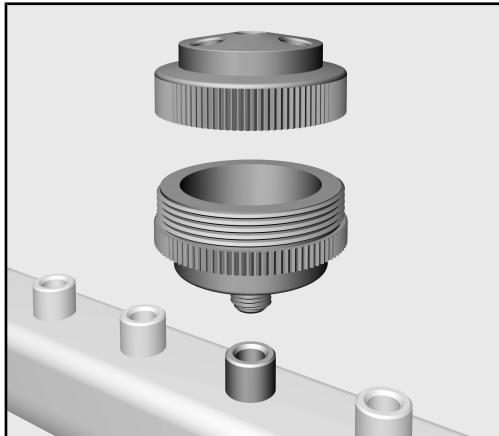

- Screw the bottom part of the adapter in at this position.
- Screw the top part into the bottom part.
- Then screw the nozzles and A 815 silicone hoses into the rinse ports. Alternatively, you can also connect different rinse fittings from the Miele range of accessories to the adapter. Ports that are not needed must be sealed with E 362 blind stoppers.

The wash water used for reprocessing narrow lumen instruments such as transmission instruments needs to be filtered.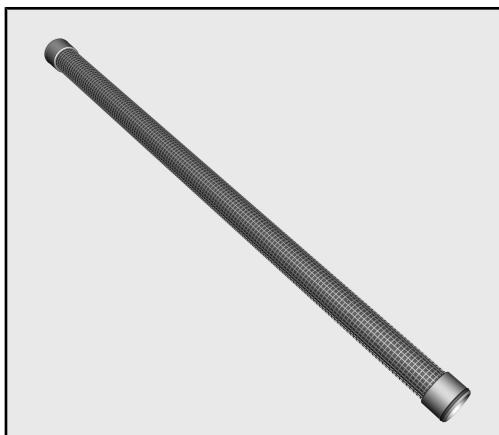

- The injector manifold must be fitted with an A 800 tubular filter.

The tubular filter is supplied with its own operating instructions. Follow the instructions on how to perform regular cleaning on the tubular filter.

Reprocessing

For validation purposes please make sure that you follow the loading instructions given on the template.

⚠ To minimise the risk of injury, the mobile unit should be loaded from the back forwards and unloaded in the reverse order.

⚠ Any hollow items that have become disconnected from their injector nozzles, adapters or holders during processing must be re-processed.


Reprocessing transmission instruments with spray channels

To reprocess transmission instruments with spray channels, 1 to 4 A 815 nozzles with silicone hoses must be connected to the adapter.

⚠ Ports that are not in use must be sealed with blind stoppers.

Check the silicone hoses for visible damage. Damaged silicone hoses must be replaced prior to reprocessing.

- Check that the transmission instrument and the spray channel are free of blockages, following the specifications of the instrument manufacturer.
- Insert the transmission instrument into an A 803 holder or AUF 1 holder.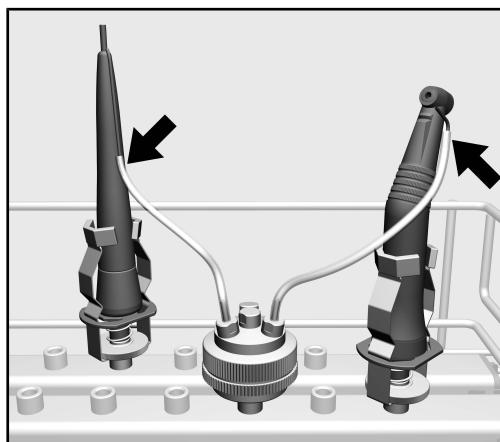

- Connect the free end of the silicone hose to the lower end of the spray channel.

en - Areas of application

After repro- cessing

- Remove the silicone hose once reprocessing is complete. Then remove the transmission instrument from the A 803 or AUF 1 holder.
- After processing dry internal sections of transmission instruments with sterilised compressed air and then sterilise as required following manufacturer's instructions and national health and safety regulations.

Before using transmission instruments again following reprocessing, a function check must be carried out, e.g. by spraying into a basin, to ensure they are clear.

Cleaning or replacing filters

Cleaning the A 800 tubular filter


The tubular filter must be cleaned **at least once a week**.

- Follow the cleaning instructions in the operating instructions for the tubular filter.

Replacing FP 20 filter plates

Borosilicate filter plates must be replaced **approx. every 2 weeks or after 20 cycles** have been run.

- To do this, unscrew the adapter.
- Unscrew the top part from the bottom part.

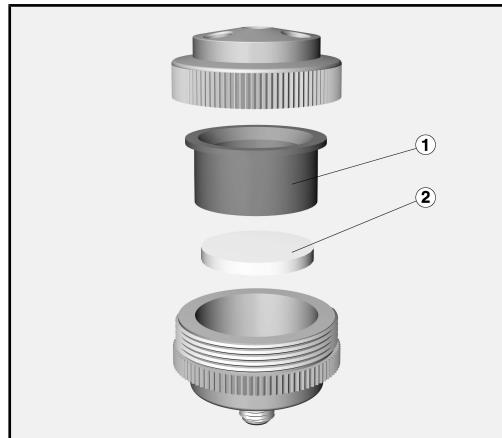- Remove the silicone adapter (1) and the filter plate (2) from the bottom part.
- Rinse out the bottom part.
- Insert a new FP 20 filter plate into the bottom part. The plate must rest fully on the ring-shaped support of the bottom part.

⚠ Damaged or used filter plates must not be inserted under any circumstances.

- Insert the silicone adapter (1) into the bottom part.
- Screw the top part and bottom part together.
- Screw the adapter back onto the injector manifold.

Do not insert a filter plate into the holder if the injector manifold has been fitted with an A 800 tubular filter.

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