# A 317

<b>de</b>	Gebrauchsanweisung Einsatz für Dentaltrays
<b>da</b>	Brugsanvisning Indsats til dentalbakker
<b>en</b>	Operating instructions Insert for dental trays
<b>es</b>	Instrucciones de manejo Complemento para bandejas dentales
<b>fi</b>	Käyttöohje Teline hammaslääkärin instrumenttitarjottimille
<b>fr</b>	Mode d'emploi Complément pour plateaux dentaires
<b>hr</b>	Upute za uporabu Umetak za stomatološke podloške
<b>it</b>	Istruzioni d'uso Inserto per tray Dental
<b>nl</b>	Gebruiksaanwijzing Inzet voor tandheelkundige trays
<b>no</b>	Bruksanvisning innsats for dentalbrett
<b>pt</b>	Instruções de utilização Complemento para tabuleiros da área dental
<b>sl</b>	Navodila za uporabo Vložek za zobozdravniške pladnje
<b>cs</b>	Návod k obsluze Nástavec pro dentální tácy

**de** ..... 4  
**da** ..... 12  
**en** ..... 20  
**es** ..... 28  
**fi** ..... 36  
**fr** ..... 44  
**hr** ..... 52  
**it** ..... 60  
**nl** ..... 68  
**no** ..... 76  
**pt** ..... 84  
**sl** ..... 92  
**cs** ..... 100

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<b>Notes about these instructions</b> .....	20
Queries and technical problems.....	20
<b>Intended use</b> .....	21
<b>Items supplied</b> .....	22
<b>Warning and Safety instructions</b> .....	23
<b>Areas of application</b> .....	24
Installation .....	24
Removing the holders .....	24
Preparing the load .....	24
Reprocessing .....	25
Before loading the machine and before starting a programme .....	26

### Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

### Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

### Additional information and comments

Additional information and comments are contained in a simple frame.

### Operating steps

Operating steps are indicated by a black square bullet point.

#### Example:

- Select an option using the arrow buttons and save your choice with *OK*.

### Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

#### Example:

Settings  menu.

### Queries and technical problems

In the event of queries or technical problems, please contact Miele. Contact details can be found at the end of the operating instructions for your cleaning machine or at [www.miele.com/professional](http://www.miele.com/professional).

This insert can be used to reprocess machine-reprocessable medical devices in a Miele washer-disinfector. Follow the operating instructions for your washer-disinfector as well as the instructions of the medical device manufacturer on how to reprocess their items by machine.

The A 317 insert will hold up to 6 dental trays.

The insert can be placed into the following baskets or carriers:

A 105/1

A 105

A 101

A 102

A 151

APWD 060

APWD 061

Disposable items must not be reprocessed.

The washer-disinfector is generally referred to as “the machine” in these operating instructions. Reprocessable medical devices are referred to as “items” if they are not more closely defined.

## en - Items supplied

---

- A 317 insert, length 465 mm, width 154 mm, height 143 mm  
Further components are available from Miele as optional extras.

## en - Warning and Safety instructions

---

Read the operating instructions carefully before using this insert. Keep these operating instructions in a safe place.

 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

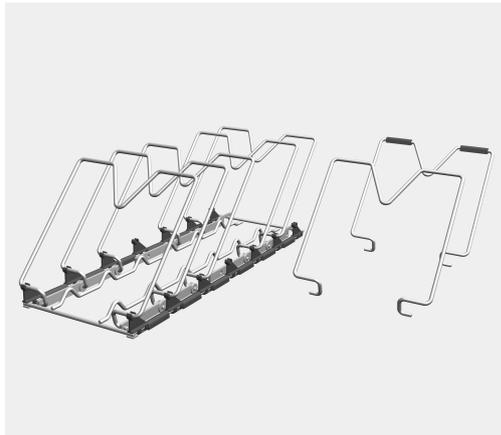
- ▶ The inserts are only approved for use in the applications stated in the operating instructions. Applications that go beyond those listed and use of the inserts with additional components from the Miele range of accessories are described in the operating instructions of the accessories or are to be agreed with Miele. Conversions, modifications and any other use are not permitted.
- ▶ Prior to being used for the first time, new inserts must undergo thorough cleaning in the washer-disinfector without load items.
- ▶ Inspect all mobile units, baskets, modules and inserts as described in the “Maintenance” section of the operating instructions for your cleaning machine.
- ▶ For your own safety, wear gloves and, if necessary, a surgical mask and protective goggles when sorting the instruments and unloading the machine. To avoid recontamination, wear clean gloves when unloading the machine.
- ▶ Load from back to front and unload from front to back.
- ▶ The last rinse, as a minimum, must be carried out with DI water.

### Removing the holders

#### Installation

The insert is supplied assembled complete with 7 holders. The holders tilt back when the insert is loaded with wash items. The last holder leans against the wall of the washer-disinfector when the insert is loaded with wash items. Depending on the size of the trays and the extent to which they cause the holders to tilt back, it may be necessary to remove the last or the fourth holder.

- Squeeze the holders carefully and pull them off the rail.

The holders are attached to the frame by the plastic retainers on the sides. The plastic retainers must remain attached to the frame once the holders have been removed.

#### Preparing the load

For validation purposes please make sure that you follow the loading instructions given on the template.

 Risk of injury caused by load.

There is a risk of injury when loading and unloading due to possible sharp edges, rims or pointed ends.

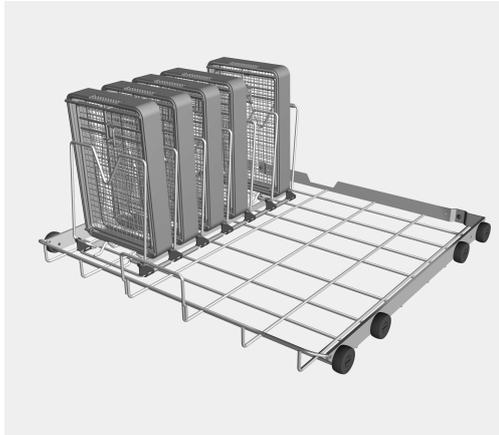
To minimise the risk of injury, loading should take place from the back forwards and unloading should take place in the reverse order.

### Suitable mesh trays

Non-Miele mesh trays, cassettes and other containers can be used for reprocessing instruments in Miele washer-disinfectors if they have been risk assessed and specified as suitable for such use and if reprocessing procedures have been successfully validated with their use.

### Reprocessing

- Place the insert into an upper or lower basket where space allows.
- Load the insert with the dental trays that are to be cleaned. The holders are positioned vertically.

- Tilt the holders back with the dental trays loaded.

The loaded insert can be processed in the cleaning machine.

## en - Areas of application

---

### **Before loading the machine and before starting a programme**

- Check that the dental trays are not colliding with the wash cabinet walls.
- Check that the door of the washer-disinfector will close.

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