# APWD 325

- de** Gebrauchsanweisung Einsatz mit Adapter für die Aufbereitung von Schläuchen von dentalen Lachgassedierungssystemen
- da** Brugsanvisning Indsats med adapter til genbehandling af slanger til dentale lattergassystemer
- en** Operating instructions Insert with adapter for reprocessing dental nitrous oxide sedation hoses
- es** Instrucciones de manejo Complemento con adaptador para la preparación de mangueras y sistemas de sedación con óxido nitroso
- fi** Käyttöohje Sovittimilla varustettu teline hammashuollon ilokaasusedaatiojärjestelmien letkujen käsittelyä varten
- fr** Mode d'emploi Complément avec adaptateur pour traiter les tuyaux de systèmes dentaires de sédation au protoxyde d'azote.
- hr** Upute za uporabu Umetak s adapterom za obradu crijeva stomatoloških sustava za sedaciju dušikovim oksidom
- it** Istruzioni d'uso Inserto con adattatore per il trattamento di tubi di sistemi per la sedazione con ossido di azoto in odontoiatria
- nl** Gebruiksaanwijzing Inzet met adapter voor de herverwerking van slangen van tandheelkundige lachgassedatiesystemen
- no** Bruksanvisning innsats med adapter for klargjøring av slanger i dentale bedøvelsessystemer med lystgass
- pt** Instruções de utilização Complemento com adaptador para o reprocessamento de mangueiras de sistemas de sedação com óxido nitroso dentário
- sl** Navodila za uporabo Vložek z adapterjem za pripravo cevi sistemov za uporabo smejalnega plina v zdravstvu
- cs** Návod k obsluze Nástavec s adaptérem pro přípravu hadic dentálních sedačních systémů na rafinský plyn

<b>de</b>	.....	4
<b>da</b>	.....	16
<b>en</b>	.....	28
<b>es</b>	.....	40
<b>fi</b>	.....	52
<b>fr</b>	.....	64
<b>hr</b>	.....	76
<b>it</b>	.....	88
<b>nl</b>	.....	100
<b>no</b>	.....	112
<b>pt</b>	.....	124
<b>sl</b>	.....	136
<b>cs</b>	.....	148

---

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---

<b>Notes about these instructions .....</b>	28
<b>Intended use .....</b>	29
<b>Items supplied .....</b>	30
Optional components .....	30
Disposal of the packing material .....	30
<b>Warning and Safety instructions .....</b>	31
<b>Areas of application .....</b>	32
Installation .....	32
Installing the adapters .....	32
Installing the extension set hose holders .....	33
Preparing the load .....	35
Preparing the load items – standard set.....	35
Preparing the load items – extension set .....	36
APWD 325 maximum load .....	37
Reprocessing .....	38
Reprocessing double hoses used in nitrous oxide sedation systems .....	38
To check before loading the machine and before starting a programme .....	38

## Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property.

Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

## Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

## Additional information and comments

Additional information and comments are contained in a simple frame.

## Operating steps

Operating steps are indicated by a black square bullet point.

### Example:

- Select an option using the arrow buttons and save your choice with OK.

## Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

### Example:

Settings  menu.


The APWD 325 insert with adapter can be used to reprocess reprocessable hoses used in dental nitrous oxide sedation systems in a washer-disinfector. Follow the operating instructions for your washer-disinfector as well as the instructions issued by the medical device manufacturer on how to reprocess their items using a machine.

The insert with adapter can be used in the following baskets and modules:

A 105/1 upper basket

A 151 lower basket carrier with A 315 module

The washer-disinfector is generally referred to as “the machine” in these operating instructions. Reprocessable medical devices are referred to as “load items” if they are not more closely defined.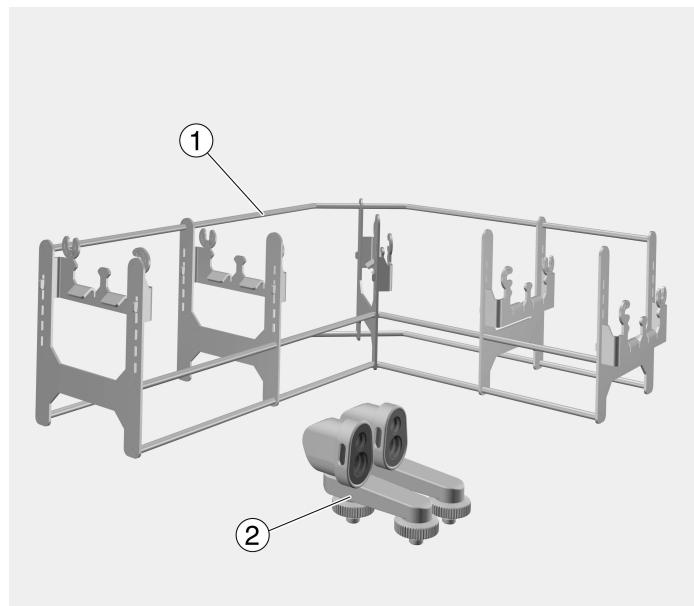

① Hose guide insert

② Adapter

### APWD 325 set:

1 x insert for double hoses, length 397 mm, width 397 mm, height 397 mm

2 x adapters for double hoses, length 78 mm, width 78 mm, height 78 mm

### Optional components

- APWD 326, retrofit set for double hoses used in dental nitrous oxide systems, 2 x adapters
- APWD 328, extension set for reprocessing up to 5 double hoses used in dental nitrous oxide systems, 5 x hose holders
- E 146, mesh tray with lid and hinged handles, height 55 mm, width 150 mm, depth 255 mm

### Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

Read the operating instructions carefully before using this insert.  
Keep these operating instructions in a safe place.

 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.


- The inserts are only approved for use in the applications stated in the operating instructions. Applications that go beyond those listed and use of the inserts with additional components from the Miele range of components are described in the operating instructions of the components or are to be agreed with Miele.  
Conversions, modifications and any other use are not permitted.
- Prior to being used for the first time, new inserts must undergo thorough cleaning in the washer-disinfector without load items.
- New load carriers must be cleaned in the washer-disinfector without a load prior to first use.
- Inspect all mobile units, baskets, modules and inserts as described in the "Maintenance" section of the operating instructions for your cleaning machine.
- For your own safety, wear gloves and, if necessary, a surgical mask and protective goggles when sorting the instruments and unloading the machine. To avoid recontamination, wear clean gloves when unloading the machine.
- Load from back to front and unload from front to back.

### Installation

#### Installing the adapters

Adapters from the APWD 325 set are required to reprocess the hoses used in dental nitrous oxide sedation systems. The adapters are suitable for all commercially available double hoses.

- Screw the adapters into the connections on the injector manifold.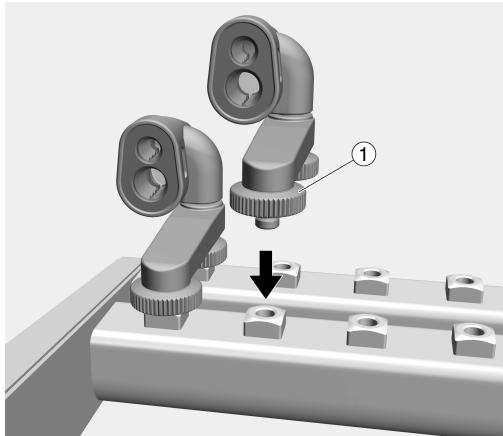

- Secure the adapters using the two ① knurled nuts and tighten them at the same time.

Additional adapters must be installed if the APWD 328 extension set is used. The adapters are available as retrofit set APWD 326.


When the adapters are used, the wash water must be filtered. The injector manifold must be fitted with an A 800 tubular filter. The wash water consists of the inflowing water, the process chemicals and the soiling.

The tubular filter is supplied with its own operating instructions. Follow in particular the instructions on how to perform regular cleaning of the tubular filter.


**Installing the extension set hose holders**

The APWD 328 extension set enables up to 5 double hoses to be cleaned and disinfected. The set contains 5 hose holders. The extension set hose holders must be installed on the fixed hose holders on the insert.

- Secure the extension set hose holder ① to the hook ② on the insert hose holder at the pivot point ③.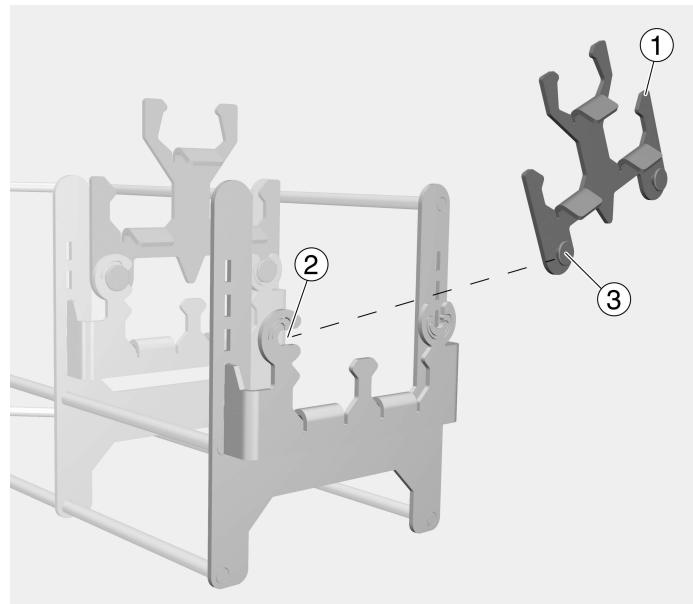


- Push the extension set hose holder down onto the retainer on the other side ④.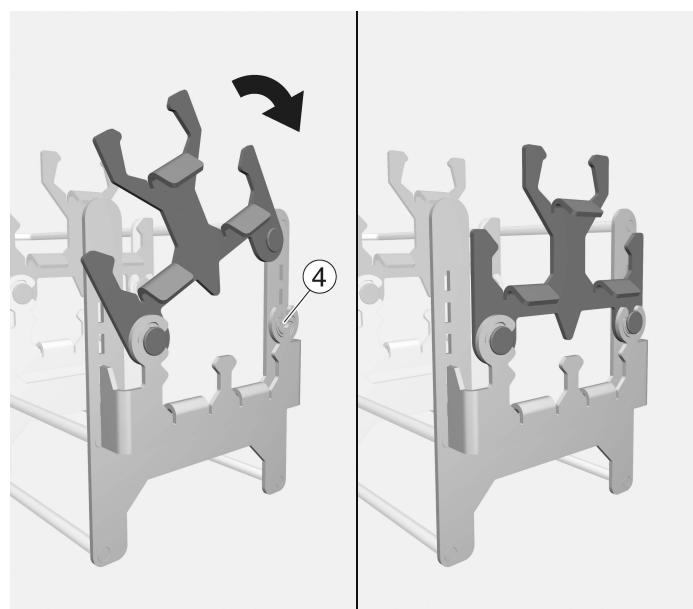

## en - Areas of application

---

- Install the other extension set hose holders on the pre-installed hose holders and place the insert in A 105/1 or A 151 with A 315.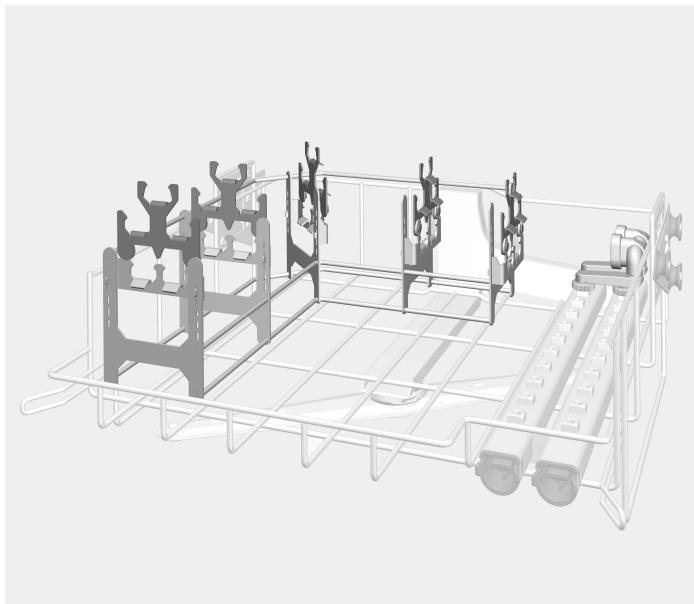

Hose guide insert with A 105/1

## Preparing the load

For validation purposes please make sure that you follow the loading instructions given on the template.

 Risk of injury caused by load.


There is a risk of injury when loading and unloading due to possible sharp edges, rims or pointed ends.

To minimise the risk of injury, loading should take place from the back forwards and unloading should take place in the reverse order.

## Preparing the load items – standard set

- Insert the double hoses into the silicone insert ① on the adapter.

Make sure that the diameter of the hoses is the same as the diameter of the silicone insert on the adapter.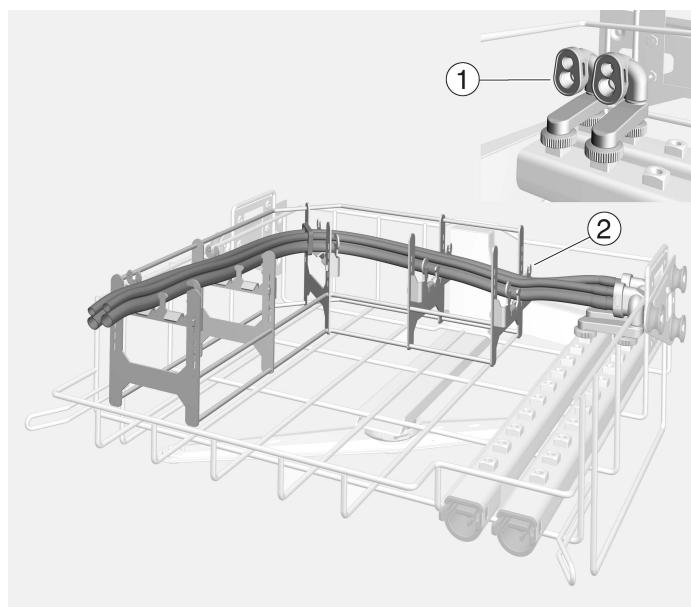

- Place the double hose on the hose holder ② fastened to the insert.


Make sure that the hose sets are not twisted when they are put in place.

2 double hoses can be placed on the hose holder. If the insert is fitted with the APWD 328 and APWD 326 extension set, 3 additional double hoses can be positioned.

## en - Areas of application

### Preparing the load items – extension set


- Open the extension set hose holder ③.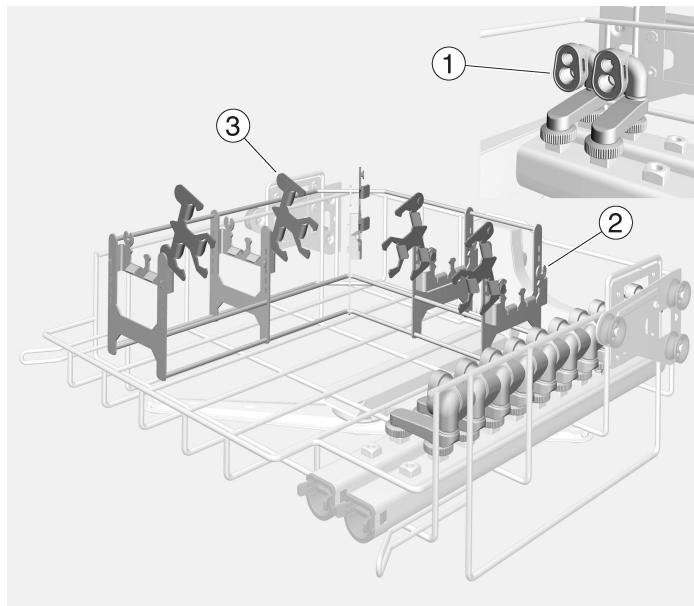

- Insert the double hoses into the silicone insert ① on the adapter.

Make sure that the diameter of the hoses is the same as the diameter of the silicone insert on the adapter.

- Place the double hoses on the hose holder ② fastened to the insert.
- Close the extension set hose holder ③ until the pivot point locks in place.
- Insert the other double hoses into the silicone insert ① on the additional adapters.
- Place the double hoses on the extension set hose holder ③.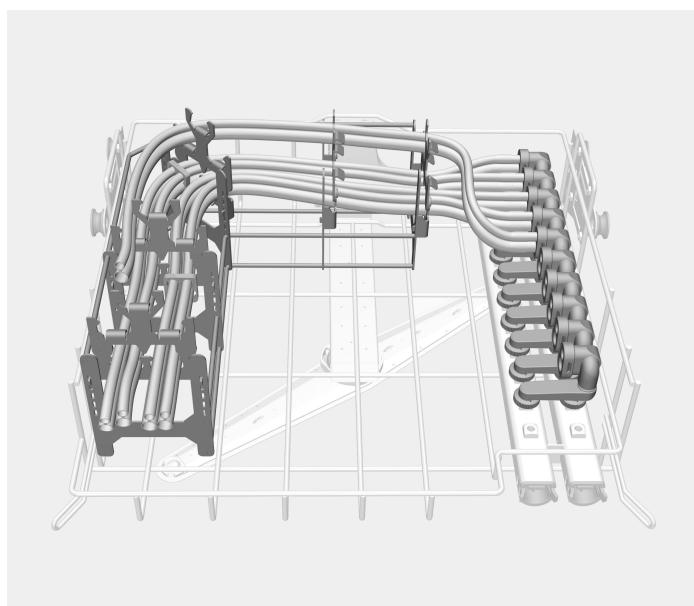


**APWD 325 maximum load**

Basket/module	Cleaning programme	Maximum load
A 105/1 + A 151	Vario TD Dental	2 hose sets
A 105/1 + A 151 + A 315	Vario Dental Plus	4 hose sets
Optional:		
A 105/1 + A 151	Vario Dental Plus	5 hose sets

If the APWD 325 insert has been installed in the A 105/1 upper basket or in the A 151 lower basket carrier with the A 315 module, the remaining space can be filled with other dental instruments or inserts.

One option is to reprocess the mouthpieces of the nitrous oxide sedation system in the E 146 mesh tray. Check that the mouthpieces are suitable for reprocessing in the washer-disinfector before reprocessing them.

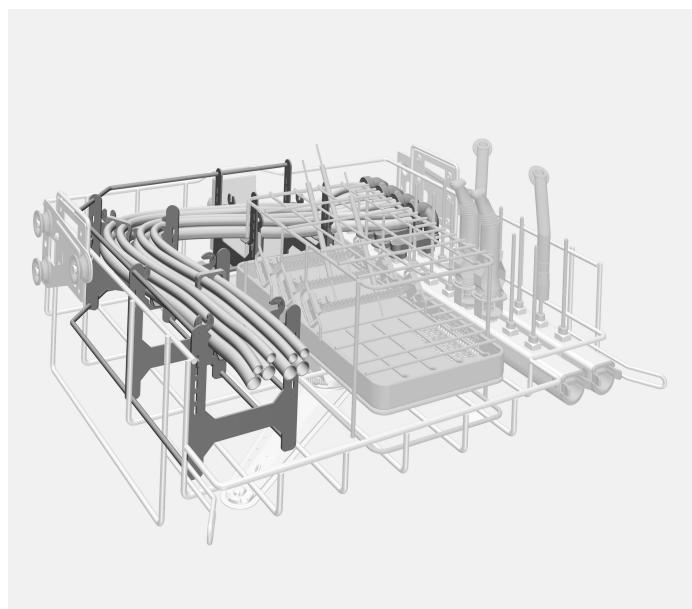Fully loaded A 105/1 with APWD 325

### Reprocessing

 Risk of infection due to insufficient disinfection.

Load items that become detached from the irrigation connector during reprocessing will not be sufficiently disinfected on the inside.

Any items that become detached from the irrigation connector during reprocessing must be reprocessed again.

#### Reprocessing double hoses used in nitrous oxide sedation systems

#### To check before loading the machine and before starting a programme

- Make sure that the double hoses can be flushed through.

- Is the silicone insert in the hose adapter in the correct position?
- Are the ends of the double hose inside the basket area?

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