A 203

de	Gebrauchsanweisung Injektorwagen
da	Brugsanvisning Injektorvogn
en	Operating instructions Injector unit
es	Instrucciones de manejo Carros inyectoros
fi	Käyttöohje Suorasuihkuvaunu
fr	Mode d'emploi Chariot à injection
hr	Upute za uporabu Injektorska kolica
it	Istruzioni d'uso Carrello a iniezione
nl	Gebruiksaanwijzing injectorwagen
no	Bruksanvisning injektorvogn
pt	Instruções de utilização Carro injetor
sv	Bruksanvisning Injektorvagn
sl	Navodila za uporabo Injektorski voziček

de	3
da	18
en	33
es	48
fi	63
fr	78
hr	93
it	108
nl	123
no	138
pt	153
sv	168
sl	183

Warnungen

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Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or property.
Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

■ Select an option using the arrow button and save your choice with *OK*.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

Re-usable medical devices can be reprocessed using this mobile unit in a Miele washer-disinfector. Follow the instructions for your washer-disinfector as well as the instructions of the medical device manufacturer on how to reprocess their items by machine.

Internal cleaning and disinfection to an appropriate standard is only possible with instruments which can be sufficiently dismantled and with adequate use of injector nozzles, irrigation sleeves and adapters. The reprocessing result of the inside of instruments must be checked as necessary using a test other than a visual check.

The A 203 mobile injector unit is suitable for reprocessing MIS instruments up to a maximum length of 550 mm.

The mobile unit is divided into two levels.

On the upper level there are various connectors for processing hollow instruments, e.g. trocars, cannulae and shafts of MIS instruments.

The lower level is equipped with inserts and mesh trays for processing instruments without lumens. Loading dimensions: height 105 mm, width 480 mm, depth 500 mm.

The mobile unit can be used in the following washer-disinfectors:

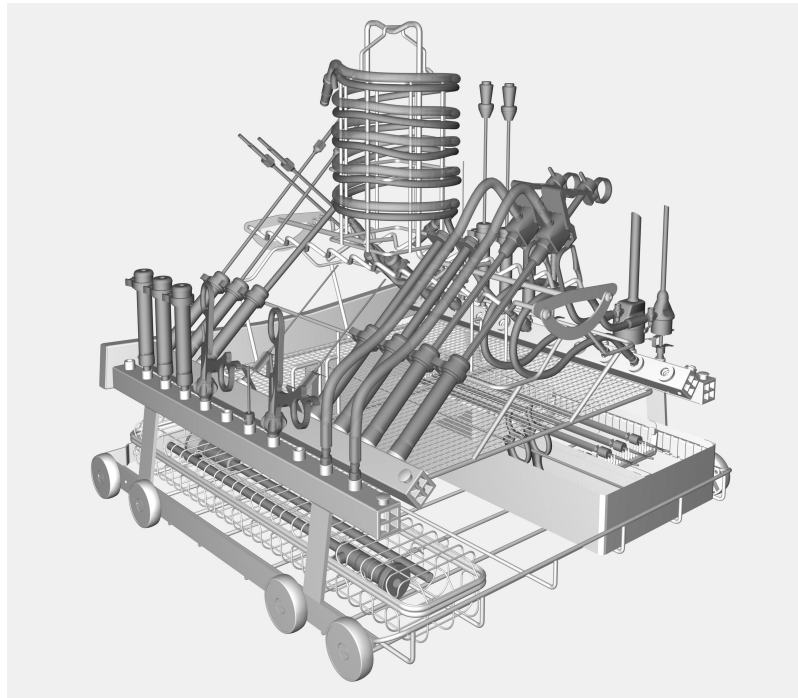
- PG 8582
- PG 8582 CD
- PG 8592

Medical devices for reprocessing are referred to as the wash load if they are not more closely defined.

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your Washer-Disinfector or go to

www.miele-professional.com.

- A 203 Mobile injector unit with two levels, height 520 mm, width 531 mm, depth 542 mm

Items supplied

- 3 x E 336, MIBO irrigation sleeves, 121 mm long, Ø 11 mm
- 2 x E 362 blind stoppers
- 15 x E 442 irrigation sleeves for instruments Ø 4 - 8 mm, 121 mm long, Ø 11 mm, cap with aperture Ø 6 mm
- 5 x E 443, Sleeves for instruments Ø 8 - 8.5 mm, length 121 mm, Ø 11 mm, cap with aperture Ø 10 mm
- 1 x E 444, Drum for fibre optic cables and suction tubes, with folding clamp, height 168/214 mm, Ø 140 mm
- 1 x E 445, 12 caps for sleeves; aperture Ø 6 mm,
- 1 x E 446, 12 caps for sleeves; aperture Ø 10 mm
- 3 x E 447, Luer Lock adapters ♀ for Luer Lock ♂
- 6 x E 448 silicone tubes with Luer Lock adapter ♂ for Luer-Lock ♀, 300 mm long
- 5 x E 449, Luer Lock adapters ♂ for Luer Lock ♀
- 1 x E 451, Mesh tray with lid for small items, height 55 mm, width 150 mm, depth 255 mm
- 3 x E 452 injector nozzle, 60 mm long, Ø 2.5 mm
- 8 x E 453 injector nozzle with retaining clamp, 100 mm long, Ø 4 mm
- 6 x E 454 injector nozzle for hollow instruments Ø 10 to 15 mm, e.g. trocar sleeves, with height adjustment clip, 150 mm long, Ø 8 mm

en - Items supplied

- 4 x E 456, Spring stays for MIS instruments
- 3 x E 464, Holders for injector jet E 454 for hollow instruments
Ø 20 mm, with height adjustment spring, length 65 mm, Ø 13 mm
- 1 x E 472, Spring clamp for injector jets Ø 4 mm

Optional accessories

The following and other accessories are available to order from Miele:

- E 457, Insert for 8 to 12 MIS instruments with holders for handles and tool inserts which cannot be dismantled, height 62 mm, width 192 mm, depth 490 mm
- E 499 for reprocessing suction and rinsing cannulae with a variety of connector types as well as Veress cannulae.

Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites. Ensure that any plastic wrappings, bags etc. are disposed of safely and kept out of the reach of babies and young children. Danger of suffocation.

To avoid the risk of accidents and damage to this mobile unit please read these instructions carefully before using it for the first time.
Keep these instructions in a safe place where they are accessible to users at all times.

 Please also read the Operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ This mobile unit is only for use in the areas of application as defined in the Operating instructions. Components such as the jets must only be replaced with Miele accessories or original parts. Any other application, modification or change is unauthorised and could be dangerous.
Miele cannot be held liable for damage caused by unauthorised use or incorrect operation.
- ▶ New mobile units and baskets must be washed in the washer-disinfector without a wash load prior to first use.
- ▶ Inspect all mobile units, baskets, modules and inserts daily as described in the "Maintenance" section in the Operating instructions for the washer-disinfector.
- ▶ Only items which have been declared by their manufacturer as suitable for machine reprocessing may be processed. The manufacturer's specific reprocessing instructions must be observed. Disposable instruments must not be put into the machine for processing.
- ▶ The processing programme used should include thermal disinfection, and must be suitable for the material quality of the wash load.
- ▶ To clean and disinfect hollow instruments thoroughly from the inside they must be attached to the relevant holders or injector nozzles on the injector manifold. The connectors must not come loose during a programme!
Hollow instruments which have come away from their connectors during the programme must be processed again.
- ▶ The last rinse should be carried out with demineralised water wherever possible.

Miele cannot be held liable for damage caused by non-compliance with these Warning and Safety Instructions.

en - Areas of application

For validation purposes please make sure that you follow the loading instructions given on the template.

⚠ To minimise the risk of injury, the mobile unit should be loaded from the back forwards and unloaded in the reverse order.

⚠ All MIS instruments must be dismantled for reprocessing according to the manufacturer's instructions. Shafted instruments with narrow lumens which cannot be dismantled must be pre-treated in accordance with their manufacturer's instructions.

⚠ Do not place any items for processing which will cover other items or trays which are overloaded on the lower level, as the cleaning solution will not be able to reach the items sufficiently.

Before connecting hollow instruments, please make sure that all hollow sections are free of blockages. Otherwise the cleaning solution will not be able to access all areas properly.

⚠ Any hollow items that have become disconnected from their injector nozzles, adapters or holders during processing must be reprocessed.

Instruments with monopolar or bipolar HF electrodes generally require manual pre-treatment, e.g. with 3% hydrogen peroxide solution. Refer to the manufacturer's instructions.

Instruments which cannot be dismantled, which have a rinsing connection can only be satisfactorily reprocessed if the manufacturer has provided specific measures for pre-treatment.

Before loading the machine and before starting a programme

- That the silicone holders in the injector bars are all present and undamaged.
- That the fittings, such as sleeves, jets and funnels are firmly pushed onto the silicone holders or tightly screwed into position.

⚠ In order that all fittings give a standardised wash pressure, all screw connectors must be fitted with jets, adapters or blanking screws.

Do not use damaged fittings such as jets, adapters or sleeves.

Silicone holders which are missing or damaged by fluid ingress must be replaced with new ones before the mobile unit, basket or module can be used again. In this case the silicone holders will standardise the wash pressure themselves.

Fittings which are not being used do not need to be replaced with blanking screws.

- Check that the mobile unit is correctly connected to the water supply in the washer-disinfector.

Examples of use

Irrigation sleeves Irrigation sleeves are suitable for reprocessing hollow instruments and instruments with the following external diameters:

- E 442, irrigation sleeves for instruments Ø 4 - 8 mm, 121 mm long, Ø 11 mm, cap with aperture Ø 6 mm
- E 443, irrigation sleeves for instruments Ø 8 - 8.5 mm, 121 mm long, Ø 11 mm, cap with aperture Ø 10 mm

The aperture can be varied using optional replacement caps:

- E 445, 12 caps for irrigation sleeves, aperture Ø 6 mm
- E 446, 12 caps for irrigation sleeves, aperture Ø 10 mm

The diameter of the instruments and apertures in the sleeves or caps should be matched to each other.

Shafts and tool inserts

- Place the shafts of MIS instruments in irrigation sleeves with caps.
- Place HF forceps and electrodes in irrigation sleeves with caps, with the functional ends pointing downwards.

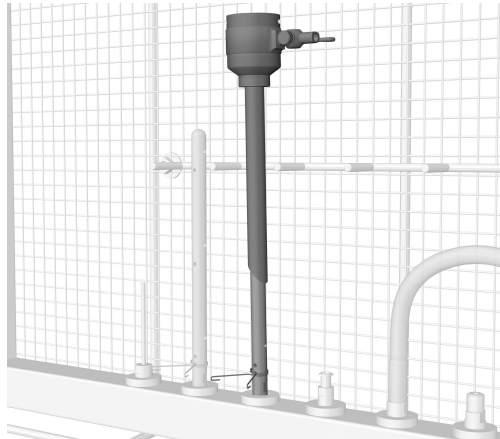
Shafted instruments which cannot be dismantled

- Place instruments which cannot be dismantled in an irrigation sleeve. The functional ends must be open; if necessary use an E 472 spring clamp to hold them open. Attach the side connectors for instruments with an E 448 hose adapter. Secure the hose adapter to the injector manifold using a silicone holder or with a screw fitting.

Trocars with lumens

- Push the trocar with lumen into the irrigation sleeve.
- Push trocar sleeves of less than 10 mm Ø into irrigation sleeves. Attach the side connector using an E 448 hose adapter. Secure the hose adapter to the injector manifold using a silicone holder or with a screw fitting.


Trocar sleeves Ø 10 to 20 mm


- Dismantle the trocar sleeves.

Process small parts such as seals or removable taps in a mesh tray with a lid.

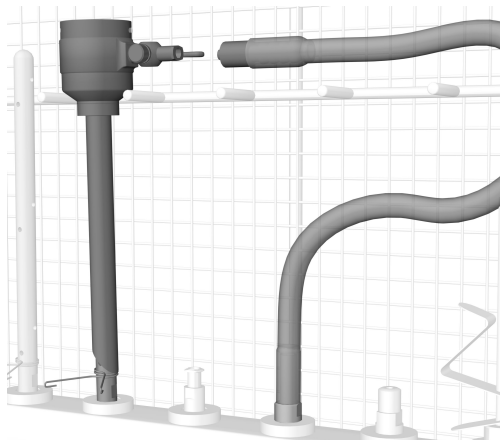- Place trocar sleeves Ø 10 to 15 mm onto an E 454 injector nozzle. For trocar sleeves Ø 15 to 20 mm use an E 464 adapter together with an E 454 injector nozzle.
- Position trocar sleeves with the spring clamps so that the cleaning solution can reach critical areas such as the valve seating through the side apertures.

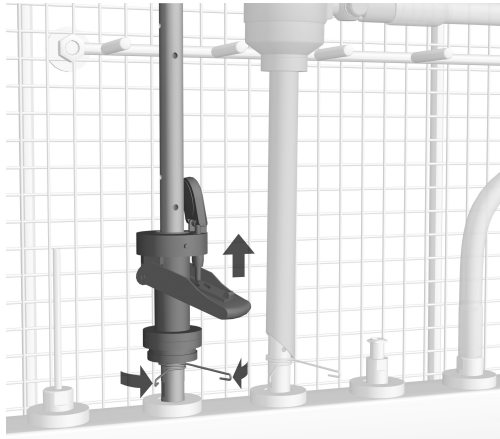
- Connect the side connector of the trocar sleeve to an E 448 hose adapter.
- Push the hose adapter into the injector manifold or into a connector in the water inlet tube.

- Place the valve onto an E 454 injector nozzle for trocar sleeves or E 453 with holding clamps.
- Position the valve with the spring clamp or holding clamp so that the cleaning solution can reach critical areas through the side apertures.

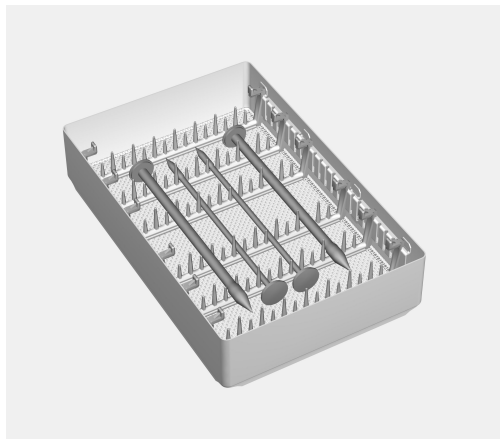
Trocars without lumens

Trocars without lumens or micro instruments can be reprocessed in the E 441/1 insert.


- Push the trocar into the retainers.
- Put the insert in a free place in one of the mobile units or baskets.

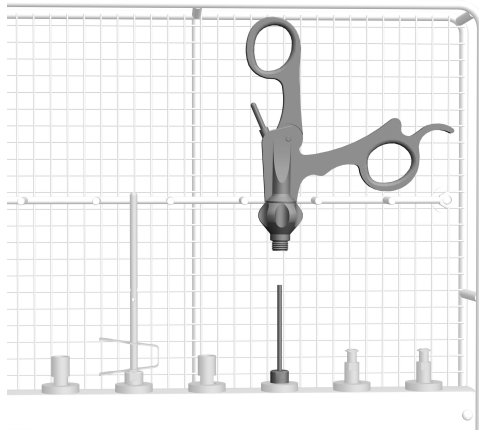Handles

- Fit contaminated handles onto an injector nozzle, e.g. E 452.

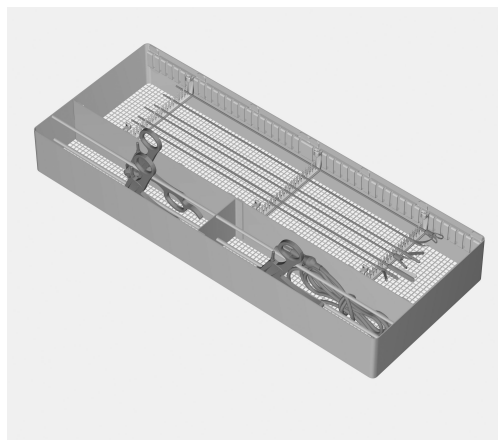Shafts

Shafts, HF cables and lightly soiled handles can be processed in the E 457 insert. The insert comes with its own Operating instructions.

Rigid fibre optics

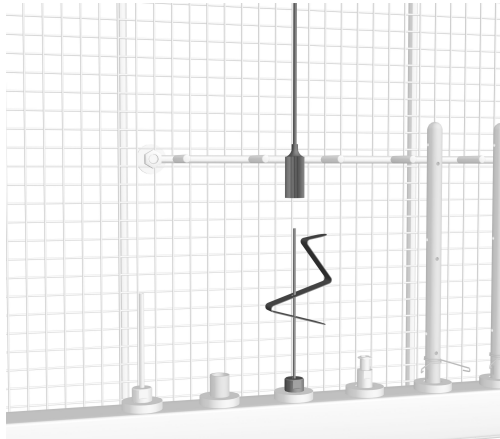
⚠ To avoid mechanical damage only process rigid fibre optics in inserts made by the rigid fibre optic manufacturer or in the special insert E 460.

- Put the insert in a free place in one of the baskets or inserts.

en - Areas of application


Veress cannulae

- Dismantling Veress cannulae
- Secure the exterior of the Veress cannulae onto an E 453 injector nozzle with E 472 spring clamps.

The top end of the Veress cannula is held in place by the wire support.

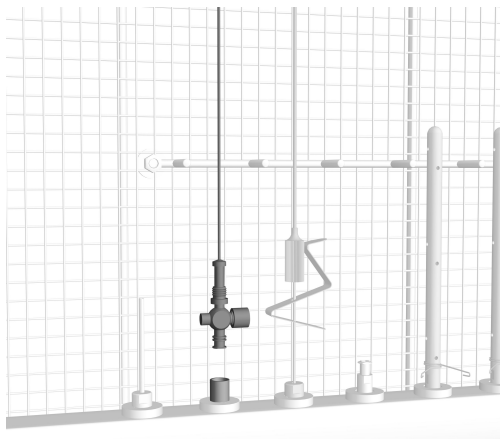
Alternatively an E 499 fitting can be used.

- Screw the interior of the Veress cannula into a Luer Lock adapter ♂. The tap must be open.


The top end of the Veress cannula is held in place by the wire support.

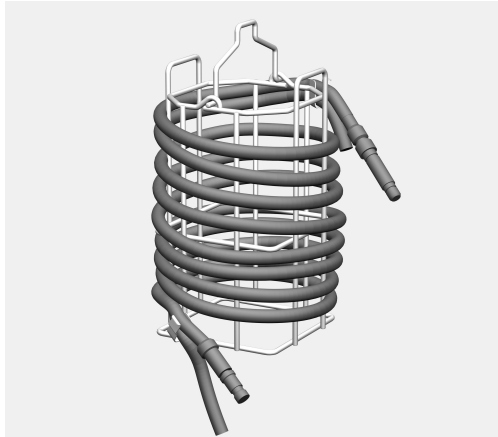Fibre optic cables and suction tubes

Fibre optic cables and suction tubes can be reprocessed using insert E 444. The insert comes with its own Operating instructions.

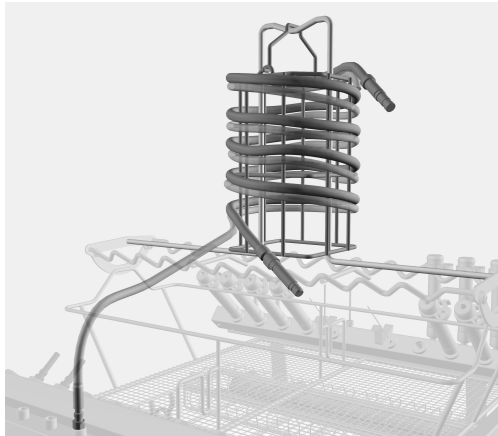
Camera cables must be fitted with protective caps according to the instructions of the manufacturer.


- Place the E 444 insert on the rack protruding upwards in the middle of the mobile unit.
- Connect the end of the suction tube to a suitable adapter so that the lumen is rinsed out upwards.

Make sure that the loose ends of fibre optic cables or suction tubes are not trapped in the mobile unit or in the door of the washer-disinfector.

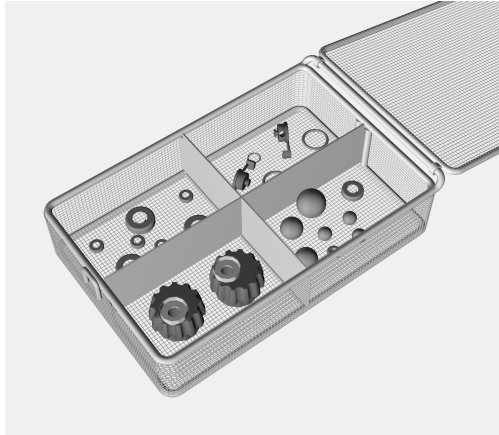Small items

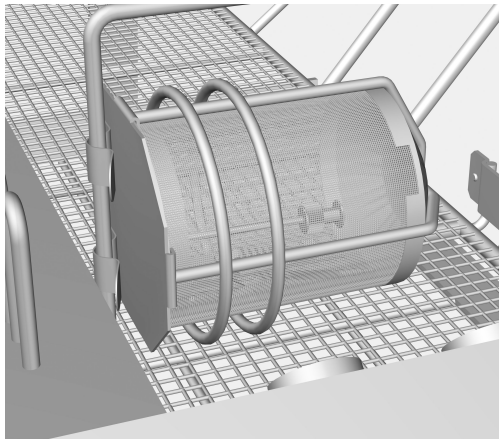- Process small items, such as seals, taps etc in mesh trays with lids such as E 451.
- Put the mesh tray with the lid closed in a free place in one of the baskets or inserts.

Make sure that extremely small items which could escape through the mesh or through the gap between the mesh tray and the lid are secured, e.g. in insert E 473/1.

- Secure insert E 473/1 to a strut either in the basket or in the mobile unit.

Fitting E 499 for suction and rinsing cannulae

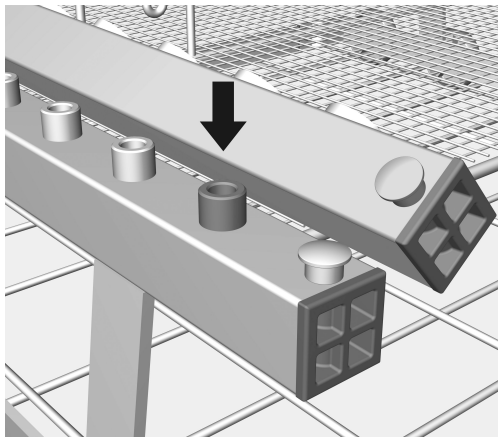
The E 499 is for reprocessing rinsing and suction cannulae, including suction breakers. It comes with its own instructions.

If several fittings are being used to reprocess hollow instruments at the same time, check that the number complies with validation requirements.

The maximum number specified for validation must not be exceeded.

Test point for measuring water pressure

The test point on the front left can be used for measuring water pressure for performance checks and for validation according to EN ISO 15883.

- To measure water pressure replace the existing fixture with a Luer Lock adapter, e.g. E 447.

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