PWA 8672

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en 4

fr 16

Information on the installation plan	4
Definition of terms	4
Machine dimensions	5
Connections	6
Connections	7
On-site connections	7
External dispensing	7
Installation notes	8
Surrounding area	8
Vapor barrier for built-under appliances.....	8
Electrical connection	8
Equipotential bonding and grounding.....	8
Electromagnetic compatibility (EMC)	9
Network connection	10
Water connection	11
Water inlet	11
Drainage	11
Technical details.....	12
Electrical connection	12
Water inlet	12
Drainage.....	13
Dimensions	13
Weight.....	13
Emission values	14
Heat dissipation	14
Sound pressure level.....	14
Operating conditions	14
Storage and transportation conditions	14

en - Information on the installation plan

For safe installation and commissioning of the dental instrument washer, please read the installation plan, the service documentation, and the operating instructions.

This installation plan includes the dimensions of the dental instrument washer, the technical data, and the requirements to be met by the customer for installation.

The dimensions in the figures are given in mm. Dimensions that deviate from this are labeled accordingly.

The machine features vary depending on the model and can be extended with additional options. The installation plan describes the maximum amount of possible features. Individual connections are not required for each item of optional equipment.

Installation requirements

Installation and commissioning work should be performed by Miele Customer Service or a suitably authorized technician in accordance with local and national safety regulations.

Installation must comply with valid regulations, legal requirements, relevant standards, and accident prevention regulations.

Definition of terms

The machine

In this installation plan, the dental instrument washer is referred to as “the machine”.


Wash water

The term “wash water” refers to water or to a mixture of water and process chemicals.


Process chemicals

All media dispensed during a program sequence are generally referred to as process chemicals, e.g., cleaning agents.


Front view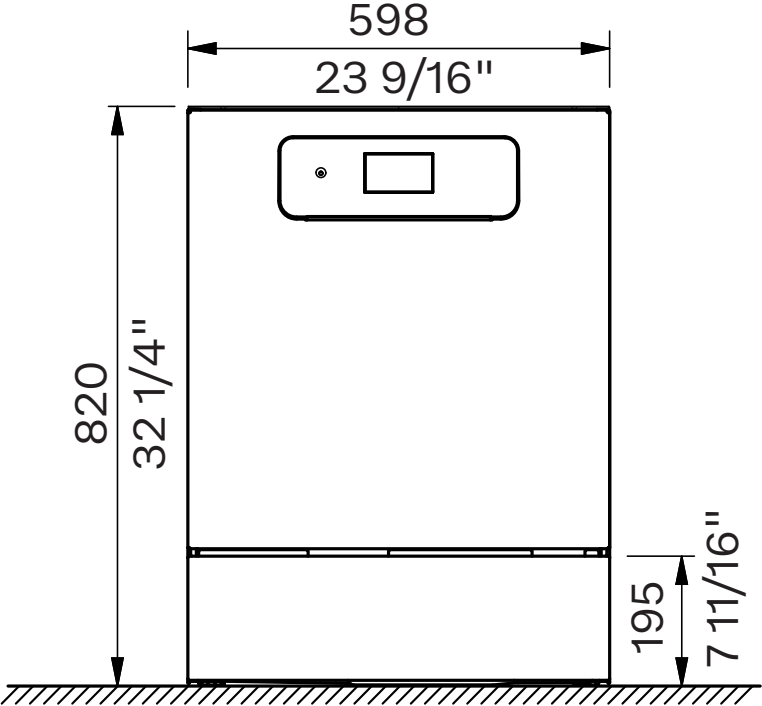

Side view

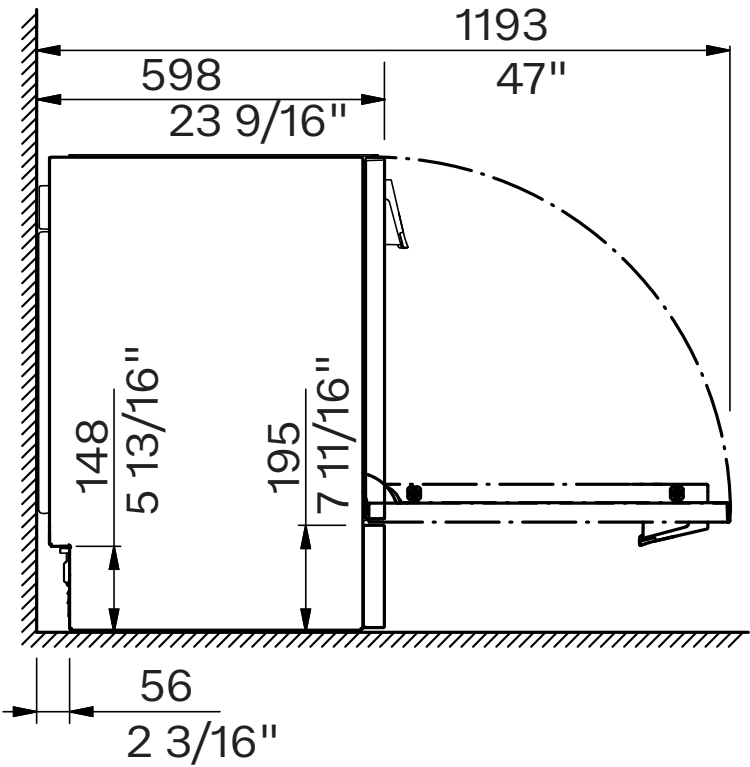
Rear view

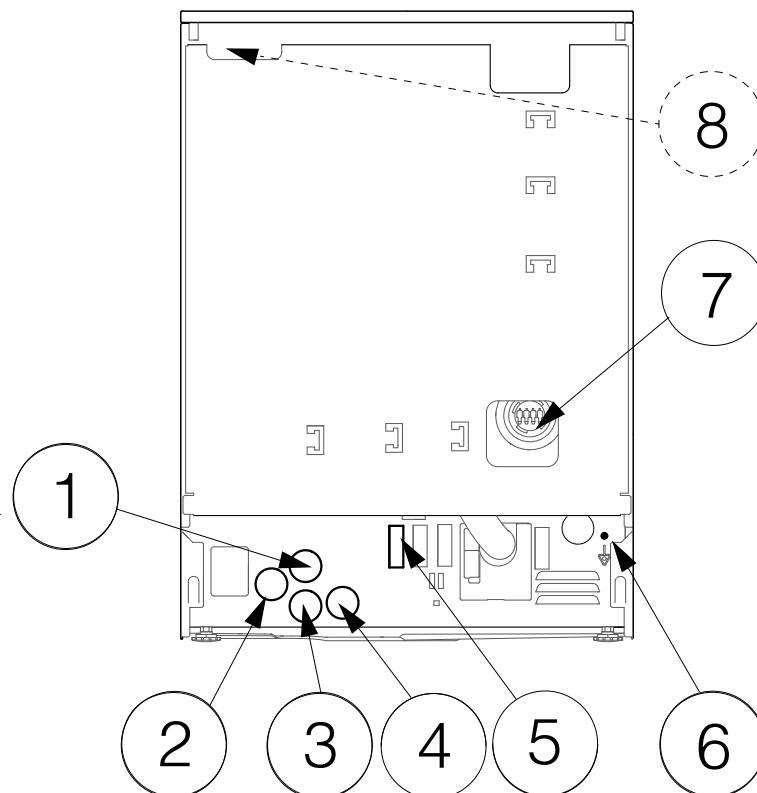
- ① Power supply cord
- ② Hot water
- ③ Cold water
- ④ Waste water
- ⑤ External dispensing, power supply connection
- ⑥ Equipotential bonding and grounding
- ⑦ External dispensing, connection for dispensing hose
- ⑧ Network connection (optional)

On-site connections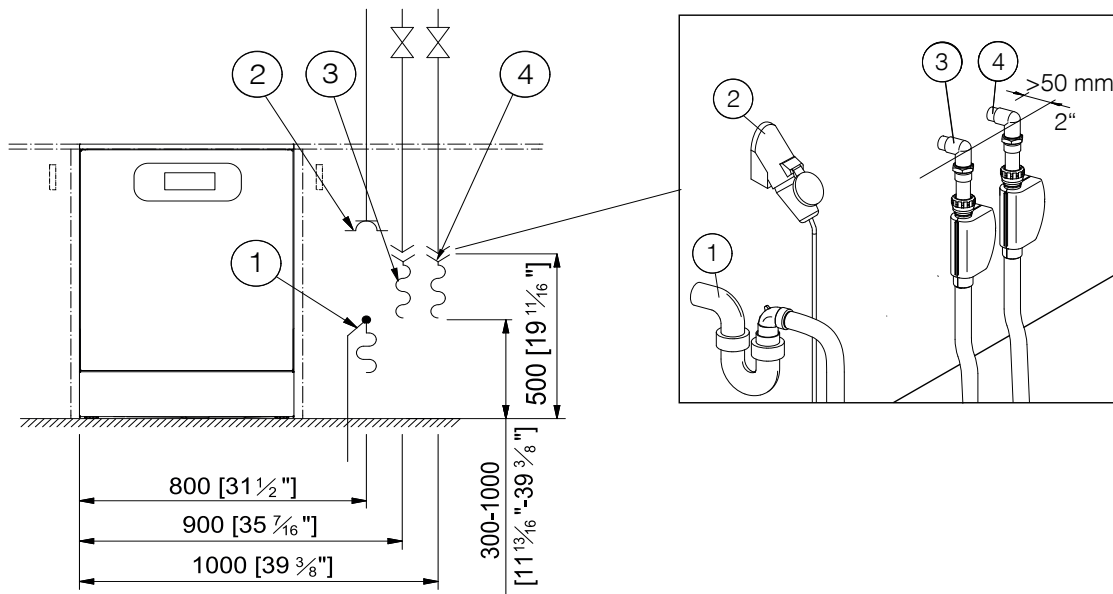

- ① Waste water
- ② Power supply cord
- ③ Hot water
- ④ Cold water

External dispensing

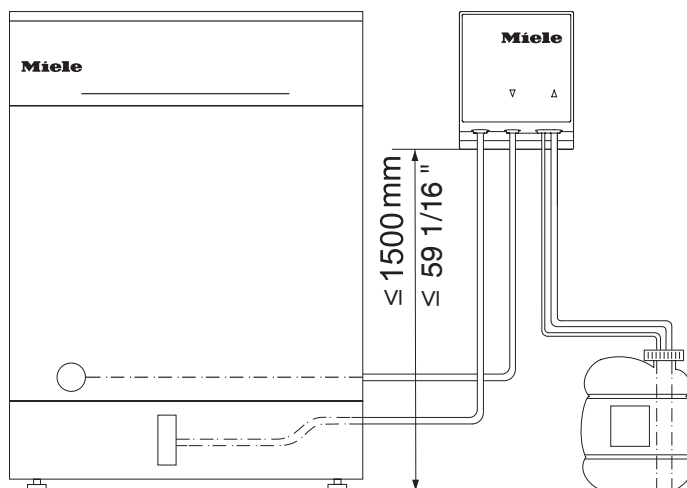Max. delivery head	4' 11" (1.5 m)
Length of dispenser hose, DOS module to suction wand	5' 11" (1.8 m)
Length of dispenser hose, back of appliance to DOS module	9' 2" (2.8 m)
Length of power cord, back of appliance to DOS module	9' 2" (2.8 m)

Place the container on the floor next to the machine or in an adjacent cabinet.
The container must not be placed on top of or above the machine.

Surrounding area

Condensate can build up in the area surrounding the machine. Any cabinetry and fittings in the room must, therefore, be suitable for purpose.

Vapor barrier for built-under appliances

The enclosed vapor barrier protects the countertop from damage caused by water vapor that may escape when the door is opened. Place the vapor barrier above the door under the countertop.

Electrical connection

All work on the electrical connection must be carried out by Miele Service or a qualified electrician.

The customer's installation of the machine must comply with over-voltage category CAT II in accordance with IEC 61010-1.

The voltage may deviate from the rated voltage by max. $\pm 10\%$.

Power switch

Install a power switch capable of disconnecting the machine at all poles at the customer's site. The power switch must:

- Have a contact gap of at least 1/8" (3 mm).
- Be designed to operate at the rated current of the machine.
- Be able to be locked in the zero position.
- Be accessible after the machine has been installed.

The power cord must be laid protected from the risk of thermal damage.

Plug connection

The machine should be connected to the electricity supply via a electrical outlet.

Hard-wired

Alternatively, the machine can be connected via a hard-wired connection.

Residual current device (RCD)

For added safety, each machine should be protected by a type A residual current device with a trip current of 30 mA.

The residual current device must be installed so that it is easily accessible.

Equipotential bonding and grounding

The machine has been set up for equipotential bonding and grounding. The connection screw (M 8) is located on the rear of the machine. Equipotential bonding and grounding should be carried out if possible on site.

Electromagnetic compatibility (EMC)

The machine has been tested for electromagnetic compatibility (EMC) in accordance with EN 61326-1 and is suitable for operation in commercial environments, such as hospitals, medical practices, and laboratories plus other similar environments which are connected to the domestic electrical supply.

The machine's high frequency (HF) energy emissions are very low and are therefore unlikely to interfere with other electronic machines in the vicinity.

Flooring in the installation area must be wood, concrete, or tiled. Synthetic flooring must be able to withstand a relative humidity level of at least 30% to minimize the risk of electrostatic discharges.

The quality of the power supply should comply with that found in a typical commercial or hospital environment. Check that the power supply voltage is within a range of +/-10% of its nominal value.

Network connection

The machine can be integrated into a customer's network via an Ethernet interface.

The operator is responsible for the secure configuration and operation of the network. Creating a secure configuration reduces the scope for the machine to be attacked. Non-secure configurations can make it easier for potential attackers to gain unauthorized access.

 Unauthorized access poses a risk.
Settings in the machine may be changed as a result of unauthorized access via the network.
Under no circumstances should it be possible to access the machine via public or unsecured networks, either directly or indirectly (e.g., using port forwarding).
The IP addresses from 192.168.10.1 to 192.168.10.255 are reserved for Miele Service. They must never be set in the internal network.

Please also note the following recommendations for the network configuration of the machine:

1. Configure the network in which the machine is located with as many restrictions as possible:
 - Only allow machines or persons access to the network if absolutely necessary.
 - Use a VLAN for network segmentation, for example.
2. Make sure to create a secure configuration for all machines connected to the network, e.g., with the help of:
 - The information on network security in the operating instructions for the connected machines.
 - The recommendations of USA: Cybersecurity & Infrastructure Security Agency (CISA - www.cisa.org), Canada: Communications Security Establishment Canada (<https://www.cse-cst.gc.ca>).
3. Pay particular attention to the following areas:
 - user administration
 - staff entering and exiting
 - authorization
 - authentication
 - update management, etc.

Water connection

Water inlet

The machine must only be connected to fully vented pipework. A brief increase in the water pressure can damage components of the machine.

The quality of the incoming water must at least correspond to the drinking water regulations of the country in which the machine is being operated.

The machine must be connected to cold and hot water. If hot water is not available, the inlet hose for hot water must be connected to the cold water supply. For this purpose, a Y-piece can be used on the cold water inlet on the customer side.

The connections with shut-off valves at the customer's site must have 3/4" (MGHT) outer threads that seal the surface.

The shut-off valves must be accessible after the machine has been installed.

Drainage

The machine should be connected to a separate on-site drainage system for the machine only. If a separate connection is not available, it is recommended to connect the hose to a dual-chamber siphon. The drain lines must be heat resistant to at least 158°F (70°C).

If the hose is to be directly connected to the drainage system on site, use the hose clip supplied in the accessory pack.

The on-site connector for the drain hose can be used for various hose diameters. If the connector extends more than 1 1/4" (30 mm) into the drain hose, it must be shortened. Otherwise, the drain hose can become clogged.

Lay the drain hose so that it does not kink and is not being subjected to pressure or tugged.

If the on-site drain connection is situated lower than the guide path for the lower basket rollers in the open door, a siphoning effect during a program can cause the appliance to empty itself of water. In this case, lay the drain hose with a bend in it so that its highest point is at least level with the guide path for the lower basket rollers.

en - Technical details

Electrical connection

	Standard	Convertible *
Voltage	2 AC 208 V	3 AC 208 V
Frequency	60 Hz	60 Hz
Fuse rating	2 x 30 A	3 x 20 A
Connection	NEMA L6-30P	NEMA L15-20P
Power cord length	1.9 m/6.2 ft	1.9 m/6.2 ft
Power cord cross-section	3 x AWG10	4 x AWG12
Heater rating	5.4 kW	5.4 kW
Total rated load	6.2 kW	6.2 kW

* conversion by Miele Service

Water inlet

Water connections		
- Cold water	1	1
- Hot water	1	1
Water temperature		
- Cold water	5–20 °C	41–68 °F
- Hot water	5–65 °C	41–149 °F
Connection hose length	1.3 m	51 3/16"
Connection at customer's site with shut-off valve, outer thread that seals the surface	3/4" Male Garden Hose Thread MGHT	3/4" Male Garden Hose Thread MGHT
Water connection pressure	200–1000 kPa	29–145 psi
Flow rate	2.0–7.5 l/min	0.5–1.98 gpm
Water hardness	0 – 10.7 mmol/l	0 – 62.56 gpg
Chloride content	<100 mg/l	<5.85 gpg sodium chloride

Drainage

Drain hose length	1.4 m	55 1/8"
Max. drain hose length	4 m	157 1/2"
Max. delivery head from floor	1 m	39 3/8"
Max. water temperature	70 °C	158 °F
Max. transient flow rate	16 l/min	4.23 gpm
Hose inner diameter (Ø)	22 mm	7/8"
Drain line	≥ DN 50	≥ DN 50

Dimensions

Height	820 mm	32 1/4"
Height adjustment	+60 mm	+2 3/8"
Width	598 mm	23 9/16"
Depth	598 mm	23 9/16"
Depth with control panel	639 mm	25 3/16"
Depth with open door	1193 mm	47"

Weight

Net weight	68 kg	150 lbs
Max. operating weight	126.5 kg	279 lbs
Maximum load capacity of open door	37 kg	82 lbs
Max. individual load (foot Max. individual load (foot Ø 38 mm/1.5"))	320 N	320 N
Max. distributed load (load-bearing capacity)	3.5 kN/m ²	3.5 kN/m ²

en - Technical details

Emission values

Heat dissipation

Heat output into the room	5.4 MJ/h	5118.2 BTU
---------------------------	----------	------------

Sound pressure level

Sound pressure level LpA during cleaning and drying	<65 dB(A) re 20 µPa
---	---------------------

Operating conditions

Ambient temperature	5–40°C	40–105°F
Relative humidity:		
Minimum	10%	10%
Maximum for temperatures up to 201°F (31°C)	80%	80%
Decreasing linearly for temperatures up to 104°F (40°C)	50%	50%
Max. altitude above sea level	2,000 m *	6,561 ft *

If installed above 4,921.26 ft/1,500 m, the boiling point of the wash water will be lower. In this case, Miele Service must adjust the disinfection temperature and the holding time.

Storage and transportation conditions

Ambient temperature	-20 – +60 °C	-4 – +140 °F
Relative humidity	10–85 %	10–85 %
Air pressure	500–1060 hPa	500–1060 hPa
Weight including packaging max.	81.5 kg	180 lbs
Maximum floor load, transport route	815 kN	815 kN

Site-access dimensions including transport pallet

Height	1000 mm	39 3/8"
Width	1170 mm	46 1/16"
Depth	740 mm	29 1/8"

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U.S.A.
Miele, Inc.
Miele Limited | limitée