# XKM 3000 L Med

|           |                                                 |
|-----------|-------------------------------------------------|
| <b>de</b> | Montageanweisung Kommunikationsmodul            |
| <b>da</b> | Monteringsanvisning kommunikationsmodul         |
| <b>en</b> | Fitting instructions Communication module       |
| <b>es</b> | Instrucciones de montaje Módulo de comunicación |
| <b>fi</b> | Asennusohje Tiedonsiirtomoduuli                 |
| <b>fr</b> | Notice de montage Module de communication       |
| <b>hr</b> | Upute za ugradnju Komunikacijski modul          |
| <b>it</b> | Istruzioni di montaggio Modulo di comunicazione |
| <b>nl</b> | Montagehandleiding communicatiemodule           |
| <b>no</b> | Monteringsveiledning Kommunikasjonsmodul        |
| <b>pt</b> | Instruções de montagem Módulo de comunicação    |
| <b>sl</b> | Navodila za montažo Komunikacijski modul        |
| <b>sv</b> | Monteringsanvisning Kommunikationsmodul         |

|                 |     |
|-----------------|-----|
| <b>de</b> ..... | 4   |
| <b>da</b> ..... | 19  |
| <b>en</b> ..... | 34  |
| <b>es</b> ..... | 49  |
| <b>fi</b> ..... | 64  |
| <b>fr</b> ..... | 79  |
| <b>hr</b> ..... | 94  |
| <b>it</b> ..... | 109 |
| <b>nl</b> ..... | 124 |
| <b>no</b> ..... | 139 |
| <b>pt</b> ..... | 154 |
| <b>sl</b> ..... | 169 |
| <b>sv</b> ..... | 184 |

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|                                        |    |
|----------------------------------------|----|
| <b>Notes about these instructions</b>  | 34 |
| <b>Intended use</b>                    | 35 |
| Intended use                           | 35 |
| Queries and technical problems         | 35 |
| <b>Items supplied</b>                  | 36 |
| Items supplied                         | 36 |
| Optional accessories                   | 36 |
| <b>Guide to the appliance</b>          | 37 |
| Overview                               | 37 |
| Switching on and off                   | 37 |
| Control LEDs                           | 38 |
| Reset button                           | 38 |
| Reset operational faults               | 38 |
| Reset factory settings                 | 38 |
| <b>Warning and Safety instructions</b> | 39 |
| <b>Fitting and installation</b>        | 40 |
| Module slot                            | 40 |
| Installation                           | 41 |
| <b>Configuring the interface</b>       | 42 |
| Configuring the network interface      | 42 |
| Opening the menu                       | 42 |
| Check the connection status            | 43 |
| Selecting settings                     | 43 |
| Accept configuration                   | 44 |
| Fault 559                              | 45 |
| Web interface                          | 45 |
| <b>Technical data</b>                  | 46 |
| <b>Caring for the environment</b>      | 47 |
| Disposal of the packing material       | 47 |

### Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

### Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

### Additional information and comments

Additional information and comments are contained in a simple frame.

### Operating steps

Operating steps are indicated by a black square bullet point.

#### Example:

- Select an option using the arrow buttons and save your choice with *OK*.

### Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

#### Example:

Settings  menu.

### **Intended use**

With Miele washer-disinfectors, cleaning processes can be documented. To enable this, Miele washer-disinfectors are equipped with a module slot on the back to take a Miele communication module.

The XKM 3000 L Med communication module enables the installation of an Ethernet interface.

For connection to a WiFi network, the Miele communication module can be connected to a wireless access point provided by you, using a cable.

Only use terminal devices (PCs etc.) which are approved according to EN/IEC 60950.

For more information on suitable software for process documentation please contact Miele.

Configuration of the interface must only be carried out by suitably qualified and competent personnel.

The communication module can be used in the following washer-disinfectors:

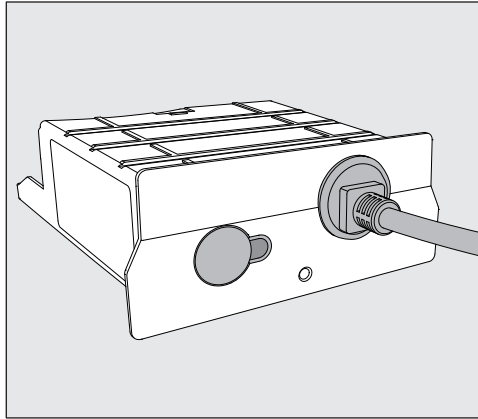
- PG 8562
- PG 8581
- PG 8582
- PG 8582 CD
- PG 8583
- PG 8583 CD
- PG 8591
- PG 8592
- PG 8593

The communication module is generally referred to as the module in these Operating instructions.

### **Queries and technical problems**

In the event of queries or technical problems, please contact Miele. Contact details can be found at the end of the operating instructions for your cleaning machine or at [www.miele.com/professional](http://www.miele.com/professional).

### Items supplied


- Communication module XKM 3000 L Med

### Optional accessories

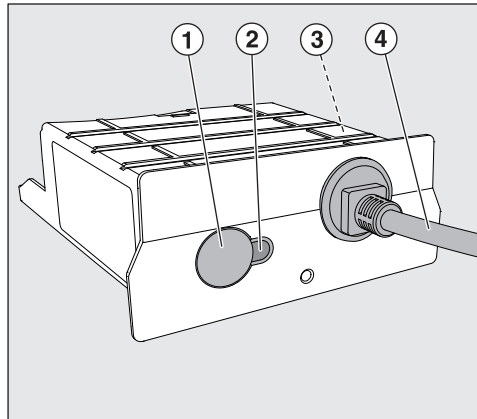- Segosoft Miele Edition, Process documentation software for use in a PC or with a network connection.
- Additional licences for Segosoft Miele Edition
- Segosoft Miele Edition: USB version, Process documentation software for data transfer using a USB stick
- Various network cables

### Overview

- ① Reset button
- ② Indicator LED,  
Indicates ready-for-use status after installation and active communication during operation
- ③ Data plate (located on the side)
- ④ Data cable

### Switching on and off

The module does not have a separate On/Off button. It is switched on and off in conjunction with the washer-disinfector.

Every time it is switched on the module takes about 90 seconds to initialise itself. Until this has happened data cannot be transferred and interface settings cannot be changed.

### Control LEDs

The control LEDs (light emitting diodes) give status information about the communication module.

| LED                                                        | Status                                                                                                                                        |
|------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------|
| <b>LIT UP GREEN</b>                                        | The module is ready for operation.                                                                                                            |
| <b>FLASHES GREEN</b>                                       | The module is receiving or transmitting data.                                                                                                 |
| <b>LIT UP YELLOW</b>                                       | The module is receiving network data, the IP address set is, however, not recognised. The interface may need to be reconfigured.              |
| <b>FLASHES YELLOW</b>                                      | Network connection being made.                                                                                                                |
| <b>LIT UP RED</b>                                          | Resetting in progress.                                                                                                                        |
| <b>FLASHES RED</b>                                         | The module is initialising. This process can take up to 90 seconds.                                                                           |
| <b>FLASHES alternately YELLOW and RED or GREEN and RED</b> | The module is in service mode and is not ready for use. Switch the washer-disinfector off and then back on again to come out of service mode. |

### Reset button

#### Reset operational faults

If the module experiences an operational fault during operation, the reset button can be used to get it operational again.

- To do so press the reset button for approx. 1-2 seconds.

 Do not press it for any longer than this as this would reset the module to its factory default settings and you would then have to reconfigure the interface.

The module will then restart (reset). This process can take up to 90 seconds. The LED next to the button will light up red during this process.

#### Reset factory settings

If you cannot get the module operational again with the reset button you will have to reset the module to its factory default settings.

 If the factory default settings are reset the interface will have to be reconfigured.

- Press and hold the reset button for approx. 5-6 seconds.

The LED will start to flash red. Resetting to default settings takes at least 90 seconds.

To avoid the risk of accidents and damage to this communication module please read these instructions carefully before using it for the first time.  
Keep these instructions in a safe place.

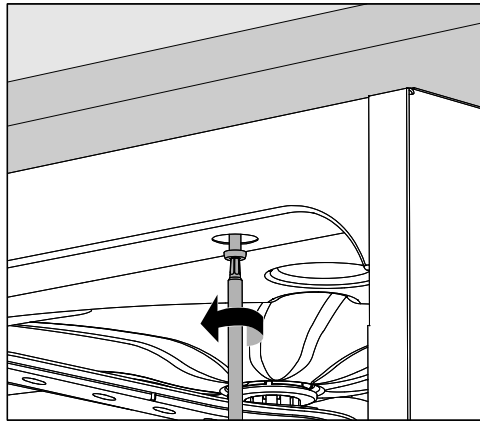
 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ This communication module must only be used for applications specified in the operating instructions. Changes or modifications to it and incorrect use of it are not permitted as this can be dangerous.
- ▶ The communication module must only be installed, commissioned and serviced by Miele service or by a suitably qualified person. Repairs are only to be undertaken by Miele Service. Repair work by unqualified persons could be dangerous.
- ▶ Check the communication module carefully before installation for any visible damage. A damaged module could be dangerous and must not be installed.
- ▶ Before installing the communication module compare the connection data (voltage and frequency) on the data plate with that on the data plate of your washer-disinfector. If the data does not match, the module could sustain damage. If in any doubt, consult an electrician.
- ▶ The communication module housing must not be opened on any account.  
Touching live electrical components or altering the electrical or mechanical components is dangerous and could cause the module to malfunction.

### Module slot

The module is inserted into the module slot on the back of the washer-disinfector.

- If the washer-disinfector is installed in a niche:
  - Pull the washer-disinfector forwards out of the niche.
- If the washer-disinfector is installed under a worktop:
  - Open the door.


- Unscrew the fixing screws.
- Pull the washer-disinfector out from under the worktop.

### Installation

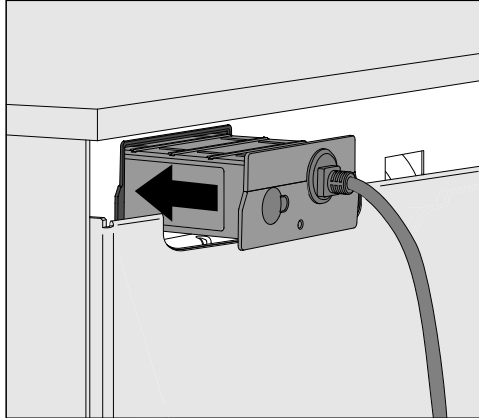⚠ Before installing the communication module compare the connection data (voltage and frequency) on the data plate with that on the data plate of your washer-disinfector. If the data does not match, the module could sustain damage. If in any doubt, consult an electrician.

- Disconnect the washer-disinfector from the mains electricity supply.
- Remove any stickers or protective caps as necessary from the slot aperture.

- Push the module into the module slot. Gently push the module until it clicks into place.
- Establish a cable connection between the module and your terminal device.  
Make sure that the cable is laid correctly and that it is not kinked or trapped anywhere. It must be protected from sharp edges and not subjected to tension.

Only use terminal devices (PC, printers etc.) which comply with EN/IEC 60950.

- Connect the washer-disinfector to the electrical supply.  
The module is supplied with electricity via the washer-disinfector. An additional electrical connection is not necessary.

### Configuring the network interface

⚠ The interface must only be configured by suitably qualified and competent persons.

#### Opening the menu

- Switch the washer-disinfector on using the ⏻ button.

You must wait at least 90 seconds before starting to configure the interface.

- Open the menu as follows:

☰ button

- ▶ Further settings
  - ▶ Interface

- Select the Ethernet interface and confirm your choice with OK.

The following options are available:

- Module status

Connection status displayed.

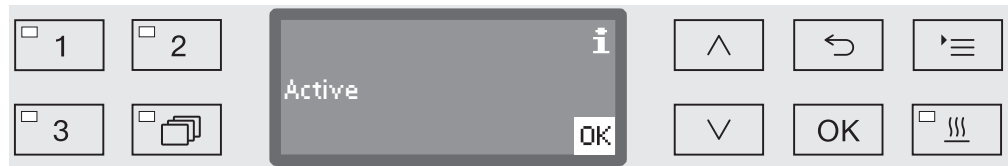
- DHCP

The Ethernet interface can either be implemented via a Dynamic Host Configuration Protocol (DHCP) or by setting one of the following parameters:

- IP address
- Subnet mask
- Standard gateway
- DNS server automatic
- DNS server 1
- DNS server 2
- Port type
- Port

### Check the connection status

- Select the Module status option and confirm your choice with **OK**.

One of the following messages may appear in the display:

- Active

The module is ready for operation.

- Initialising, module not yet active

The module has not finished initialising. This process takes approx. 90 seconds.

- Inactive

A module has not be recognised.

Check the module installation. Switch the washer-disinfector off and then back on again.

If the message appears in the display again call Miele Service.

### Selecting settings

Document all settings in case of a subsequent Service call requirement.

For manual entries DHCP must first be deactivated.

- Select the DHCP option and confirm your choice with **OK**.

- Select the Off option and confirm your choice with **OK**.

The following settings are then available:

- IP address
- Subnet mask
- Standard gateway
- DNS server automatic
- DNS server 1
- DNS server 2
- Port type
- Port

- Select an option and confirm your choice with **OK**. You can then carry out settings as shown in the display.

## en - Configuring the interface

The following display is an example of entering IP address, Subnet mask etc.:

The current setting is shown in the middle line. This can be altered using the options shown in the bottom line. The top line shows the selected option. The following input options are available:

- Numbers from 0 to 9.
- Decimal point “.”.
- Use the  symbol to delete the last position.
- Use the OK symbol to save your setting. The display will then return to the main menu.
- The  symbol in the display or the  button end the process without saving the setting. The display reverts to the initial menu.

Always enter the IP address, the Subnet mask etc. fully. For instance if the IP address is 192.168.1.23 you must enter 192.168.001.023.

- Use the arrow buttons  (right) and  (left) to move the cursor to the option you require.
- Confirm each entry with OK.
- Save your settings by selecting the OK symbol in the display and then confirm your choice with the OK button.

### Accept configuration

The interface configuration is only effective after the washer-disinfector has been restarted.

- Wait approx. 10 seconds before switching the washer-disinfector off for a restart.
- Switch the washer-disinfector off and then back on again with the  button.
- Wait approx. 90 seconds for the module to initialise.
- Then carry out a functional test of the interface.

### Fault 559

If the fault message Fault 559 is displayed during commissioning or a programme is interrupted with this message, the RS232 interface is activated and must be deactivated.

- Open the menu for configuring the RS232 interface via Further settings / Interface.
- Select the Ethernet option and confirm your choice with OK.

The RS232 interface is deactivated. After around 90 s, the module is ready for operation.

### Web interface

The XKM 3000 L Med module has its own web interface, which is used for accessing general data about the washer-disinfector, changing network settings and loading updates to the module.

The interface can be accessed from any workstation in your local network. To do so there must be a configured serial interface in the machine's controls, a current web browser and a network connection to the machine. The output language is English.

- Enter the IP address for your washer-disinfector in the address line in your browser.

The web interface for the module will continue to be shown in your browser. Data can take up to 60 seconds to transfer from the washer-disinfector to the web interface.

The web interface is arranged as follows:

#### - **Appliance information**

This page contains general information about the washer-disinfector, e.g. model number (*Model number*), serial number (*Serial number*) or MAC Address (*MAC address*).

#### - **System settings**

Data about the interface, such as the IP address (*IP address*) or the subnet mask (*Subnetmask*) can be found on this page. This data can be changed. It is password protected.

#### - **Password settings**

You can change your password on this page.

#### - **Firmware update**

If necessary, updates to the module's software can be loaded from this page. This function is password protected.

The default password is available from the Service department.

If this password has been changed it will revert to the factory default one if the module is reset.

## en - Technical data

---

|                                            |                                                                                                             |
|--------------------------------------------|-------------------------------------------------------------------------------------------------------------|
| Voltage,<br>Connected load,<br>Fuse rating | See data plate                                                                                              |
| Mains cable                                | Approx. 1.5 m                                                                                               |
| Transfer speed                             | 10/100 Mbit/s                                                                                               |
| Temperature during operation               | Up to max. 80 °C                                                                                            |
| Test certificates awarded                  | UL Recognised Component  |
| CE mark                                    | Low Voltage Directive 2006/95/EC,<br>EMC Directive 2004/108/EC,<br>RoHS Directive 2011/65/EU                |
| Manufacturer's address                     | Miele & Cie. KG, Carl-Miele-Straße 29,<br>33332 Gütersloh, Germany                                          |

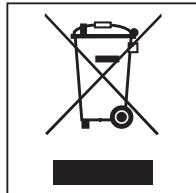
### Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

### Disposing of your old appliance

Electrical and electronic appliances contain many valuable materials. They also contain certain materials, compounds and components which were essential for their correct functioning and safety. These could be hazardous to human health and to the environment if disposed of with household waste or if handled incorrectly. Please do not, therefore, dispose of your old appliance with household waste.

Instead, please make use of officially designated collection and disposal points to dispose of and recycle electrical and electronic appliances in your local community, with your dealer or with Miele, free of charge. By law, you are solely responsible for deleting any personal data from the old appliance prior to disposal. You are legally obliged to remove any old batteries which are not securely enclosed by the appliance and to remove any lamps without destroying them, where this is possible. These must be taken to a suitable collection point where they can be handed in free of charge. Please ensure that your old appliance poses no risk to children while being stored for disposal.

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