A 800

de	Gebrauchsanweisung Filterrohr
da	Brugsanvisning Filterrør
en	Operating instructions Tubular filter
es	Instrucciones de manejo Tubo filtrante
fi	Käyttöohje Suodatinputki
fr	Mode d'emploi Tuyau de filtration
hr	Upute za uporabu Filtarska cijev
it	Istruzioni d'uso Tubo filtro
nl	Gebruiksaanwijzing filterbuis
no	Bruksanvisning Filterrør
pt	Instruções de utilização Tubo filtro
sl	Navodila za uporabo Filtrska cev
sv	Bruksanvisning Filterrör

de	4
da	14
en	24
es	34
fi	44
fr	54
hr	64
it	74
nl	84
no	94
pt	104
sl	114
sv	124

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Cleaning the tubular filter 30

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow buttons and save your choice with *OK*.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

This A 800 tubular filter is used for filtering the wash and rinse water. To do this the tubular filter is fitted in a special injector tube in Miele mobile units and baskets.

Please observe the operating instruction manuals supplied with the mobile units and baskets.

Queries and technical problems

In the event of queries or technical problems, please contact Miele. Contact details can be found at the end of the operating instructions for your cleaning machine or at www.miele.com/professional.

- Tubular filter A 800, Length 405 mm, Ø 22.5 mm

Optional accessories

- A 804, cleaning brush for tubular filter A 800, length 500 mm, brush head: length 100 mm, Ø 20 mm

Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

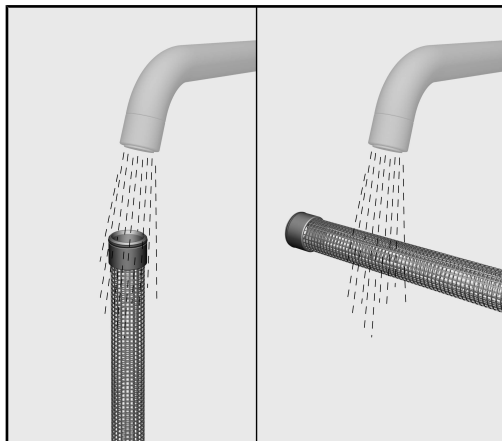
To avoid the risk of accidents and damage to this tubular filter please read these instructions carefully before using it for the first time.

Keep these instructions in a safe place where they are accessible to users at all times.

 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ The tubular filter must only be used for filtering the wash and rinse water in particular Miele mobile injector units and baskets. Alterations, conversions and modifications are not permitted.
- ▶ The tubular filter captures particles ≥ 0.07 mm. Smaller particles can pass through the filter. For this reason the user should inspect narrow lumen items to eliminate any risk.
- ▶ Check the tubular filter before use for visible damage or external soiling. A damaged or soiled tubular filter must not be used under any circumstances.
- ▶ Clean the tubular filter at least once a week to ensure it maintains the correct filtration efficiency.
- ▶ The O-rings at either end of the tubular filter provide the correct seal. The tubular must not be used without these O-rings. Worn O-rings must be replaced with original Miele O-rings only as they are the only ones suitable for use in your machine.

Before using for the first time

- Rinse the tubular filter before first use under running water to remove any possible soiling.
- Carry out a visual check to make sure there is no visible damage to the tubular filter. A damaged tubular filter must not be used.

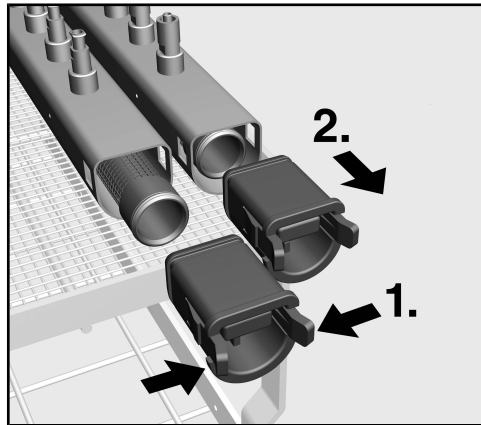
The O-rings at either end of the tubular filter provide the correct seal. The tubular filter must not be used without these O-rings.


- Check that the O rings are present and not damaged.
- Carry out a visual check to make sure the injector tube is free of any possible soiling. If necessary rinse the injector tube through before inserting the tubular filter.

Inserting the tubular filter

⚠ Before inserting the tubular filter please make sure that it is not visibly damaged or soiled.
A damaged or soiled tubular filter must not be used under any circumstances.


D-profile tube

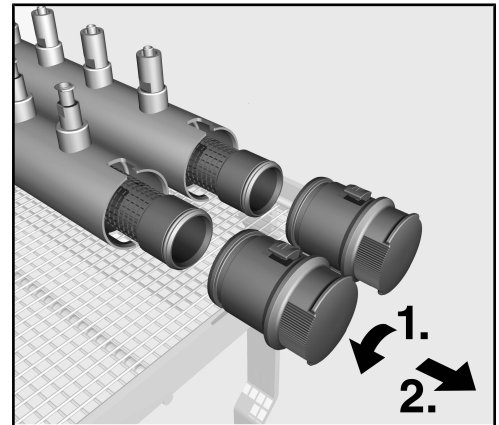
Round tube

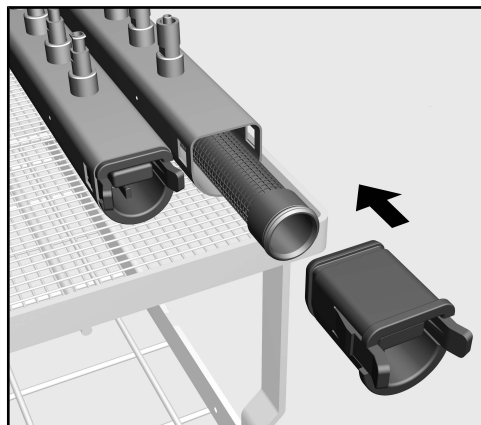- *D-profile tube:* 1. Press the tabs inwards.
Round tube: 1. Unscrew the locking cap on the injector tube in an anti-clockwise direction.
- 2. Then pull the locking cap off.
- Push the tubular filter fully into the injector tube.

D-profile tube

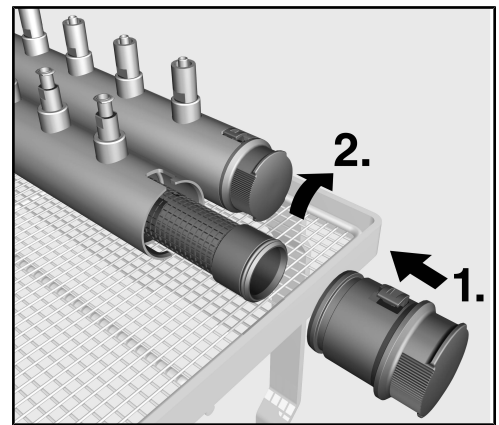Round tube

- Fit the locking cap over the end of the filter that is protruding out of the injector tube.
D-profile tube: The flat side must face upwards.
Round tube: The narrow groove must face upwards and the wide groove downwards.
- *D-profile tube:* Push the tubular filter fully into the injector tube by the locking cap.
Round tube: 1. Push the tubular filter fully into the injector tube by the locking cap.

en - Areas of application

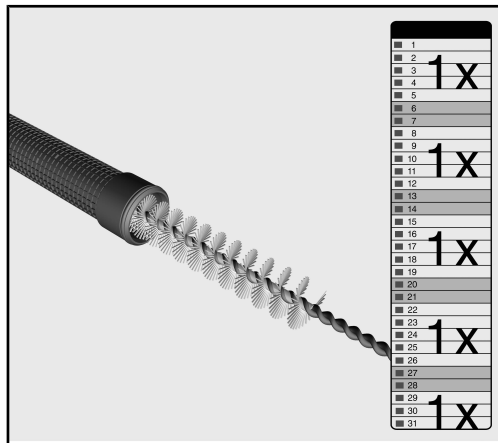
A light resistance will be felt, caused by the O rings. This is to ensure a good seal. If a resistance is not felt when pushing the filter in replace the O rings.

Only use genuine original Miele O rings as these are the only ones suitable for use in your machine. The tubular filter must not be used without these O-rings.

- *D-profile tube*: The locking cap must engage in position on both sides.


Round tube: 2. Turn the locking cap clockwise as far as it will go.

Cleaning the tubular filter


⚠ Clean the tubular filter **at least once a week** to ensure it maintains the correct filtration efficiency.

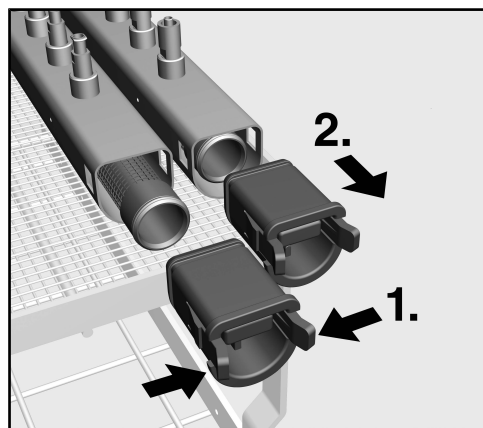Miele recommends that you clean the tubular filter after every 10 programme cycles.


D-profile tube

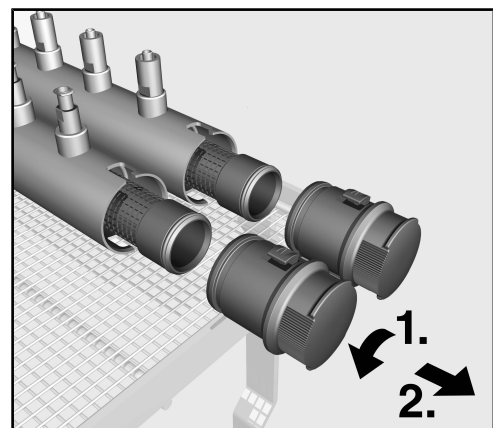
Round tube

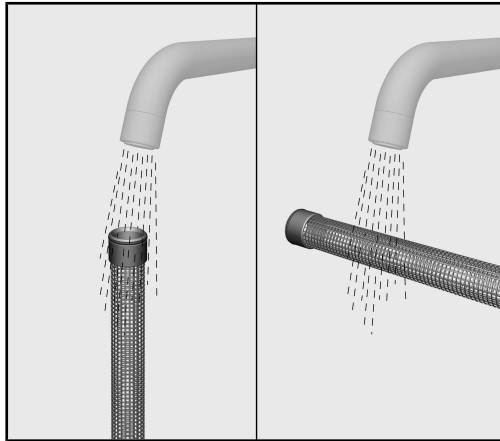
- Remove the tubular filter.

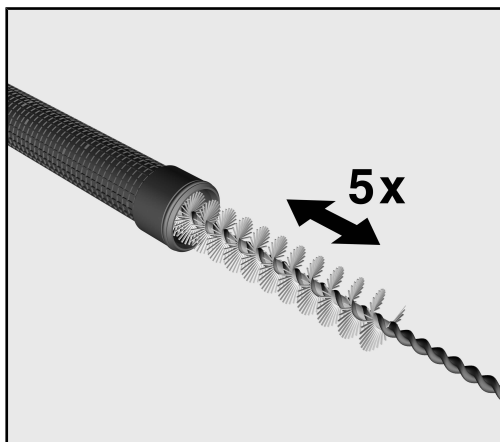- Rinse the tubular filter under running water to remove any coarse soiling.

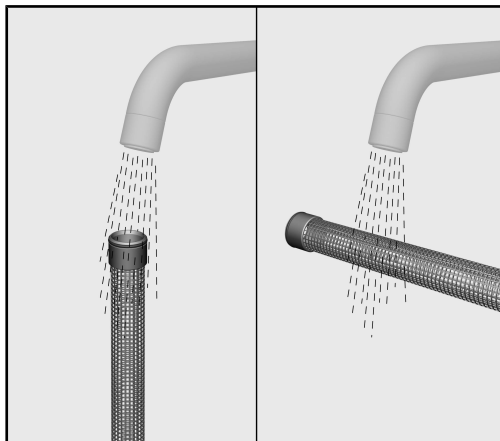
- Use the Miele A 804 brush to clean inside the filter. Move the brush in and out of the whole length of the tubular filter at least 5 times.


- Then rinse the inside of the tube thoroughly with running water.

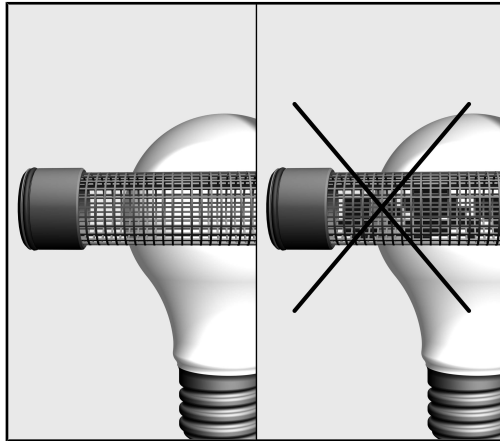
en - Areas of application

- Check that the tubular filter is clean by holding it up against a light source e.g. a window or a lamp. The filter lets light through. If light is not visible in places, or it only lets very little light through, then the tubular filter is not clean and should be cleaned again.
- Check the filter for damage e.g. small holes or tears. These are evident because these areas let more light through. Damaged tubular filter must not be used.
- Remove any soiling from the O-rings. Check the O-rings for visible signs of wear and tear.

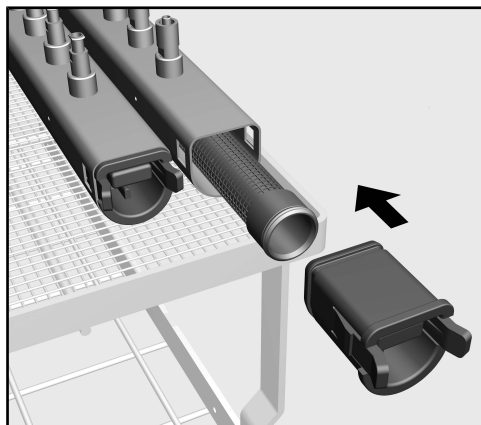D-profile tube

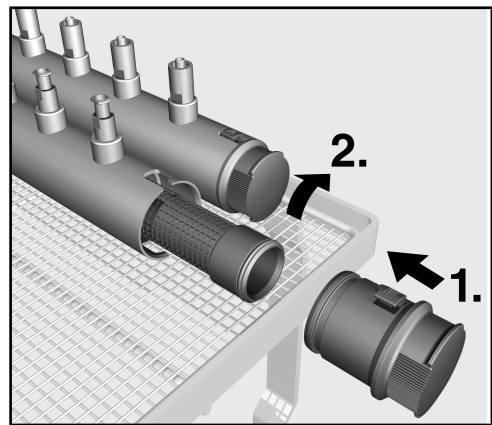Round tube

- Replace the tubular filter.
- Then rinse the brush thoroughly under running water.

If wished, the tubular filter and the brush can be cleaned in the machine after cleaning inside the filter with the brush. Place the filter and the brush loose in a basket or mobile unit to do so.

The brush will suffer wear and tear and should therefore be replaced every 2 to 3 years.

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