A 208

de	Gebrauchsanweisung Wagen
da	Brugsanvisning Vogn
en	Operating instructions: Mobile unit
es	Instrucciones de manejo Carros
fi	Käyttöohje Vaunu
fr	Mode d'emploi Chariot
hr	Upute za uporabu Injektorska kolica
it	Istruzioni d'uso Carrelli
nl	Gebruiksaanwijzing Wagen
no	Bruksanvisning vogn
pt	Instruções de utilização Carro
sv	Bruksanvisning Vagn

de	4
da	16
en	28
es	40
fi	52
fr	64
hr	76
it	88
nl	100
no	112
pt	124
sl	136
sv	148

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Notes about these instructions	28
Intended use	29
Queries and technical problems.....	29
Items supplied	30
Optional accessories.....	31
Warning and Safety instructions	32
Areas of application	33
Loading the mobile unit.....	33
Fitting the grille.....	33
Flexible level.....	34
Loading clearance	34
Before loading the machine and before starting a programme	35
Loading mesh trays	36
Hollow instruments.....	37
Processing hollow containers	37
Spray arm identification	38
Test point for measuring water pressure.....	38

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

■ Select an option using the arrow buttons and save your choice with *OK*.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

Re-usable medical devices can be reprocessed using this mobile unit in a Miele washer-disinfector. Follow the instructions for your washer-disinfector as well as the instructions of the medical device manufacturer on how to reprocess their items by machine.

The A 208 mobile unit can accommodate up to 8 DIN mesh trays or can be loaded with a variety of mesh trays and inserts for processing reusable medical products.

The mobile unit has 4 levels. The 3rd level from the bottom is removable, for instance for accommodating large items. The bottom level is equipped with a removable grille. This grille is needed when loading DIN mesh trays on the bottom level. It can be removed, if necessary, to accommodate large items.

The Vario TD Instr 8trays programme is designed for processing 8 mesh trays with medical products, e.g. surgical instruments.

The mobile unit can be used in the following washer-disinfectors:

- PG 8582
- PG 8582 CD
- PG 8592

Medical devices for reprocessing are referred to as the wash load if they are not more closely defined.

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your washer-disinfector or go to www.miele-professional.com.

- Mobile unit A 208 with 4 levels,
height 455 mm, width 530 mm, depth 542 mm
- with removable level
- with removable grille in the lower level.
- with spray arm magnets for spray arm monitoring

Items supplied

- 6 x E 362 blind stoppers

Optional accessories

The following and other accessories are available to order from Miele:

- A 810 Cover net, metal frame with replaceable silicone net A 811, 428 x 220 mm
- E 142, DIN mesh tray with a 5 mm grid and 2 hinged handles, height 45/55 mm, width, 255 mm, depth 480 mm
- E 146, mesh tray with lid and hinged handles, height 55 mm, width 150 mm, depth 255 mm
- E 447, Luer Lock Adapter ♀ for Luer Lock ♂
- E 448, silicone tube with Luer Lock adapter ♂ for Luer-Lock ♀, 300 mm long
- E 473/2 mesh tray with lid for small items, height 85 mm, width 60 mm, depth 60 mm
- E 476, 50 holders for Ø 4 to 8 mm micro instruments, for use in mesh trays with a 5 mm grid
- E 477, 20 holders for micro-instruments, for use in mesh trays with a 5 mm grid
- E 479, 50 holders for Ø 4 mm micro instruments, for use in mesh trays with a 5 mm grid
- E 492, insert with 9 holders for kidney dishes, spacing 49 mm, height 120 mm, width 256 mm, depth 474 mm

en - Warning and Safety instructions

To avoid the risk of accidents and damage to this mobile unit please read these instructions carefully before using it for the first time.
Keep these instructions in a safe place where they are accessible to users at all times.

 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ This mobile injector unit is only for use in the areas of application as defined in the Operating instructions. Components such as injector nozzles must only be replaced with Miele accessories or original parts. Any other application, modification or change is unauthorised and could be dangerous.
Miele cannot be held liable for damage caused by unauthorised use or incorrect operation.
- ▶ Inspect all mobile units, baskets, modules and inserts daily as described in the "Maintenance" section in the Operating instructions for the washer-disinfector.
- ▶ Only items which have been declared by their manufacturer as suitable for machine reprocessing may be processed. The manufacturer's specific reprocessing instructions must be observed. Disposable instruments must not be put into the machine for processing.
- ▶ The processing programme used should include thermal disinfection, and the chemical agents used must be suitable for the material quality of the wash load.
- ▶ The last rinse should be carried out with demineralised water wherever possible.
- ▶ Visual checks should be undertaken to check the cleaning result. Additional protein analysis checks should also be carried out e.g. weekly, on a random sampling basis.

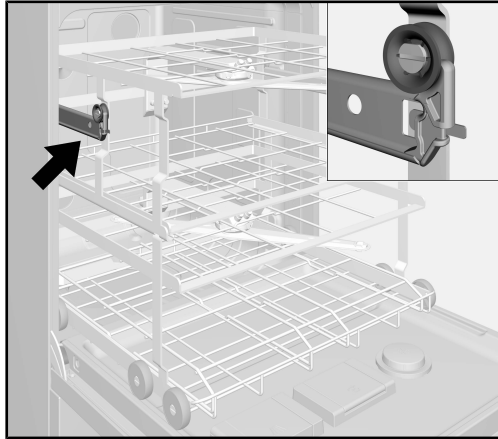
Miele cannot be held liable for damage caused by non-compliance with these Warning and Safety Instructions.

Loading the mobile unit

To ensure stability of the washer-disinfector, the maximum load of the mobile unit must not exceed 32 kg. This includes the removable level, rails and accessories, e.g. mesh trays. 1 DIN mesh trays weighs approx. 0.730 kg when empty.

The A 208 mobile unit must also be connected to the upper basket runners.

- Extend the upper basket runners.

- Place the A 208 mobile unit on the open machine door. As you do, hook the roller over the basket runners.

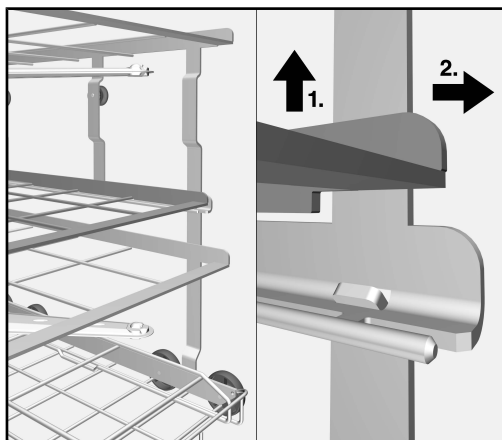
Fitting the grille

The removable grille must be fitted when loading the lower level with DIN mesh trays. It can be removed, if necessary, to accommodate large items.

- Place the grille in the lower level. The flat side must face forwards.

Flexible level

The 3rd level from the bottom is flexible and can be removed if required to accommodate large items.

- Lift level 3 upwards to release it.
- Then pull it forwards out of the mobile unit.
- To replace it place the removable level place on the runners in the mobile unit and push it in. As you do so make sure that the level connects with the stoppers.

Loading clearance

The 3rd level from the bottom and the grille are removable. The following vertical clearances are available:

Level	Vertical clearance
Level 4 (top)	75 mm
Level 3	130 mm
Level 2 with Level 3 removed	200 mm
Level 2	55 mm
Level 1 without the grille	100 mm
Level 1 (bottom)	60 mm

For validation purposes please make sure that you follow the loading instructions given on the template.

Before loading the machine and before starting a programme

- Are the spray arms able to rotate freely?
- Check spray arm jets for blockages.

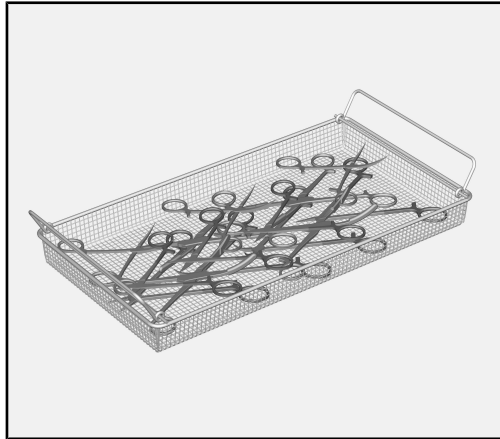
⚠ Make sure that all screw connectors are fitted with injector nozzles, adapters, irrigation sleeves or blind stoppers to ensure that all rinsing fittings in use are supplied with sufficient standardised pressure.

Damaged fittings such as injector nozzles, adapters and irrigation sleeves must not be used.

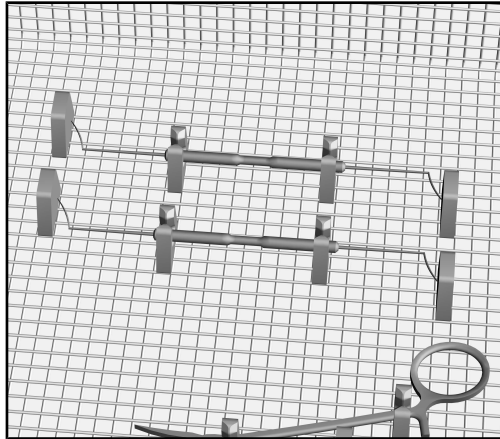
Rinsing positions not in use do not have to be fitted with blind stoppers.

- Is the mobile unit sitting correctly on the runners for the upper basket?
- Is the removable level securely positioned?
- Check that the mobile unit is correctly docked to the water path in the chamber.

Loading mesh trays

- Hinged instruments must be opened to minimise the risk of surfaces not being accessed during reprocessing.

- To prevent the risk of light instruments and other small items being moved about during reprocessing, place them in closable inserts such as the E 473/2 or secure them in silicone holders such as the E 476, E 477 and E 479. The E 473/2 insert is supplied with its own operating instructions.

Hollow instruments

Up to 6 hollow instruments can be reprocessed in the A 208 mobile unit using a Luer lock connector. The mobile unit is equipped for this purpose with appropriate connectors on the water supply pipes - 2 each on levels 2, 3 and 4.

- Replace the blind stoppers on the water pipe with silicone hoses with Luer lock adapters, e.g. the E 448.
- Connect the instruments to the silicone hoses and place the instruments in a mesh tray. Secure lightweight instruments with silicone holders e.g. E 476, E 477 and E 479.

Processing hollow containers

- Hollow containers such as beakers and kidney dishes should be loaded with the open side facing downwards. Lightweight containers should be secured with a cover net e.g. the A 810
- Special inserts should be used if necessary for processing hollow containers. These inserts can be purchased from Miele.

The containers must not hinder the processing of other instruments and must be secure so that they cannot be flipped over by the force of water during processing.

If hollow items are placed with the open side facing upwards, suds solution can collect in them. This can spill and impair cleaning results.

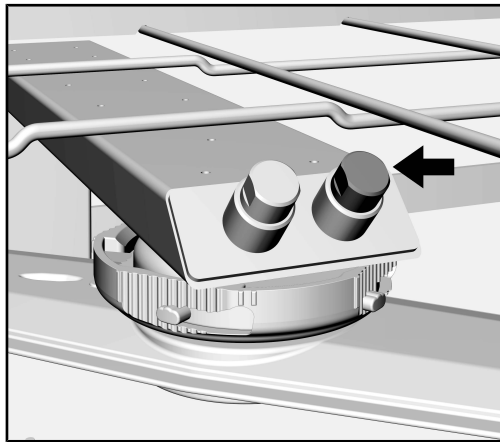
Spray arm identification

Spray arms in the A 208 mobile unit differ in the way the jets are arranged and each has a unique ID number. The ID numbers are stated on a rail on the left hand side of the mobile unit at the same height as the spray arm.

If the spray arms are dismantled, e.g. for cleaning purposes, you must ensure when replacing them that the ID number on the spray arm matches the ID number on the rail in the mobile unit. If it does not match, cleaning performance in the mobile unit will be impaired.

Instructions on how to clean the spray arms is given in the operating instruction manual for your washer-disinfector.

Test point for measuring water pressure

The right hand connection point on the upper water supply pipe serves as the test point for measuring water pressure. This connection point can be used for testing pressure within the framework of performance and validation testing in accordance with EN ISO 15883.

- To measure water pressure replace the existing blind stopper with a Luer Lock adapter, e.g. E 447.

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