APFD 203

de	Gebrauchsanweisung Einsatz
cs	Návod k obsluze nástavce
da	Brugsanvisning Indsats
el	Οδηγίες χρήσης Ένθετο
en	Operating instructions Insert
es	Instrucciones de manejo Complemento
et	Kasutusjuhend Element
fi	Käyttöohje Teline
fr	Mode d'emploi Complément
hr	Upute za uporabu Umetak
hu	Használati útmutató Betét
it	Istruzioni d'uso Inserto
lt	Naudojimo instrukcija Naudojimas
lv	Lietošanas instrukcija Ieliktnis
nl	Gebruiksaanwijzing inzet
no	Bruksanvisning innsats
pl	Instrukcja użytkowania Wkład
pt	Instruções de utilização Complemento
ru	Инструкция по эксплуатации вставки
sk	Návod na použitie Nadstavec
sv	Bruksanvisning Insats
tr	Kullanım kılavuzu Ek modül
uk	Посібник з експлуатації Вставка

de	4
cs	10
da	16
el	22
en	28
es	34
et	40
fi	46
fr	52
hr	58
hu	64
it	70
lt	76
lv	82
nl	88
no	94
pl	100
pt	106
ru	112
sk	118
sv	124
tr	130
uk	136

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Notes about these instructions	28
Symbols and conventions used in this document.....	28
Definition of terms	28
Appropriate use	29
Queries and technical problems	29
Items supplied	30
Load carriers.....	30
Disposal of the packing material	30
Safety instructions and warnings	31
Areas of application	32
Preparing the load	32
Preparing the insert.....	32
Pre-cleaning the carrier frame	32
Inserting the carrier frame.....	32
After reprocessing	32

en - Notes about these instructions

Symbols and conventions used in this document

Warnings

 Warnings contain information which is important for safety. This alerts you to the potential danger of injury to people or damage to property.
Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Notes provide information of particular importance that must be observed.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

■ Select an option using the arrow buttons and save your choice with *OK*.

Example:

■ Select an option.

Display

Display text can be identified from the special font.

Example:

Save.

Definition of terms

Cleaning machine

In these operating instructions, the dishwasher or the washer-disinfector is referred to as “the machine”.

Load items

The term “load items” is used wherever the items to be processed are not defined in any further detail.

Wash water

The term “wash water” refers to water or to a mixture of water and process chemicals.

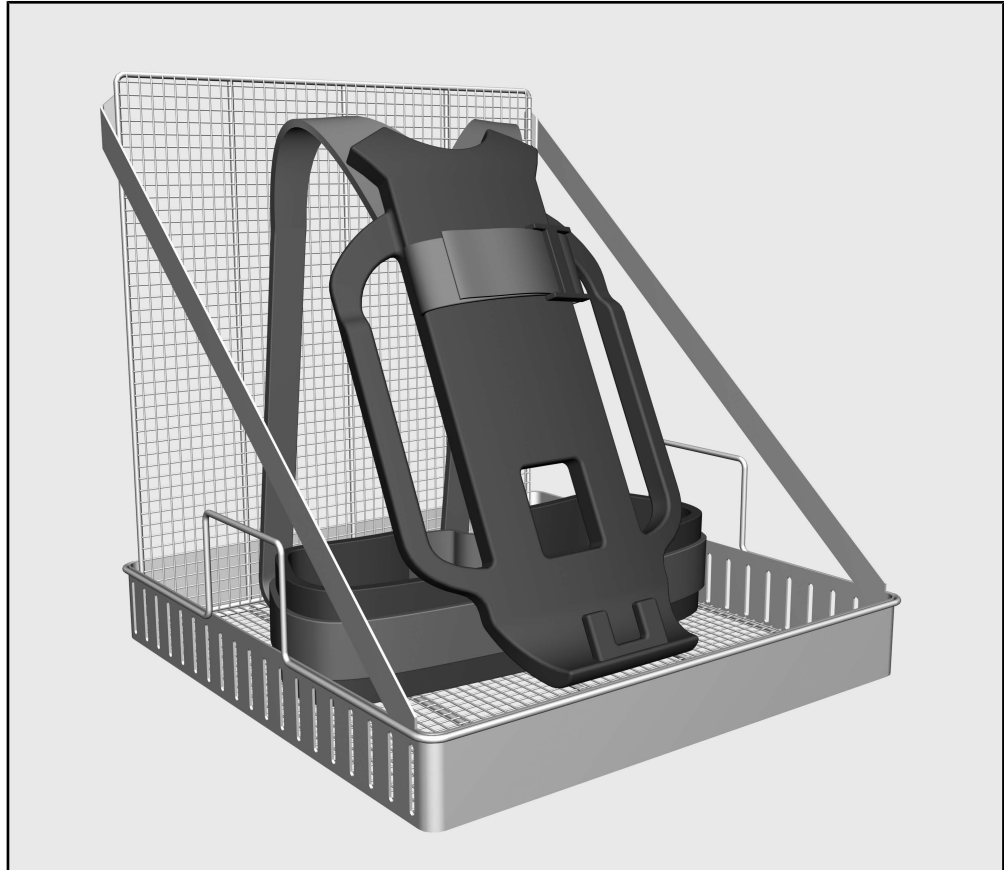
The APFD 203 insert can be used to reprocess machine-reprocessable carrier frames for compressed air units in a Miele dishwasher or a machine. Follow the operating instructions for your dishwasher or the machine as well as the instructions issued by the manufacturer of the carrier frames for compressed air units.

The APFD 203 insert can be used together with the A 151 lower basket.

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your dishwasher or the washer-disinfector or go to www.miele-professional.com.

Load carriers

- APFD 203 insert, height 481 mm, width 470 mm, depth 470 mm

Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in land-fill sites.

Read the operating instructions carefully before using this insert.
Keep these operating instructions in a safe place.

 Please also read the operating instructions for your dishwasher or the machine and pay particular attention to the Warning and Safety instructions.

- ▶ The inserts are only approved for use in the applications stated in the operating instructions. Applications that go beyond those listed and use of the inserts with additional components from the Miele range of components are described in the operating instructions of the components or are to be agreed with Miele. Conversions, modifications and any other use are not permitted.
- ▶ Prior to being used for the first time, new inserts must undergo thorough cleaning in the dishwasher or the machine without load items.
- ▶ Inspect all mobile units, baskets, modules and inserts as described in the “Maintenance” section of the operating instructions for your machine.
- ▶ Only items which have been declared by their manufacturer as suitable for machine reprocessing may be processed. The manufacturer's specific reprocessing instructions must be observed.
- ▶ Follow the warnings and instructions provided by the breathing apparatus manufacturer.
- ▶ Any seals on the load items should be protected against prolonged contact with wash water. Therefore, always close unused connections with protective caps or blind covers.
- ▶ Disposable items must not be reprocessed.

en - Areas of application

Preparing the insert

Preparing the load

- Remove the upper basket from the machine.
- Pull the lower basket out of the wash cabinet onto the machine door.
- Place the insert into the lower basket.

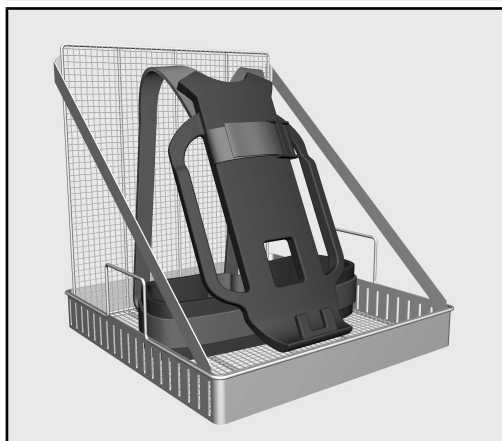
The rear panel of the insert is facing the rear panel of the wash cabinet.

Pre-cleaning the carrier frame

- Check whether the carrier frame requires manual pre-cleaning, e.g. to remove soot.
- If required, carry this pre-cleaning out in line with the manufacturer's instructions.

Inserting the carrier frame

⚠ Carry straps that point upwards can get caught in the machine spray arm and affect the cleaning result or can tear off the spray arm or damage the machine when the insert is removed.

- Place the carrier frame in the front part of the insert with the carry straps pointing downwards.
- Lean the carrier frame diagonally against the support grid in the rear part of the insert.

The carry straps must hang loose to ensure the wash water cleans and rinses them completely.

- Push the lower basket with the insert into the wash cabinet.

After reprocessing

It is essential that you pay attention to the “Application technology” chapter in the operating instructions for the dishwasher or the machine and the instructions specified by the manufacturer of the carrier frames for compressed air units.

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