APLW 630

de	Gebrauchsanweisung Einsatz
cs	Návod k obsluze nástavce
da	Brugsanvisning Indsats
en	Operating instructions Insert
es	Instrucciones de manejo Complemento
fi	Käyttöohje Teline
fr	Mode d'emploi Complément
hu	Használati útmutató Betét
it	Istruzioni d'uso Inserto
nl	Gebruiksaanwijzing inzet
no	Bruksanvisning innsats
pl	Instrukcja użytkowania Wkład
pt	Instruções de utilização Complemento
sv	Bruksanvisning Insats

de	4
cs	18
da	32
en	46
es	60
fi	74
fr	88
hu	102
it	116
nl	130
no	144
pl	158
pt	172
sv	187

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Notes about these instructions	46
Symbols and conventions used in this document.....	46
Definition of terms	46
Appropriate use	47
Queries and technical problems	47
Items supplied	48
Load carriers.....	48
Disposal of the packing material	48
Optional accessories.....	48
Warnings and safety notes	49
Areas of application	50
Colour coding.....	50
Preparing the load	51
Preparing the insert.....	51
Pre-cleaning air regulators.....	51
Dismantling air regulators.....	51
Inserting air regulators	52
Using the adapter	52
Connecting the hose	52
Securing the hose.....	53
Completely loading the insert.....	53
Filling the pressure chamber.....	54
Placing the insert in the load carrier	55
After reprocessing	55
Maintenance	57
Routine check	57
Condensation water in the pressure chamber.....	58
Tools required	58
Emptying the pressure chamber	58

Symbols and conventions used in this document

Warnings

 Warnings contain information which is important for safety. This alerts you to the potential danger of injury to people or damage to property.
Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Notes provide information of particular importance that must be observed.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

■ Select an option using the arrow buttons and save your choice with *OK*.

Example:

■ Select an option.

Display

Display text can be identified from the special font.

Example:

Save.

Definition of terms

Machine

In these operating instructions, the laboratory washer is referred to as “the machine”.

Load items

The term “load items” is used wherever the items to be processed are not defined in any further detail.

Wash water

The term “wash water” refers to water or to a mixture of water and process chemicals.

This APLW 630 insert can be used to reprocess machine-reprocessable air regulators in a Miele washer-disinfector. Follow the operating instructions for your washer-disinfector as well as the instructions of the air regulators manufacturer on how to reprocess their items by machine.


The APLW 630 insert can be used together with the following load carriers:

Cleaning machine	Load carrier
PLW 7111 Safety	APLW 107 upper basket
	APLW 152 lower basket
PLW 8615 Safety	E 975/2 mobile unit
PLW 8616 Safety	
PLW 8617 Safety	

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your dishwasher or the washer-disinfector or go to www.miele-professional.com.

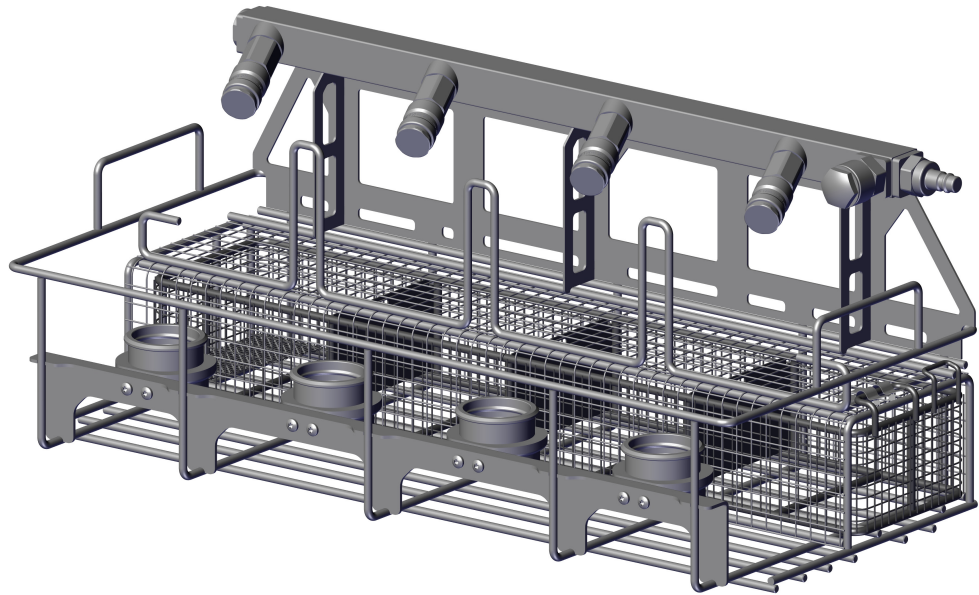Load carriers

Items supplied

Disposal of the packing material

Optional accessories

- APLW 630 insert, height 221 mm, width 250 mm, depth 541 mm

- Small parts basket, four compartments

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in land-fill sites.

The following and other accessories are available to order from Miele:

- APLW 873, hangers in 6 colours
- APLW 892, adapter for holding air regulators with M45 x 3 threaded connection according to EN 148-3
- APLW 893, adapter for holding air regulators with special plug connectors, e.g. from MSA[®]

Read the operating instructions carefully before using this insert.
Keep these operating instructions in a safe place.

 Please also read the operating instructions for your dishwasher or the washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ The inserts are only approved for use in the applications stated in the operating instructions. Applications that go beyond those listed and use of the inserts with additional components from the Miele range of components are described in the operating instructions of the components or are to be agreed with Miele. Conversions, modifications and any other use are not permitted.
- ▶ Prior to being used for the first time, new inserts must undergo thorough cleaning in the dishwasher or the washer-disinfector without load items.
- ▶ Inspect all mobile units, baskets, modules and inserts as described in the “Maintenance” section of the operating instructions for your machine.
- ▶ Only items which have been declared by their manufacturer as suitable for machine reprocessing may be processed. The manufacturer's specific reprocessing instructions must be observed.
- ▶ Follow the warnings and instructions provided by the breathing apparatus manufacturer.
- ▶ Any seals on the load items should be protected against prolonged contact with wash water. Therefore, always close unused connections with protective caps or blind covers.
- ▶ The last rinse, as a minimum, must be carried out with DI water.
- ▶ Disposable items must not be reprocessed.

en - Areas of application

The APLW 630 insert for reprocessing air regulators must be supplied with pressure for reprocessing. The pressure ensures that wash water cannot enter the air regulators during reprocessing, assuming the correct adapters are used.

Risk of infection

The compressed air must correspond to the quality of breathing air as a minimum.

For this, observe the national and regional regulations concerning compressed air.

Risk of injury caused by compressed air

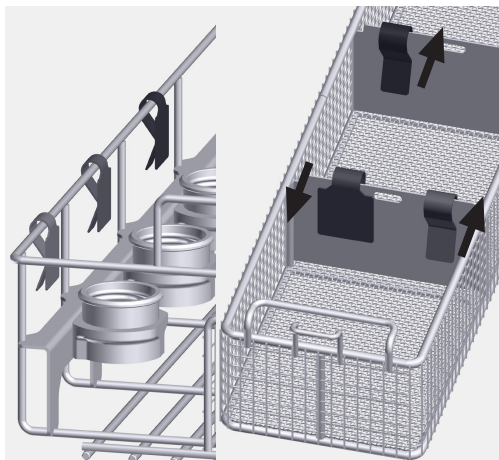
A flexible compressed air connection with defined quick-release couplings must be provided on site.

Use a compressed air connection with one of the following quick-release couplings:

- CEJN series 341
- Parker Rectus series 25S

Colour coding

The small parts of a dismantled air regulator are reprocessed in a compartment of the small parts basket. Colour-code the compartments and the holders to ensure that the small parts can be assigned to the right air regulators after reprocessing.

Legend

For each colour, attach one tag next to the holders and one tag to the small parts basket compartments.

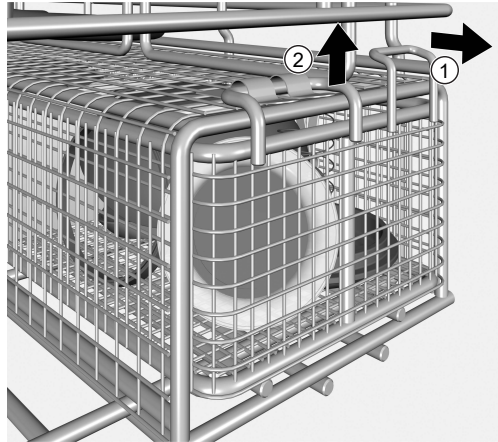
If you are using clip labels, they must be attached to the dividers inside the small parts compartments. We recommend that the colour-coded tag is attached to the side facing the surface (see arrows in image above).

When using other systems for identification, ensure that these are suitably chemical-resistant and temperature-resistant.

Preparing the load

Preparing the insert

- Place the insert on the worktop and ensure that it is securely positioned.

Legend

- Pull the small parts basket out of the insert using the handle ①.

The locking mechanism ② provides haptic feedback. This ensures that the small parts basket is fully inserted. Take care not to pull on the locking mechanism, as it could break off.

- Place the small parts basket next to the insert.

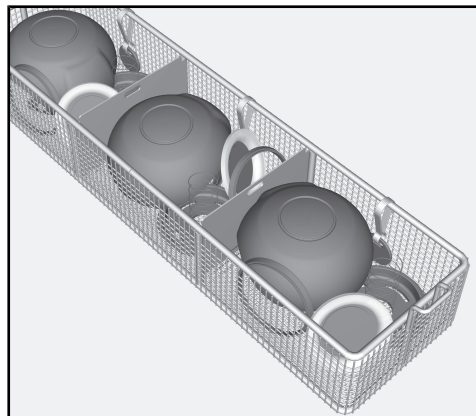
The insert holders and the adjacent compartments of the small parts basket must have the same colour.

Pre-cleaning air regulators

- Check whether the air regulator requires manual pre-cleaning, e.g. to remove soot.
- If required, carry this pre-cleaning out in line with the manufacturer's instructions.

Dismantling air regulators

- Dismantle the air regulator in line with the manufacturer's instructions.

Legend

- Place the small parts in a compartment in a small parts basket.
- Do not position small parts in such a way that they will collect water. For example, insert rubber covers with the curved hollow side facing down.

en - Areas of application

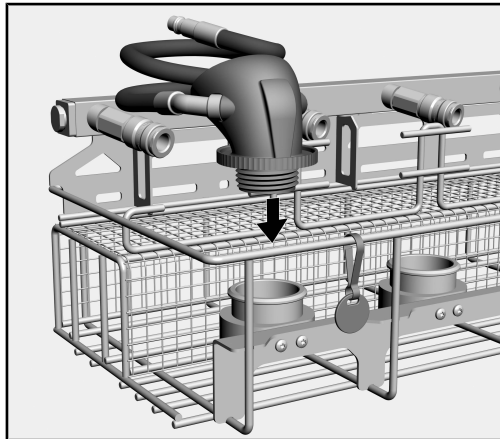
Note the colour-coding for the small parts basket compartment to place the air regulator in the holder with the same colour.

Inserting air regulators

The holders are suitable for air regulators with an ESA plug connector to DIN 58600 or an RD 40 round threaded connection to DIN EN 148-1.

Various adapters are required for air regulators with other connections, see section “Items supplied – Optional accessories”.

- Select the holder with the same colour marking as on the compartment of the small parts basket with the associated small parts.


Legend

- Screw or insert the coupling of the air regulator into the holder with the opening facing downwards.

Using the adapter

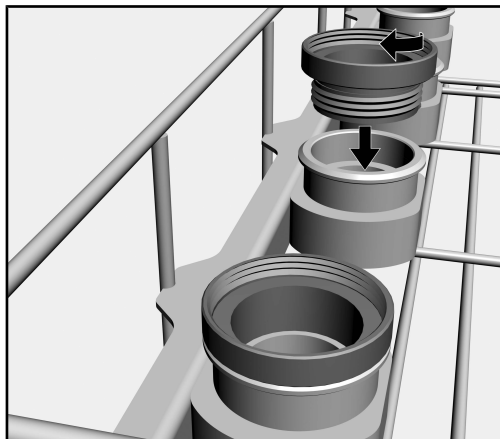- Select the adapter suitable for connection to the air regulator.


Legend

- Screw the required adapter firmly into the insert holder.

Connecting the hose

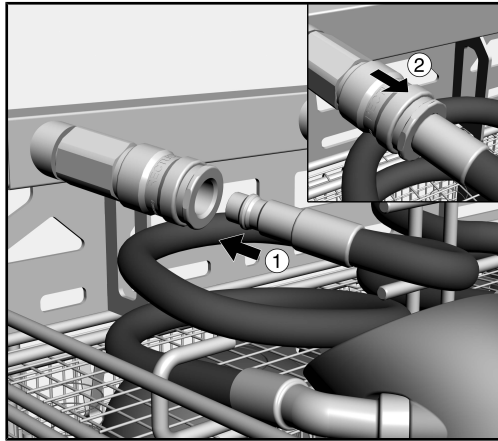To supply the air regulator with compressed air, its hose must be connected to the insert.

Legend

- Insert the hose adapter into the quick-release coupling on the insert ① until it engages with an audible click.

The collar on the coupling is pushed diagonally upwards in the process.

As soon as the collar has snapped back ②, the hose is pressure-tight.

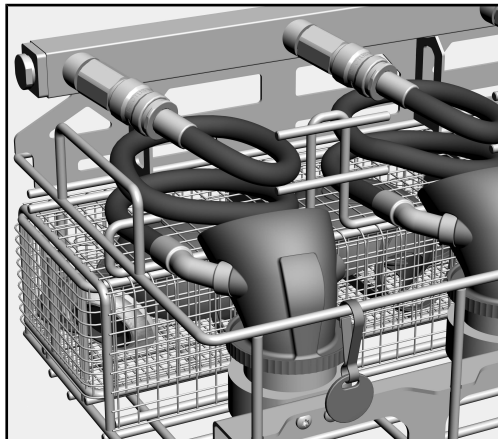
Danger to health

Damaged quick-release couplings can force wash water into the air regulator.

Have the quick-release coupling replaced with an original spare part if it is no longer possible to hear the click clearly or the collar does not snap back fully.

Securing the hose

Legend

- Lay the hose around the hose guide for the insert and underneath the quick-release couplings.

The hose is held in position by laying it underneath the quick-release couplings.

If hoses are not secured, they may block the spray arms during reprocessing.

This will affect the reprocessing results.

Secure all hoses using the locking pins on the hose guide.

Completely loading the insert

- Equip the remaining positions of the insert in the same way.

en - Areas of application

- Sliding the small parts basket into the insert

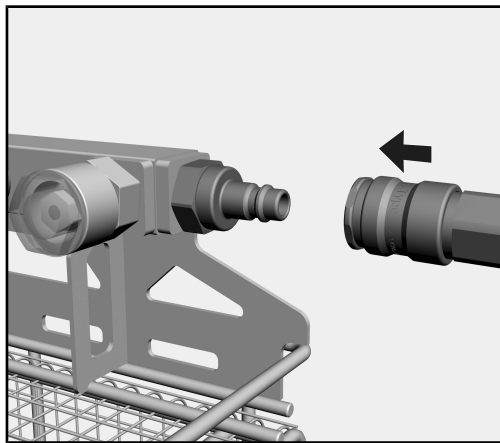
The retainer of the small parts basket engages in the locking mechanism of the insert.

Filling the pressure chamber

⚠ Risk of injury caused by compressed air.
The jet of compressed air could be dangerous for the user.
Users must be trained on how to use the compressed air system on the customer's premises.


The pressure chamber is designed for pressure ranging up to 600 kPa.

⚠ Damage to the air regulators caused by pressure.
Do not provide more than 600 kPa of pressure to the pressure chamber. Higher pressure can damage the air regulators.
Observe the air regulator manufacturer's instructions.

Legend

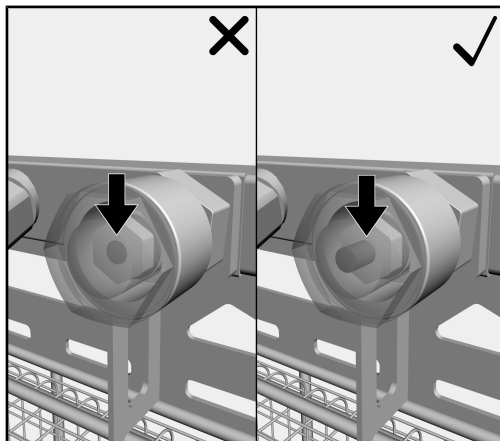- Connect the compressed air hose on the customer's premises to the quick-release coupling on the pressure chamber of the insert.

Legend

- Add pressure to the pressure chamber until the red compressed air indicator pokes out ②.
- X If the red compressed air indicator does not poke out, the pressure is not sufficient.
- ✓ If the red compressed air indicator pokes out, the pressure is sufficient.

The compressed air indicator can fog up from the inside. This is not an indication that the pressure chamber is leaking.
Carry out regular routine checks in order to check that the pressure chamber is airtight.

⚠ Danger to health

If the connection between the quick-release couplings on the pressure chamber and the hoses for the air regulators is not absolutely tight, the pressure will drop over time.
If the pressure drops too much, wash water and particles may get into the air regulators and impair their function.
Therefore, supply the pressure chamber with pressure as close to the programme start as possible.

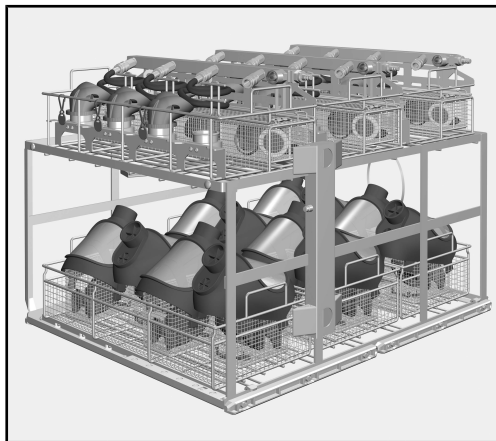
Placing the insert in the load carrier

- Lift the insert at the side handles.
- Place the insert in the load carrier of the machine.

Loading the
APLW 152

Combinations of ALPW 630 and APLW 631 are not intended to be used in the APLW 152 lower basket.

Example load for
E 975/2

Legend

E 975/2 with 3 x APLW 630 for breathing masks and 3 x APLW 631 for air regulators

After reprocessing

⚠ Danger to health

If the pressure supplied to air regulators during reprocessing is too low or if hoses have disconnected from the quick-release couplings, wash water and particles can get inside the air regulators. The wash water may contain cleaning agents and disinfectants that are harmful to health.

In both cases, the air regulators must be rinsed and dried internally in line with the manufacturer's instructions before being used again.

- Take the insert out of the load carrier of the cleaning machine.
- Check whether the hoses have disconnected from the quick-release couplings and whether the compressed air indicator is still showing sufficient pressure in the pressure chamber.

en - Areas of application

If the compressed air indicator is showing insufficient pressure or if the hoses have disconnected from the quick-release couplings, the air regulators must be rinsed and dried internally in line with the manufacturer's instructions before being used again.

 Risk of injury.

When the quick-release couplings are disconnected, the escaping compressed air can cause the hose to spin around uncontrollably. The adapter at the end of the hose could injure you.

Hold the hose adapter firmly so that it is not spun around by the escaping compressed air.

- Disconnect the hoses by pulling back the collars on the quick-release couplings.
- Remove the air regulators and empty any remaining water into the wash cabinet or an on-site slops basin before removing items.
- Follow the air regulator manufacturer's instructions regarding drying, assembly and replacing wear parts.

 Risk of infection

Before using the breathing apparatus again following reprocessing, carry out a function check in line with the manufacturer's instructions.

Routine check

 Danger to health

Damaged seals in the quick-release couplings can force wash water and particles into the air regulator. This prevents the air regulator from working properly.

Have the quick-release coupling with damaged seals replaced with an original spare part.

The seals and airtightness of the pressure chamber must be checked regularly to make sure the insert is functioning correctly.

Daily

- During the daily routine, visually check the seals in the quick-release couplings to ensure they are intact.

Weekly

- During the weekly routine, check that the pressure chamber is airtight.

To do this:

- Fit the quick-release couplings on the insert with blind covers.
- Apply compressed air to the pressure chamber of the insert.

The compressed air indicator must remain switched after an hour.

Condensation water in the pressure chamber

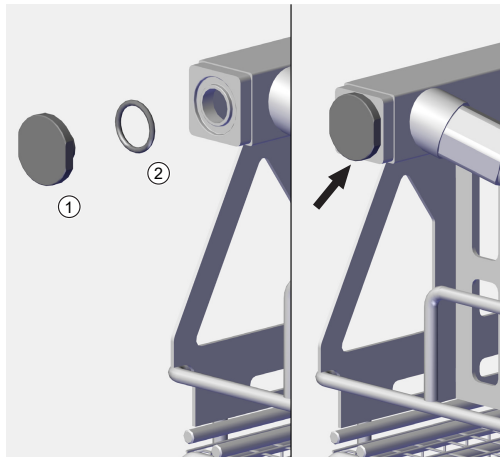
Humidity in the compressed air can cause condensation water to form in the pressure chamber of the APLW 630 insert. Compressed air that is of breathing air quality is usually sufficiently dehumidified to ensure that no significant condensation water forms in the pressure chamber. However, should condensation water have formed in the pressure chamber nevertheless, it must be emptied as follows. The drain screw is located on the side of the pressure chamber opposite the compressed air connection.

Tools required

- Open spanner, 17 mm (WAF 17)

Emptying the pressure chamber

- Ensure that the pressure chamber of the insert is not under pressure, see compressed air indicator.

Legend

- Unscrew the drain screw ①.
- Empty the pressure chamber.
- Allow the pressure chamber to dry.
- Refit the O-ring ②.
- Screw the drain screw back in.
- Tighten the drain screw as far as it will go.

A pressure test must be performed before the insert is next used. To do this, apply compressed air to the pressure chamber of the insert and check the compressed air indicator after one hour. If the compressed air indicator is no longer switched, check that the drain screw and the O-ring are positioned correctly.

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