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
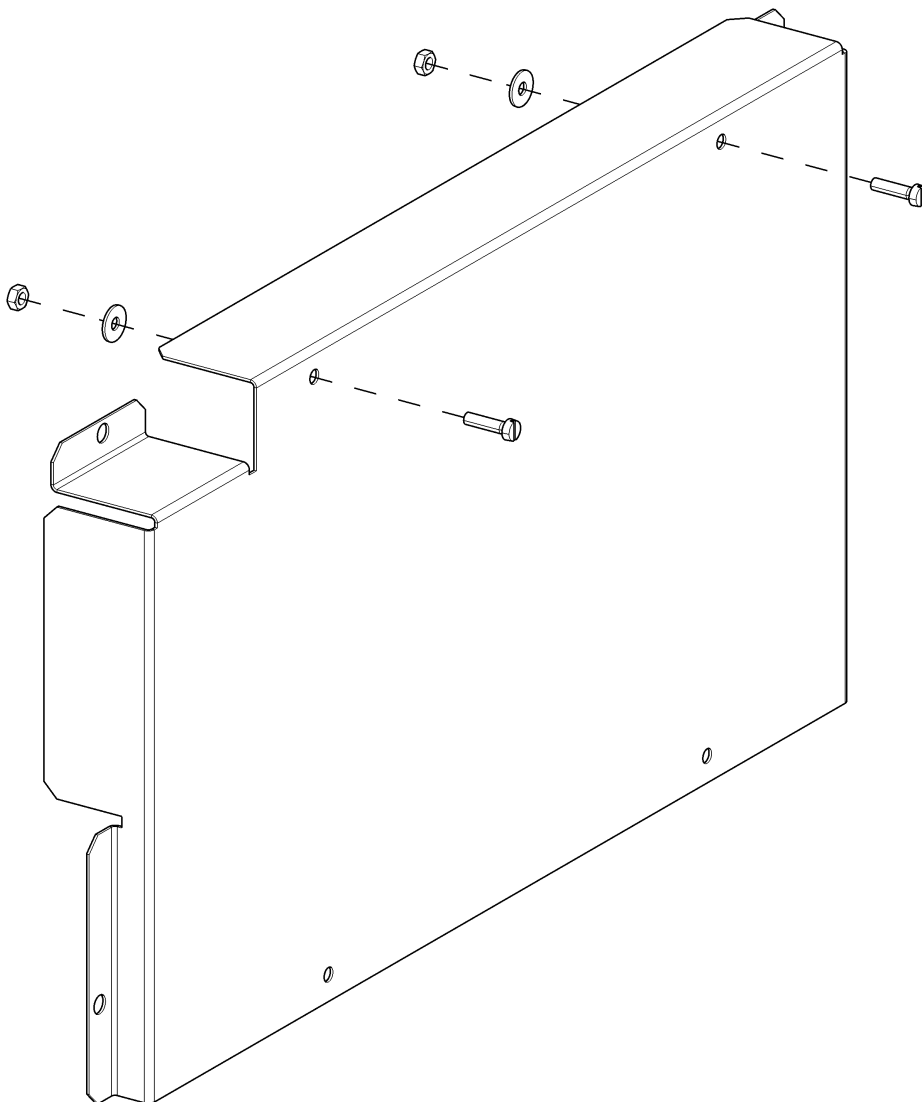
M.-Nr. 11819260

1 von 16


de	Montageanweisung Connector-Box APDR001 an PDR510, PDR910	en	Installation instructions for APDR001 Connector Box on PDR510, PDR910	cs	Montážní návod připojovacího boxu APDR001 na PDR510, PDR910
da	Monteringsanvisning connector-box APDR001 på PDR510, PDR910	el	Οδηγίες τοποθέτησης του Connector-Box APDR001 στο PDR510, PDR910	es	Instrucciones de montaje Connector-Box APDR 001 en PDR 510, PDR 910
fr	Instructions de montage Connector-Box APDR001 sur PDR510, PDR910	it	Istruzioni di montaggio Box Connector APDR001 su PDR510, PDR910	nl	Montage-instructie connectorbox APDR001 op PDR510, PDR910
pl	Instrukcja montażu skrzynki przyłączeniowej APDR001 do PDR510, PDR910	pt	Instruções de montagem para Connector Box APDR001 em PDR510, PDR910	ru	Инструкция по монтажу коммуникационной коробки APDR001 к PDR510, PDR910
sv	Monteringsanvisning Connector Box APDR001 för PDR510, PDR910				

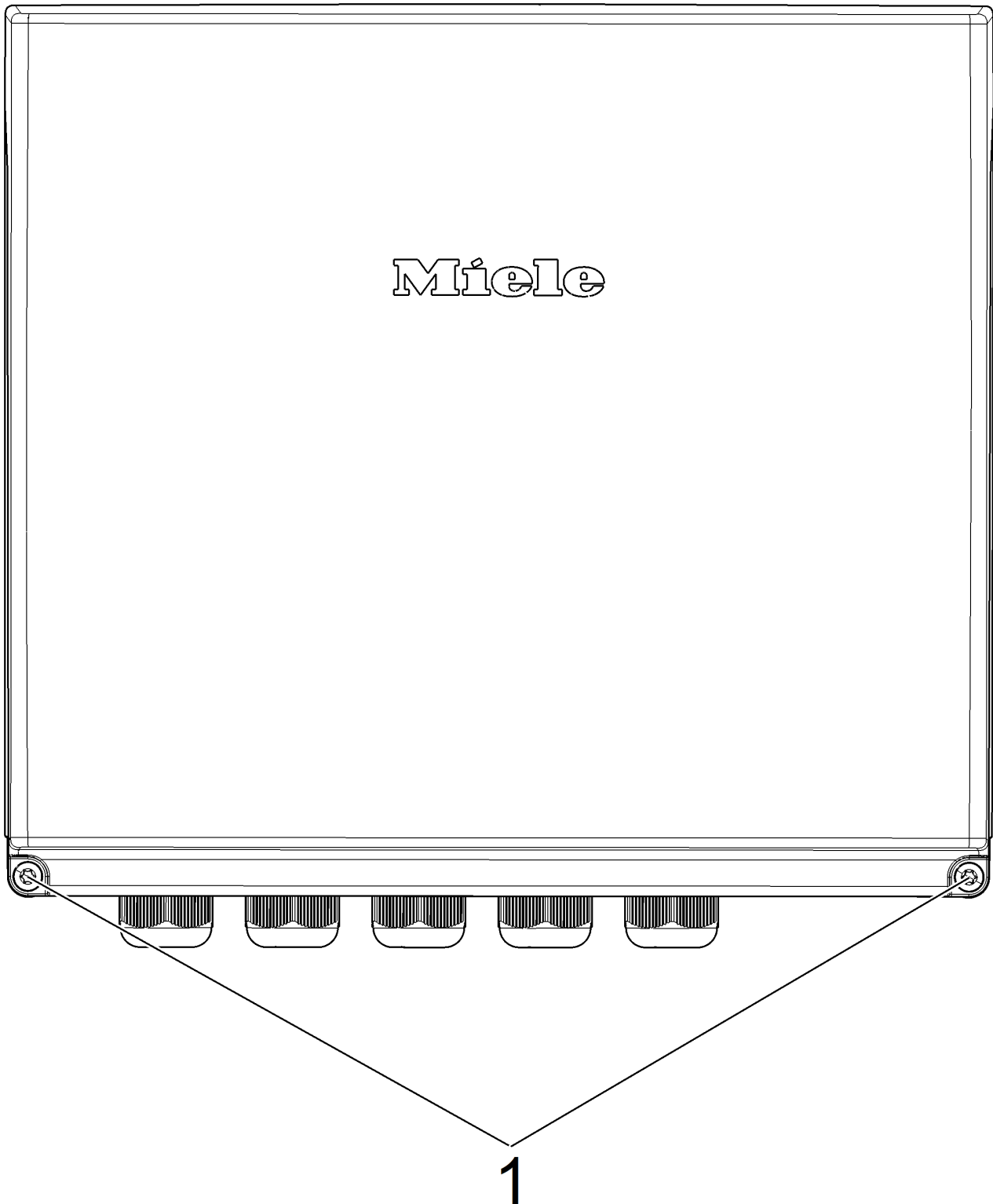
1

2

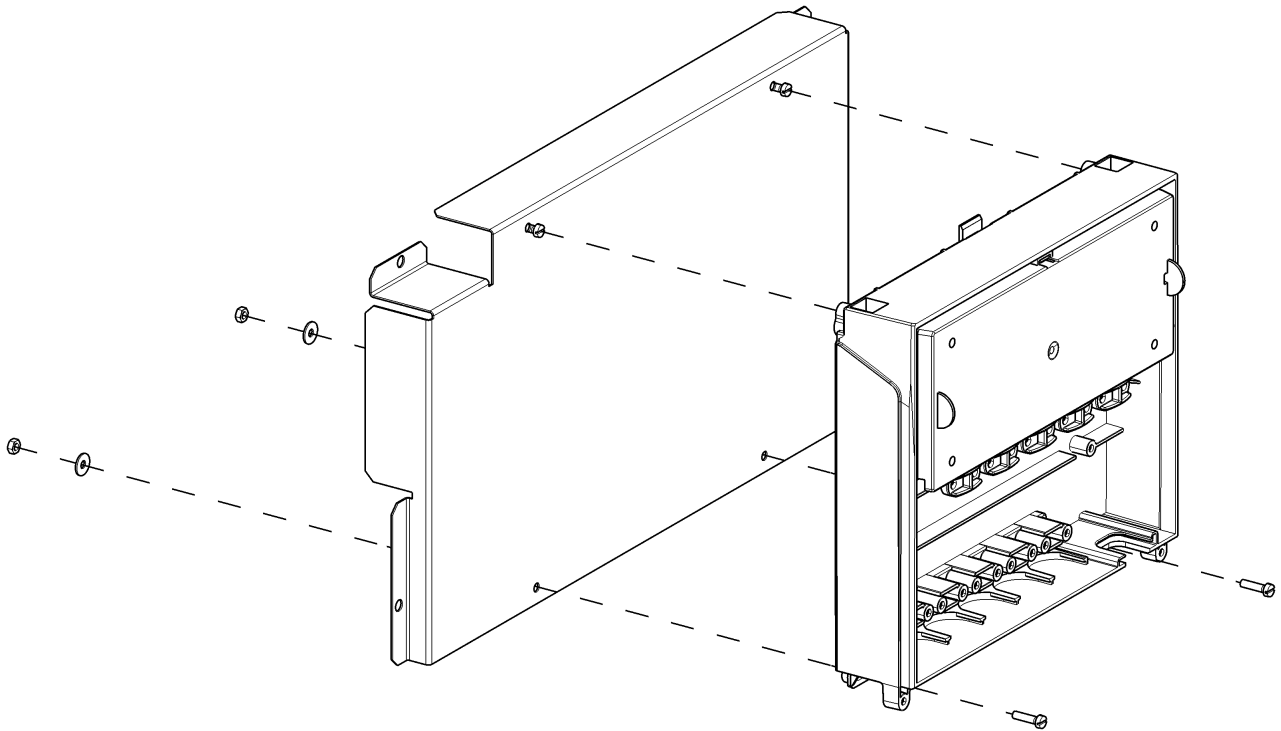3

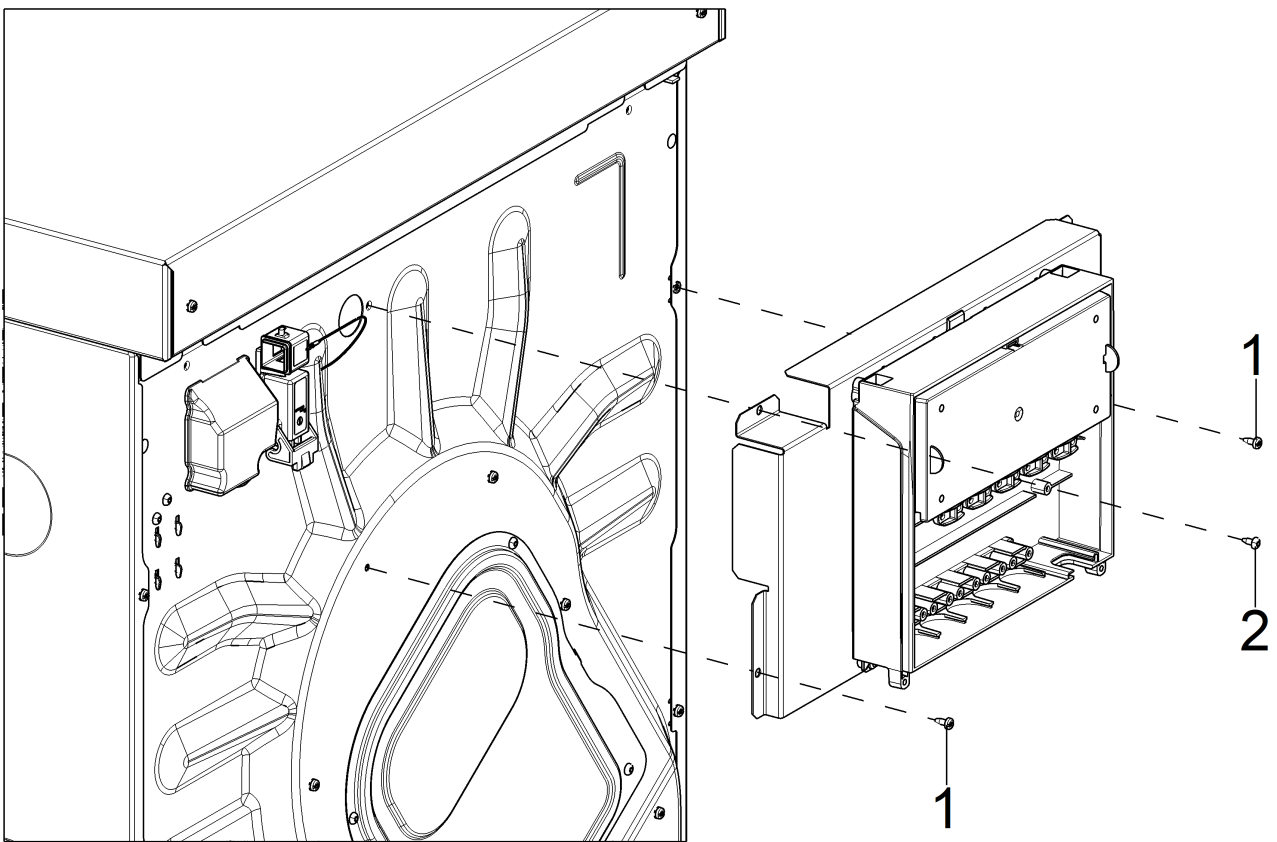4

de**Montageanweisung Connector-Box APDR001 an PDR510, PDR910****Enthaltene Teile**

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de

en

Installation instructions for APDR001 Connector Box on PDR510, PDR910

Included parts

No.	Mat. no.	Designation
1		Connector Box
1		Holding plate
4		Cheese-head screw M4x16
4		Hex nut M4
4		Washer B4.3
5		Raised-head screw M4 x 12.5

Danger due to incorrect service and repair work

WARNING

Danger due to incorrect service and repair work.

There is a risk of serious damage, injury and even death if service and repair work is carried out incorrectly.

- ✂ Service and repair work should only be carried out by a suitably qualified electrician with specialist training, knowledge and experience.
- ✂ All appropriate safety requirements must be taken into account.
- ✂ Wear personal protective equipment.
- ✂ First read the Miele service documentation, then commence work.
- ✂ Ensure that no people or animals can access the work area while service work is being carried out.

The “Miele Service Documentation (MSD)” service application is required when carrying out maintenance work on the appliance

The safety instructions and warnings in the Miele Service Documentation (MSD) for the corresponding model must be observed.

Work on the appliance must be carried out in accordance with the work instructions.

Fitting the Connector Box with holding plate to the tumble dryer

- ✂ Screw 2 cheese-head screws loosely to the holding plate with 1 washer and 1 nut attached to each of them, see Fig. 1.
- ✂ Remove the screws (1) from the lid of the Connector Box, see Fig. 2.
- ✂ Fold the lid of the Connector Box up and back.
- ✂ Position the top of the Connector Box on the screws on the holding plate, see Fig. 3.
- ✂ Screw the bottom of the Connector Box to the holding plate using 2 cheese-head screws with 1 washer and 1 nut attached to each of them, see Fig. 3.
- ✂ Screw the nuts tight at the top.
- ✂ Remove the 2 screws (1) on the back of the tumble dryer, see Fig. 4. Use the 2 screws (1) and 1 cheese-head screw M4 x 12.5 (2) to screw the Connector Box to the tumble dryer with the holding plate, see Fig. 4.
- ✂ Connect the Connector Box as described in the operating instructions.
- ✂ Fit the lid to the Connector Box.

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