A 602

de	Gebrauchsanweisung Injektormodul
da	Brugsanvisning Injektormodul
en	Operating instructions Injector module
es	Instrucciones de manejo Módulo inyector
fi	Käyttöohje Suorasuihkumoduuli
fr	Mode d'emploi Module à injection
it	Istruzioni d'uso Modulo a iniezione
nl	Gebruiksaanwijzing injectormodule
no	Bruksanvisning injektormodul
pt	Instruções de utilização Módulo injetor
sv	Bruksanvisning injektormodul

de	4
da	17
en	30
es	43
fi	56
fr	69
it	82
nl	95
no	108
pt	121
sv	134

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Notes about these instructions	30
Intended use	31
Queries and technical problems.....	31
Items supplied	32
Load carriers	32
Optional accessories.....	32
Disposal of the packing material	32
Warning and Safety instructions	33
Areas of application	34
Routine checks for cleaning results	34
To check before loading the machine and before starting a programme	34
Manual pre-cleaning.....	34
Loading.....	35
Arranging and connecting shafted instruments	35
A 808 holder for Xi staplers	37
Cleaning the filter plate in the water intake	38
Pressure test point	40
Installation.....	41
Secure the holder in the module	41
Tools required:	41

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property.

Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow buttons and save your choice with OK.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

The A 602 module can be used in the A 502 injector unit. The injector unit is supplied with its own operating instructions.

8 mm shafted instruments for da Vinci® Xi system robots can be reprocessed in the A 602 module. A special A 808 holder is additionally required for the da Vinci® Xi stapler.

Miele have tested the reprocessing of these instruments with the ROBOTVARIO X-tra programme and thermosept® X-tra cleaning agent (manufactured by Schülke & Mayr) and recommend the use of this procedure.

This programme does not come as standard for PG 8527 and PG 8528 washer-disinfectors and must be added to the Profitronic unit retrospectively by Miele.

The washer-disinfector is generally referred to as “the machine” in these operating instructions. Reprocessable medical devices are referred to as “load items” if they are not more closely defined.

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your washer-disinfector or go to www.miele-professional.com.

Load carriers

A 602 module with 8 holders for shafted instruments for robot-assisted surgery, H 265, W 573, D 318 mm

Optional accessories

The following and other accessories are available to order from Miele:

- A 808 holder for da Vinci® Xi staplers

Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

To avoid the risk of accidents and damage to this module please read these instructions carefully before using it for the first time. Keep these instructions in a safe place where they are accessible to users at all times.

 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- The module is approved solely for the applications specified in the operating instructions. Components such as nozzles may only be replaced with Miele accessories or genuine Miele original spare parts. Alterations or conversion of the machine, or using it for purposes other than those for which it was intended, are not permitted and could be dangerous.
- The load carrier is set up by Miele during the commissioning process.
- New load carriers must be cleaned in the washer-disinfector without a load prior to first use.
- Inspect all load carriers as described in the "Maintenance" section in the operating instructions for the cleaning machine.
- Any hollow items that have become disconnected from their injector nozzles, adapters or holders during processing must be re-processed.

Miele cannot be held liable for damage caused by non-compliance with these Warning and Safety Instructions.

Routine checks for cleaning results

The process must be validated before using for the first time. Instruments should be dismantled on a sample basis for routine cleanliness checks after their last operative use and machine reprocessing.

To do this cap the Bowden cables on the guide wheels and pull off and dismantle the distal functional part.

The inner distal cylinder through which the Bowden cables are passed to the functional end must be checked optically with magnification and by protein analysis.

To check before loading the machine and before starting a programme

- Check that the filter plate in the water inlet pipe is in place and clean.
- Is the holder correctly connected to the module? (if present)

For validation purposes please make sure that you follow the loading instructions given on the template.

 Make sure that all screw connectors are fitted with injector nozzles, adapters, irrigation sleeves or blind stoppers to ensure that all rinsing fittings in use are supplied with sufficient standardised pressure.

Damaged fittings must not be used.

Damaged silicone tubes must be replaced with new ones before the module is used again.

Rinsing positions not in use do not have to be fitted with blind stoppers.


Manual pre-cleaning

 On-site pre-treatment and preparation for machine reprocessing must be carried out according to the manufacturer's instructions.


Instruments, particularly the distal functional ends, must be optically clean before they are loaded into the A 602 module.

Loading

Arranging and connecting shafted instruments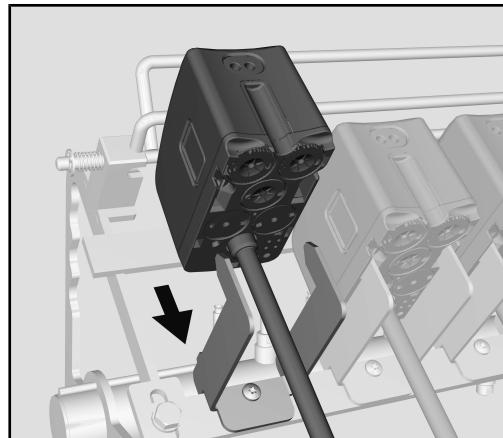


- Push the instrument into the holder from above.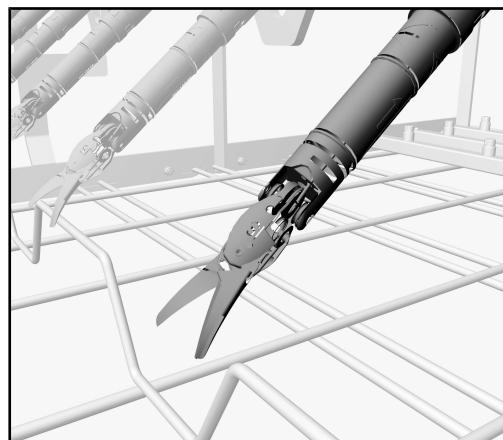

- Align the joints at the functional ends straight and lengthways to the shaft.


The joints at the functional ends must not be angled. The wash fluid can only access all surfaces if the functional ends are straight.

- Open the functional ends of the instruments.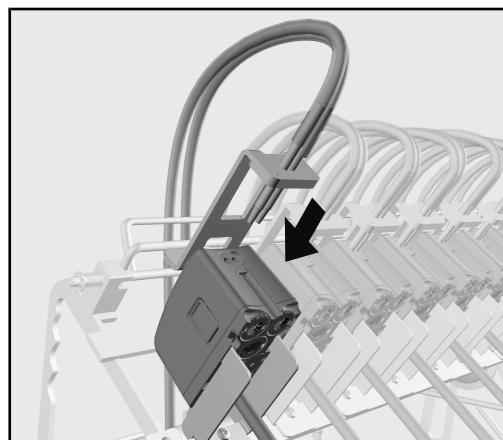

- Push the double tube connection into the rinsing attachment on the instrument.

en - Areas of application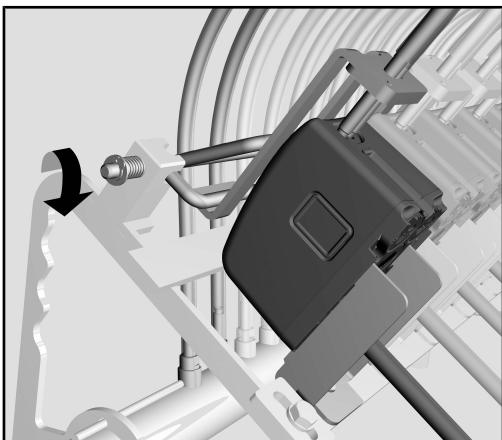


- When all instruments have been arranged, lower the rail to secure the connectors and click it into place.

The water supply pipe retaining plates are secured in their positions by the rail.

Water supply pipes which are not being used must be pushed into the holder behind the water inlet tube so that they are not thrown about by the force of the water during the cleaning process.


Instruments which cannot be secured by the retaining plates and rail cannot be reprocessed in this unit.

**A 808 holder for
Xi staplers**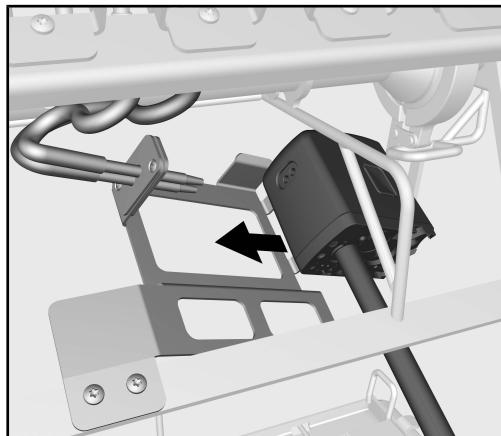

- Push the stapler into the holder underneath the cross rail in the unit.

The pipe connections on the holder must be pushed into the rinse connections on the stapler.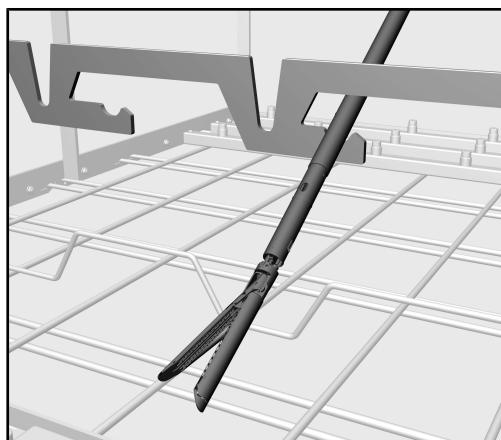


- Position the shaft of the stapler in the holder on the supporting rail of the A 502 injector unit.
- Align the joint in the functional end of the stapler in the direction of the water pipe.
- Open the functional end of the stapler.

Cleaning the filter plate in the water intake


There is a filter plate in the water inlet pipe in front of the double connections for shafted instruments.

This prevents soiling from other disciplines from being deposited in the lumen of instruments.


 The filter plate must be cleaned before every programme.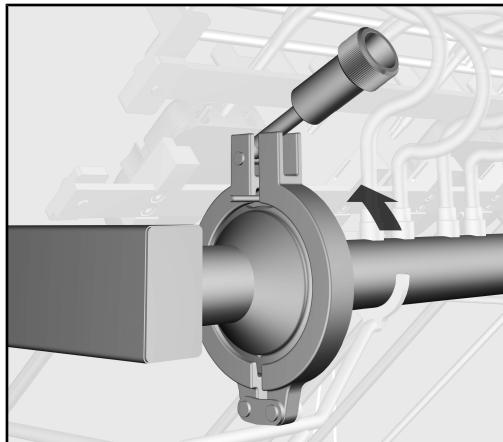


- Unscrew the nut on the clip around the holder for the filter plate and open the clip.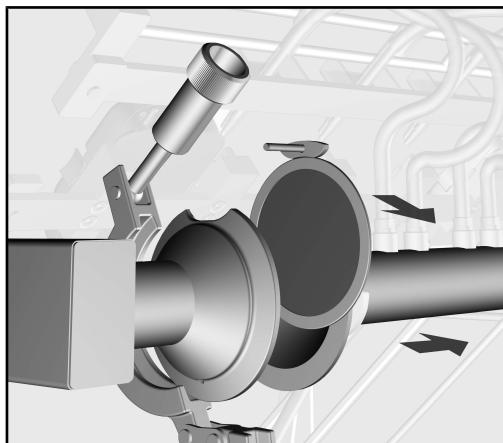

- Pull the right side of the water inlet pipe out to the side.
- Remove the filter plate.
- Remove deposits and soiling from the filter plate with running water, using a brush if necessary.
- Blow the filter plate from the back with compressed air against the direction of flow.

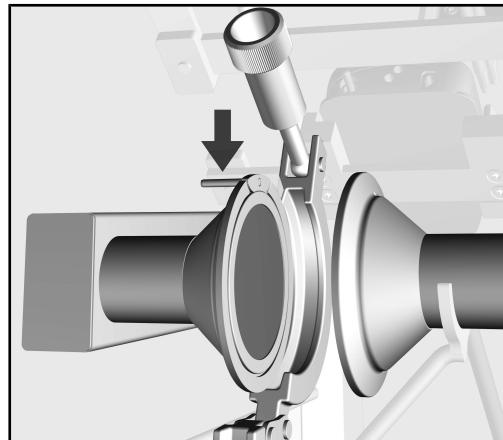
- Place the filter plate in the left hand side of the holder and hold by the top edge.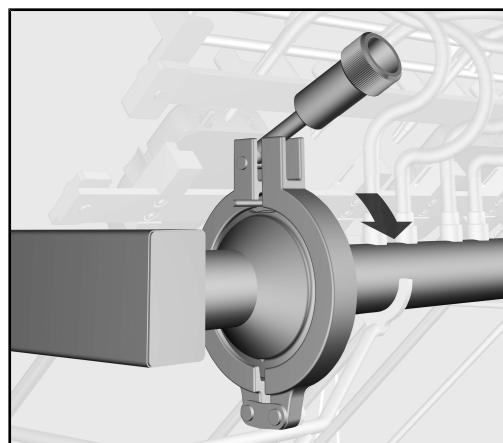

- Push the right hand side of the water inlet pipe to the left until the holder is closed.
- Place the clip around the edge of the holder and tighten using the nut.

Pressure test point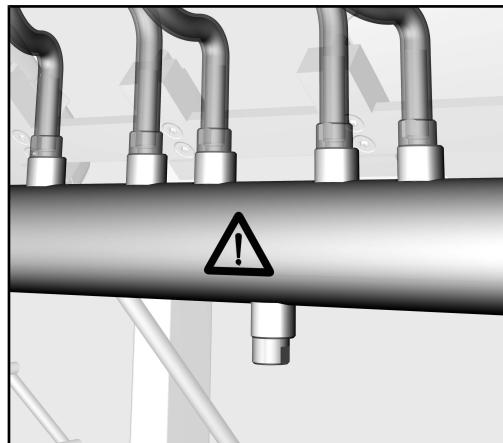

The pressure test point for performance checks and for validation according to EN ISO 15883 is located at the bottom on the water inlet pipe for the spray arm.

- To measure water pressure replace the blind stopper with a Luer Lock adapter, e.g. E 447.

Under no circumstances must instruments, nozzle manifolds etc. be connected to the pressure test point.

After taking measurements the pressure test point must be capped with the blind stopper.


Secure the holder in the module

Tools required:

- 9 mm Spanner (WAF 9)


- TORX T20 screwdriver

There are holes in the cross rail of the module which the holders are screwed into.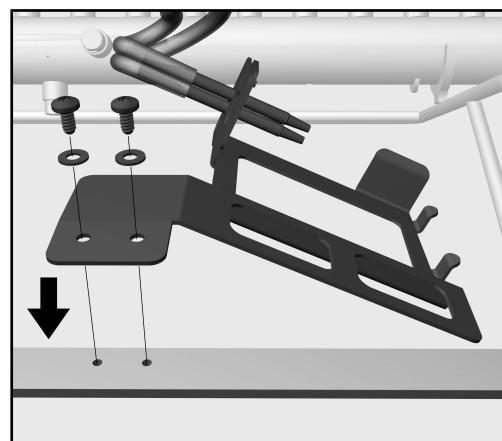

- Place the holder along the cross rail in the module.
- Secure the screws together with washers.

The two hoses for the rinse attachments on the casing of the Xi stapler must be connected to the module water inlet tube.

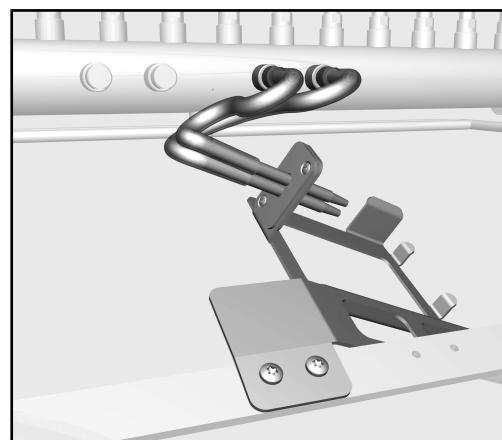- Remove two blind stoppers on the front side of the water inlet tube.
- Screw the hoses from the holder to the water inlet tube.

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