# A 600

<b>de</b>	Gebrauchsanweisung Injektormodul
<b>da</b>	Brugsanvisning Injektormodul
<b>en</b>	Operating instructions Injector module
<b>es</b>	Instrucciones de manejo Módulo inyector
<b>fi</b>	Käyttöohje Suorasuihkumoduuli
<b>fr</b>	Mode d'emploi Module à injection
<b>it</b>	Istruzioni d'uso Modulo a iniezione
<b>nl</b>	Gebruiksaanwijzing injectormodule
<b>no</b>	Bruksanvisning injektormodul
<b>pt</b>	Instruções de utilização Módulo injetor
<b>sv</b>	Bruksanvisning injektormodul

<b>de</b>	.....	4
<b>da</b>	.....	22
<b>en</b>	.....	39
<b>es</b>	.....	56
<b>fi</b>	.....	74
<b>fr</b>	.....	91
<b>it</b>	.....	109
<b>nl</b>	.....	128
<b>no</b>	.....	146
<b>pt</b>	.....	163
<b>sv</b>	.....	181

---

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# en - Contents

---

<b>Notes about these instructions .....</b>	39
<b>Intended use .....</b>	40
Queries and technical problems.....	40
<b>Items supplied .....</b>	41
Load carriers .....	41
Optional accessories.....	41
Disposal of the packing material .....	41
<b>Warning and Safety instructions .....</b>	42
<b>Areas of application .....</b>	43
Routine checks for cleaning results .....	43
To check before loading the machine and before starting a programme .....	43
Manual pre-cleaning.....	43
Loading.....	44
Arranging and connecting shafted instruments .....	44
A 805 holder for stapler S/Si .....	46
A 806 holder for Single-Site® instruments with rinse connection on the shaft .....	47
A 807 holder for PK™ dissecting forceps .....	50
Cleaning the filter plate in the water intake .....	51
Pressure test point .....	53
<b>Installation.....</b>	54
Secure the holder in the module .....	54
Tools required:.....	54

## Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property.

Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

## Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

## Additional information and comments

Additional information and comments are contained in a simple frame.

## Operating steps

Operating steps are indicated by a black square bullet point.

### Example:

- Select an option using the arrow buttons and save your choice with OK.

## Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

### Example:

Settings  menu.

## en - Intended use

The A 600 module can be used in the A 502 injector unit. The injector unit is supplied with its own operating instructions.

5 mm and 8 mm shafted instruments for da Vinci® S and Si system robots that have two rinse connections on the upper end of the casing can be reprocessed in the A 600 module.

Special instruments cannot be reprocessed in the A 600 module holders.

Additional holders are required for reprocessing these. See chart.

Miele have tested the reprocessing of instruments with the ROBOTVARIO X-tra programme and thermosept® X-tra cleaning agent (manufactured by Schülke & Mayr) and recommend the use of this procedure.

This programme does not come as standard for PG 8527 and PG 8528 washer-disinfectors and must be added to the Profitronic unit retrospectively by Miele.


Suitable instruments	A 600
EndoWrist® 8 mm	•
EndoWrist® 5 mm	•
Single-Site® without rinse connection on the shaft	•
Single-Site® with rinse connection on the shaft	+ A 806
PK™ Dissecting forceps	+ A 807
da Vinci® S/Si stapler	+ A 805

The washer-disinfector is generally referred to as “the machine” in these operating instructions. Reprocessable medical devices are referred to as “load items” if they are not more closely defined.

## Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your washer-disinfector or go to [www.miele-professional.com](http://www.miele-professional.com).

## Load carriers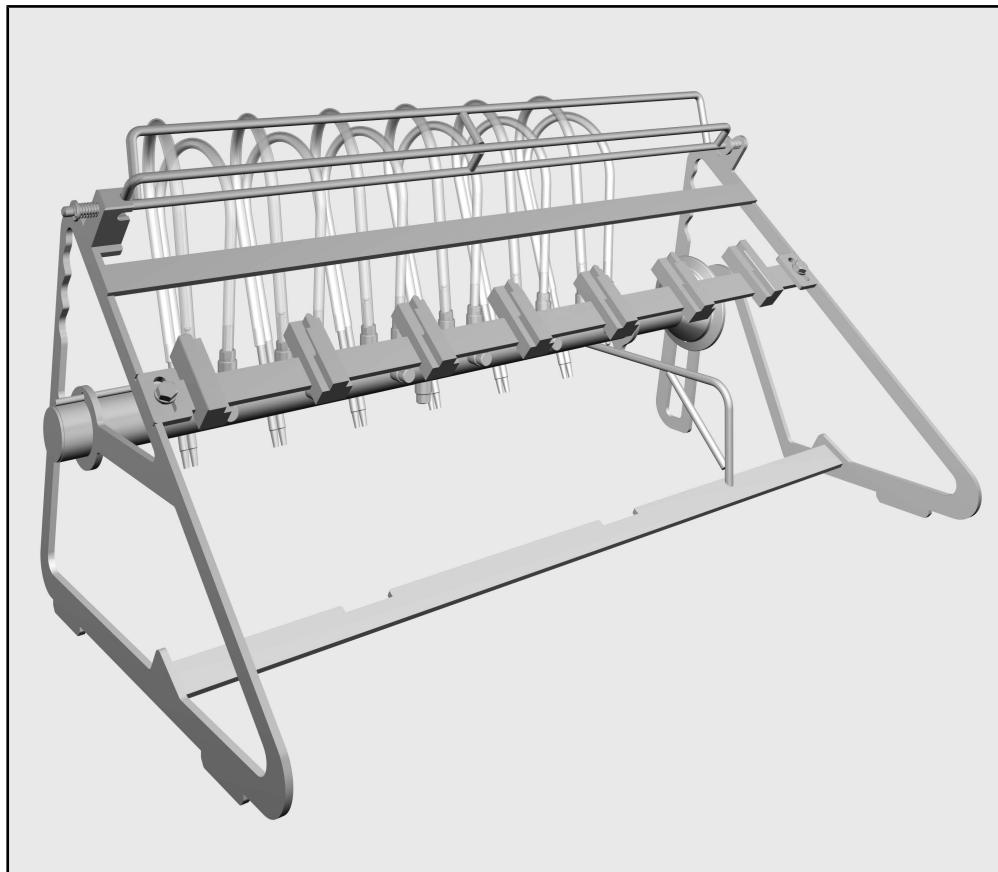

A 600 module with 6 holders for shafted instruments for robot assisted surgery, H 265, W 573, D 318 mm

### Optional accessories

The following and other accessories are available to order from Miele:

- A 805, holder for da Vinci® S/Si stapler and hose adapter for the rinse connection on the shaft of the stapler
- A 806, holder for Single-Site® with rinse connection on the shaft and hose adapter for the rinse connection on the shaft of the instrument
- A 807, holder for PK™ dissecting forceps

### Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

## en - Warning and Safety instructions

To avoid the risk of accidents and damage to this module please read these instructions carefully before using it for the first time. Keep these instructions in a safe place where they are accessible to users at all times.

 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- The module is approved solely for the applications specified in the operating instructions. Components such as nozzles may only be replaced with Miele accessories or genuine Miele original spare parts. Alterations or conversion of the machine, or using it for purposes other than those for which it was intended, are not permitted and could be dangerous.
- The load carrier is set up by Miele during the commissioning process.
- New load carriers must be cleaned in the washer-disinfector without a load prior to first use.
- Inspect all load carriers as described in the “Maintenance” section in the operating instructions for the cleaning machine.
- Any hollow items that have become disconnected from their injector nozzles, adapters or holders during processing must be re-processed.

Miele cannot be held liable for damage caused by non-compliance with these Warning and Safety Instructions.

## Routine checks for cleaning results

The process must be validated before using for the first time. Instruments should be dismantled on a sample basis for routine cleanliness checks after their last operative use and machine reprocessing.

To do this cap the Bowden cables on the guide wheels and pull off and dismantle the distal functional part.

The inner distal cylinder through which the Bowden cables are passed to the functional end must be checked optically with magnification and by protein analysis.

## To check before loading the machine and before starting a programme

- Check that the filter plate in the water inlet pipe is in place and clean.
- Is the inserted holder correctly connected to the module (if present)?
- Is the hose adapter correctly connected to the rinse connection on the shaft (if present)?

For validation purposes please make sure that you follow the loading instructions given on the template.

 Make sure that all screw connectors are fitted with injector nozzles, adapters, irrigation sleeves or blind stoppers to ensure that all rinsing fittings in use are supplied with sufficient standardised pressure.

Damaged fittings must not be used.

Damaged silicone tubes must be replaced with new ones before the module is used again.

Rinsing positions not in use do not have to be fitted with blind stoppers.


## Manual pre-cleaning

 On-site pre-treatment and preparation for machine reprocessing must be carried out according to the manufacturer's instructions.


Instruments, particularly the distal functional ends, must be optically clean before they are loaded into the A 600 module.

## Loading

### Arranging and connecting shafted instruments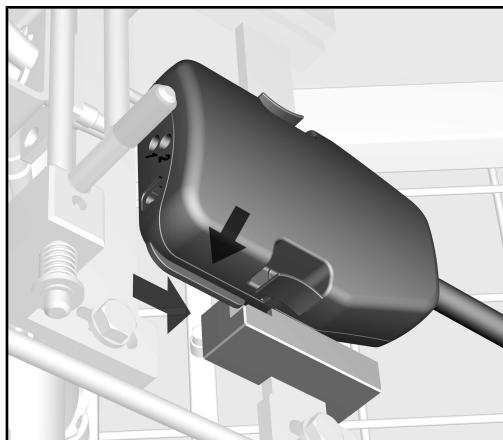


- Push the instrument into the holder from above.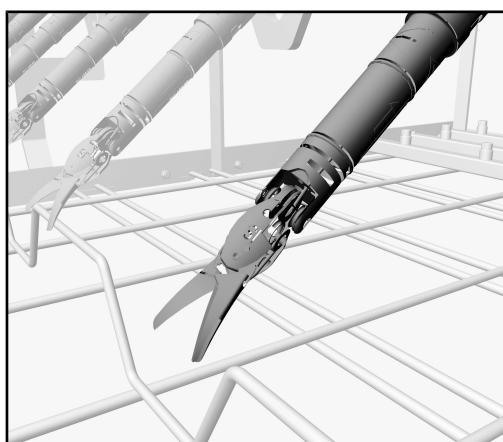

- Align the joints at the functional ends of the instruments lengthways in line with the shaft.


The joints at the functional ends must not be angled. The wash fluid can only access all surfaces if the functional ends are straight.

- Open the functional ends of the instruments.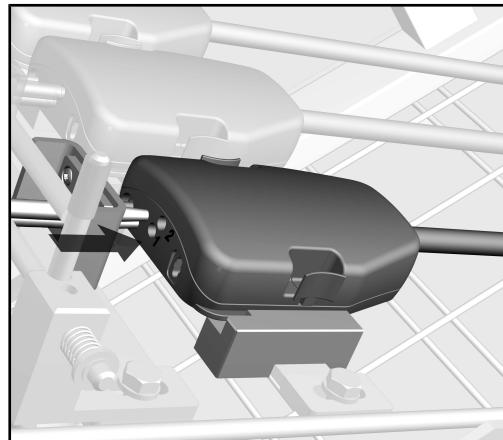

- Push the twin connector into the rinse attachment on the instrument.

The third rinse aperture on the shaft of 5 mm rigid instruments does not need to be connected for machine reprocessing.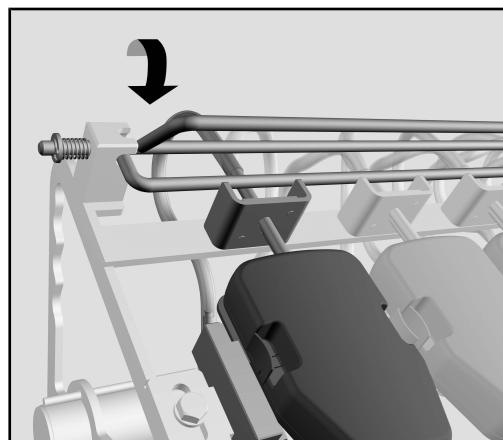

- When all instruments have been arranged, lower the rail to secure the connectors and click it into place.

The water supply pipe retaining plates are secured in their positions by the rail.

Water supply pipes which are not being used must also be positioned under the rail so that they are not thrown about by the force of the water during the cleaning process.

Instruments which cannot be secured by the retaining plates and rail cannot be reprocessed in this module.


At the end  
of a programme

- Take the instruments out of the module by carrying out the process steps in reverse.


## en - Areas of application

### A 805 holder for stapler S/Si

The rinse connection on the shaft of the stapler must face upwards.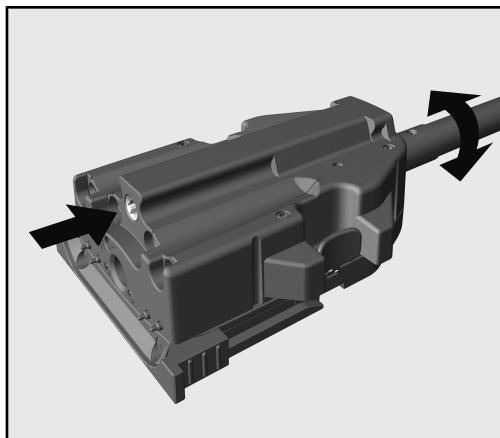

- Press in the middle motor connection on the upper end of the housing slightly and turn the shaft until the rinse connection is facing upwards.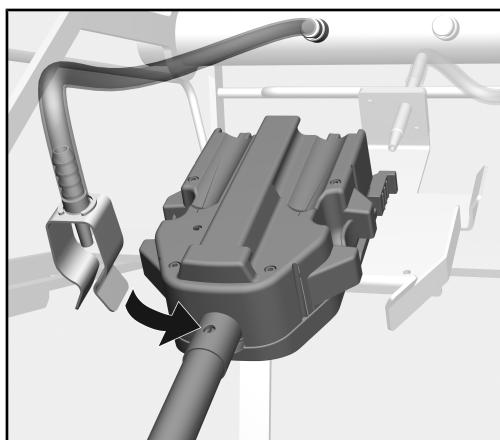


- Connect the hose adapter to the rinse connection on the shaft of the stapler.

The rinse pipe must click into place in the rinse connection. To do this:

- If required, turn the clamp around the shaft until the rinse pipe clicks into position.


The clamp is used to secure the hose adapter.

If the hose adapter has come loose from the rinse connection on the stapler during reprocessing, it must be reprocessed again.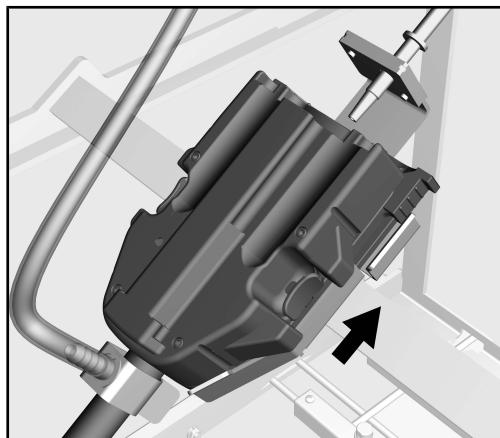

- Fit the stapler into the holder from below.

The pipe connection on the holder must be pushed into the rinse connection on the stapler.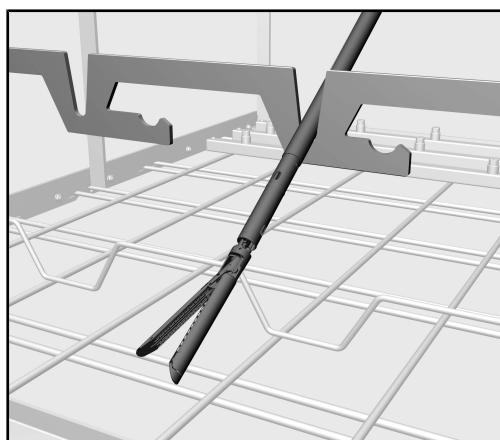

- Position the shaft of the stapler in the V-shaped recess in the supporting rail of the A 502 injector unit.
- Align the joint in the functional end of the stapler in the direction of the water pipe.
- Open the functional end of the stapler.

At the end  
of a programme


- First take the stapler with the hose adapter attached out of the holder.
- Then detach the hose adapter from the shaft of the stapler.

Important. Take care as the hose adapter clamp is very strong.

The hose adapter must be unscrewed and replaced with a blind stopper if the holder is not in use. Otherwise the loose end of the hose adapter will be thrown about by the force of the water and could cause damage.

## en - Areas of application


A 806 holder for Single-Site® instruments with rinse connection on the shaft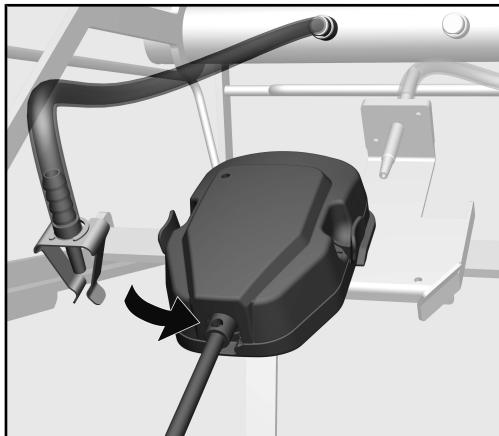

- Push the hose adapter into the rinse connection on the shaft of the instrument.


The clamp is used to secure the hose adapter.

If the hose adapter has come loose from the rinse connection on the instrument during reprocessing, it must be reprocessed again.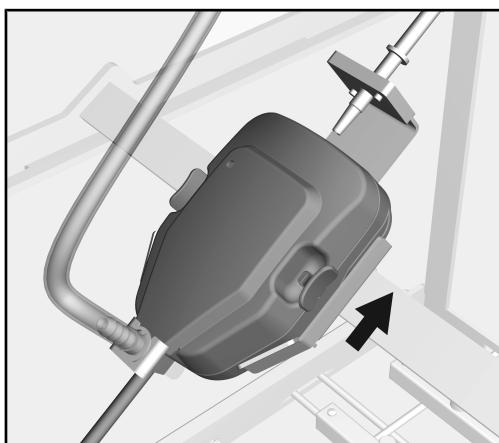

- Fit the instrument into the holder from below.

The pipe connection on the holder must be pushed into the rinse attachment on the instrument.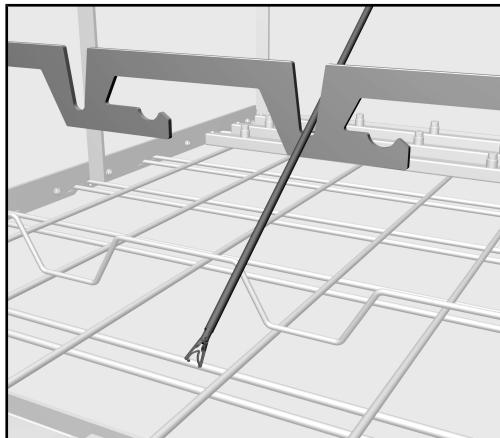

At the end  
of a programme


- Position the shaft of the instrument in the V-shaped recess in the supporting rail of the A 502 injector unit.
- Align the joint at the functional end of the instrument lengthways in line with the shaft.
- Open the functional end of the instrument.
- First take the instrument together with the attached hose adapter out of the holder.
- Then detach the hose adapter from the instrument shaft.

Important. Take care as the hose adapter clamp is very strong.

**⚠** The hose adapter must be unscrewed and replaced with a blind stopper if the holder is not in use. Otherwise the loose end of the hose adapter will be thrown about by the force of the water and could cause damage.


## en - Areas of application

### A 807 holder for PK™ dissecting forceps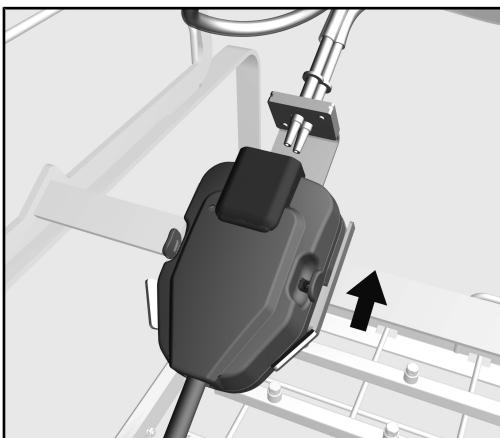

- Fit the instrument into the holder from below.

The pipe connections on the holder must be pushed into the rinse attachments on the instrument.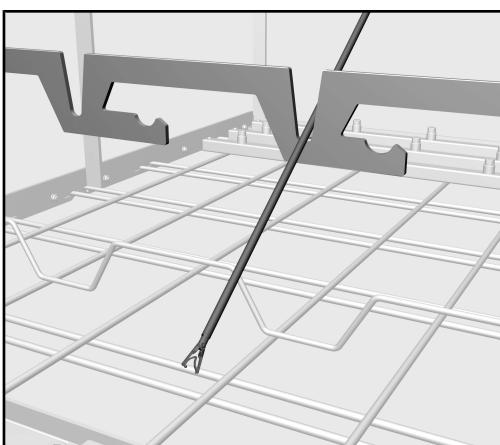


- Position the shaft of the instrument in the V-shaped recess in the supporting rail of the A 502 injector unit.
- Align the joint at the functional end of the instrument lengthways in line with the shaft.
- Open the functional end of the instrument.

## Cleaning the filter plate in the water intake


There is a filter plate in the water inlet pipe in front of the double connections for shafted instruments.

This prevents soiling from other disciplines from being deposited in the lumen of instruments.

**⚠ The filter plate must be cleaned before every programme.**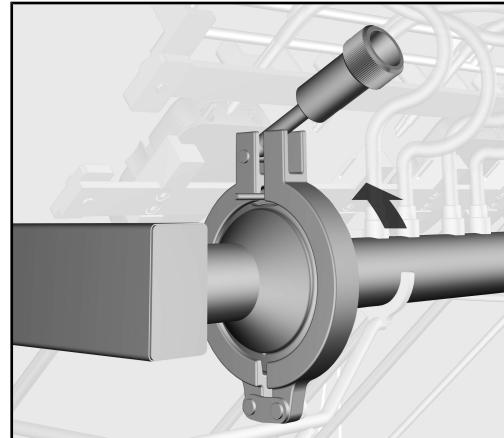


- Unscrew the nut on the clip around the holder for the filter plate and open the clip.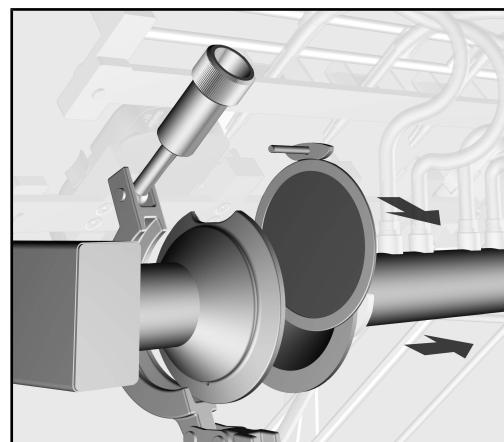


- Pull the right side of the water inlet pipe out to the side.
- Remove the filter plate.
- Remove deposits and soiling from the filter plate with running water, using a brush if necessary.
- Blow the filter plate from the back with compressed air against the direction of flow.

## en - Areas of application

---

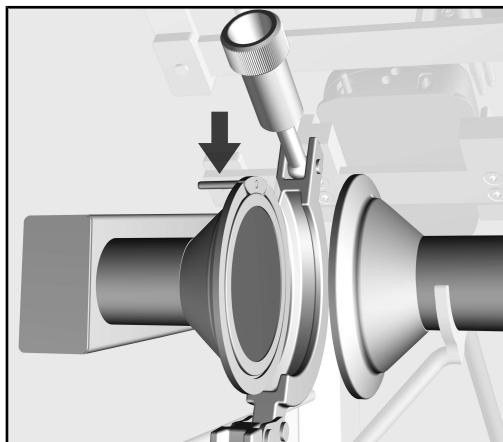
- Place the filter plate in the left hand side of the holder and hold by the top edge.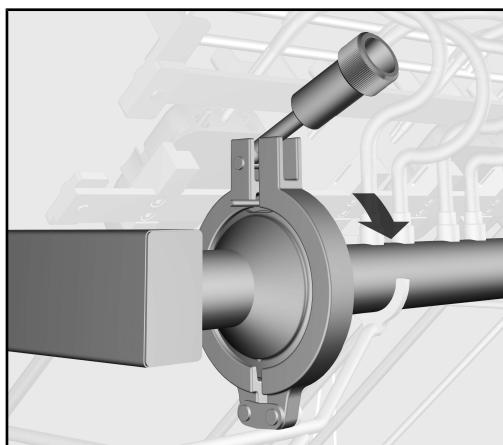

- Push the right hand side of the water inlet pipe to the left until the holder is closed.
- Place the clip around the edge of the holder and tighten using the nut.

### Pressure test point

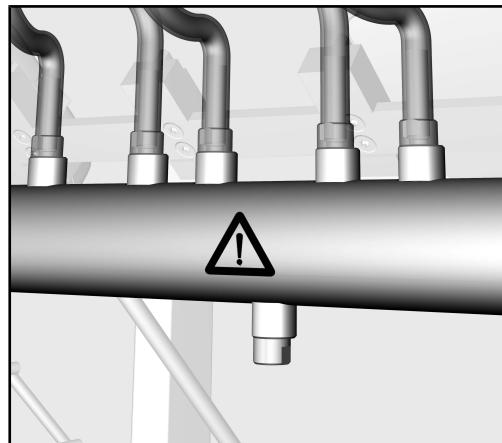The pressure test point for performance checks and for validation according to EN ISO 15883 is located at the bottom on the water inlet pipe for the spray arm.

- To measure water pressure replace the blind stopper with a Luer Lock adapter, e.g. E 447.

Under no circumstances must instruments, nozzle manifolds etc. be connected to the pressure test point.


After taking measurements the pressure test point must be capped with the blind stopper.

## Secure the holder in the module

### Tools required:


- 9 mm Spanner (WAF 9)

A 805, A 806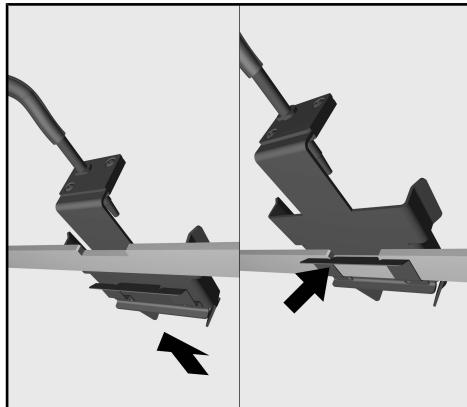

The hose for the rinse attachment at the upper end of the casing must be connected to the module water inlet pipe. A special hose adapter is required for the rinse connection on the shaft of the stapler or instrument. This must also be connected to the water inlet pipe.


A 807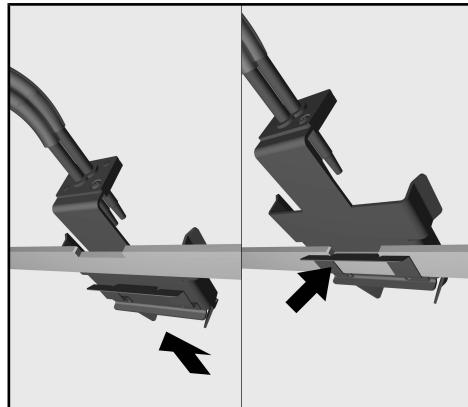

The two hoses for the rinse attachments at the upper end of the casing of the PK™ Dissecting Forceps must be connected to the module water inlet pipe.

Two notches are located on the cross rail of the module to which the holders can be secured.

- Push the holder onto one of the notches on the module until it clicks into place.

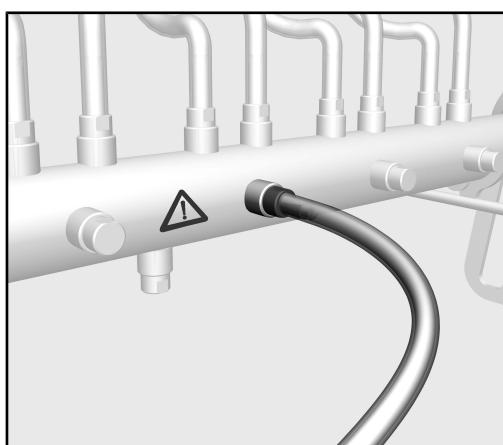- Remove the caps from the front side of the water inlet pipe.
- Screw the hoses from the holder and the hose adapter (if present) to the water inlet pipe.

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