Mobile units E 910/4, E 911/3 and E 912/2 for containers

Items supplied:

Mobile unit E 910/4 for 6 DIN containers plus lids, H 454, W 636, D 790 mm, with magnetic strip for automatic mobile unit recognition

Mobile unit E 911/3 for 4 DIN containers plus lids, H 454, W 636, D 790 mm, with magnetic strip for automatic mobile unit recognition

Mobile unit E 912/2 for 4 ISO containers plus lids, H 454, W 636, D 790 mm, with magnetic strip for automatic mobile unit recognition

⚠ Please read the Warning and Safety instructions in the Operating Instructions for your G 7827/28 or PG 8527/28 Washer-disinfector carefully. You should only process instruments/medical devices in this mobile unit if they have been declared as suitable for machine processing by the manufacturer. Please also observe the manufacturer's care instructions. The programme selected must be suitable for the materials from which the medical devices to be processed are made. Where possible, the programme should include thermal disinfection. The final rinse should, if possible, always be carried out with fully de-mineralised water.

Areas of application:

The E 910/4 Mobile unit can be used to process six shallow DIN containers plus their lids (160 x 300 x 600 mm*).

The E 911/3 Mobile unit can be used to process four deep DIN containers plus their lids (270 x 300 x 600 mm*).

The E 912/2 Mobile unit can be used to process four ISO containers plus their lids (180 x 400 x 600 mm).

*Dimensions are given in mm for containers including their lids (height x width x length), manufactured in accordance with DIN EN 868-8

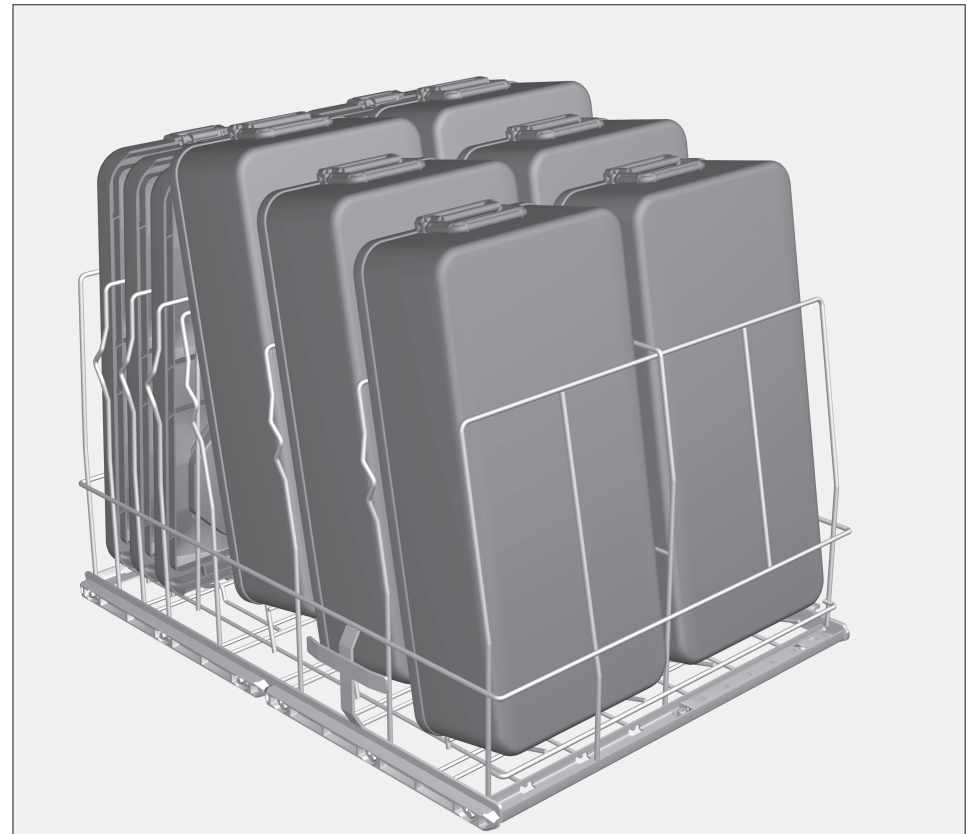
Important:

A pH neutral cleaning agent must be used for anodised aluminium containers. Coloured anodised aluminium containers must be cleaned, thermally disinfected and rinsed using fully de-mineralised water.


- Add a suitable rinsing agent when processing plastic containers and lids for quicker and better drying results.
- Remove any textile or paper filters from lids before processing.
- Teflon filters may be left in lids unless manufacturer's instructions state otherwise.
- Check that the valve is functioning correctly after processing disposal containers with valves.

Note:

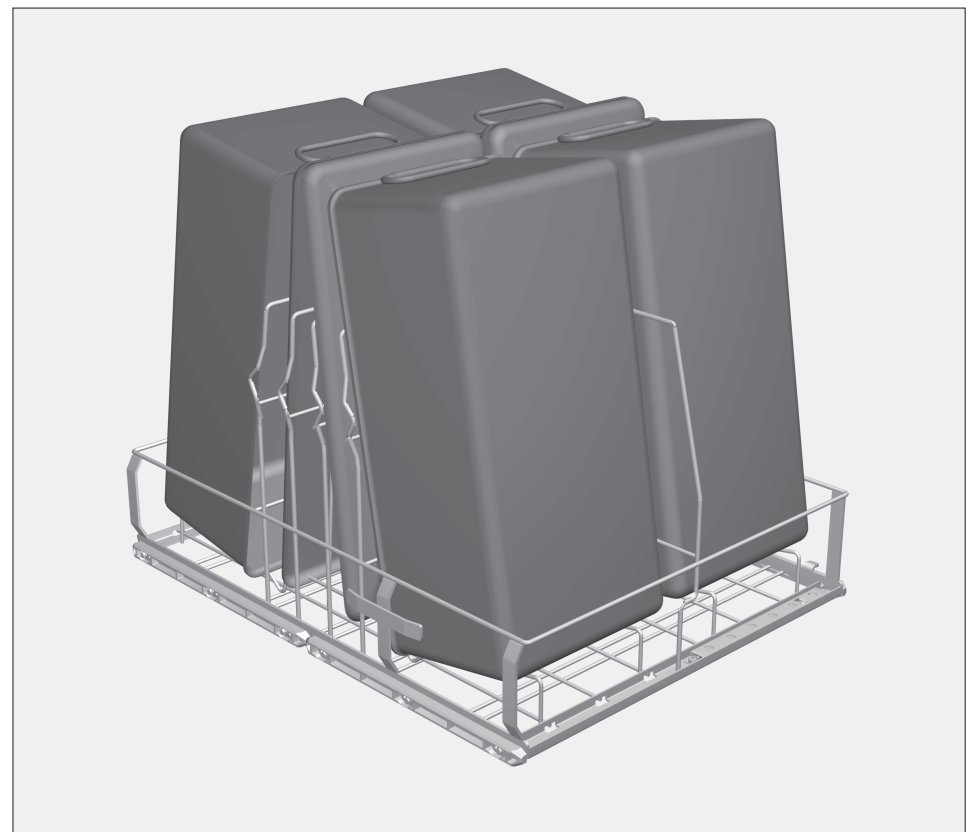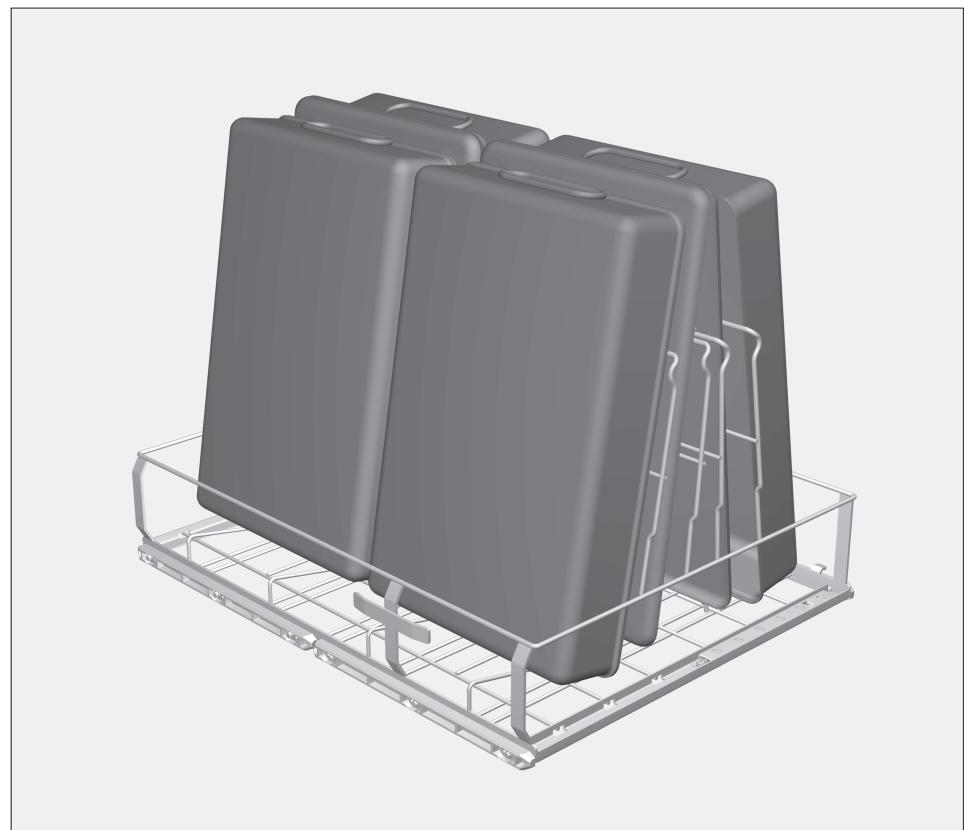
Containers generally present no danger of injury to personnel. However, the person responsible for hygiene standards should clarify whether an A_0 - value of 60 or 600 should be achieved with thermal disinfection.

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