Add-on module E 702/1

Items supplied:

Add-on module E 702/1 for E 701/2 and E 525/2 (H 160, W 530, D 560 mm)

⚠ Please read the Warning and Safety instructions in the Operating Instructions for your washer-disinfector carefully. You should only process instruments/medical products in this unit if they have been declared as suitable for machine processing by the manufacturer. Please also observe the manufacturer's care instructions. The programme selected must be suitable for the materials from which the medical products to be processed are made. Where possible the programme should include thermal disinfection. The final rinse should, if possible, always be carried out with fully demineralised water. If using automatic mobile unit recognition, please make sure that the coding is correctly set.


Areas of application:

Additional mesh trays containing surgical instruments can be processed in mobile units 701/2 and E 525/2 when the add-on module is attached. This requires the add-on module to be connected to the water intake for the mobile unit.

⚠ Please also observe the mobile unit Operating instructions for surgical instruments.

Before loading the machine and before starting a programme, please check:


- that the spray arms rotate freely,
- that the spray arm jets are clean and free of any blockages,
- that the mobile unit is correctly fitted into the cabinet and that it has connected correctly with the water intake.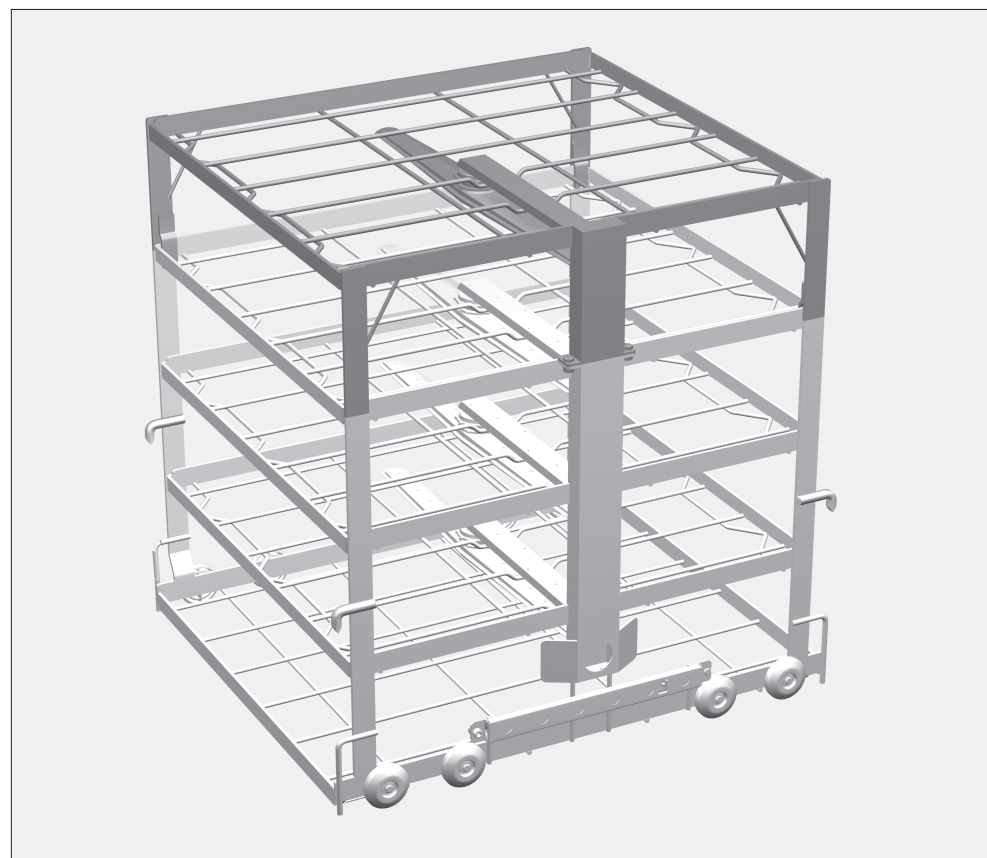

E 701/2 fitted with E 702/1 add-on module

For validation purposes please make sure that you follow the loading instructions given on the template.

To fit the add-on module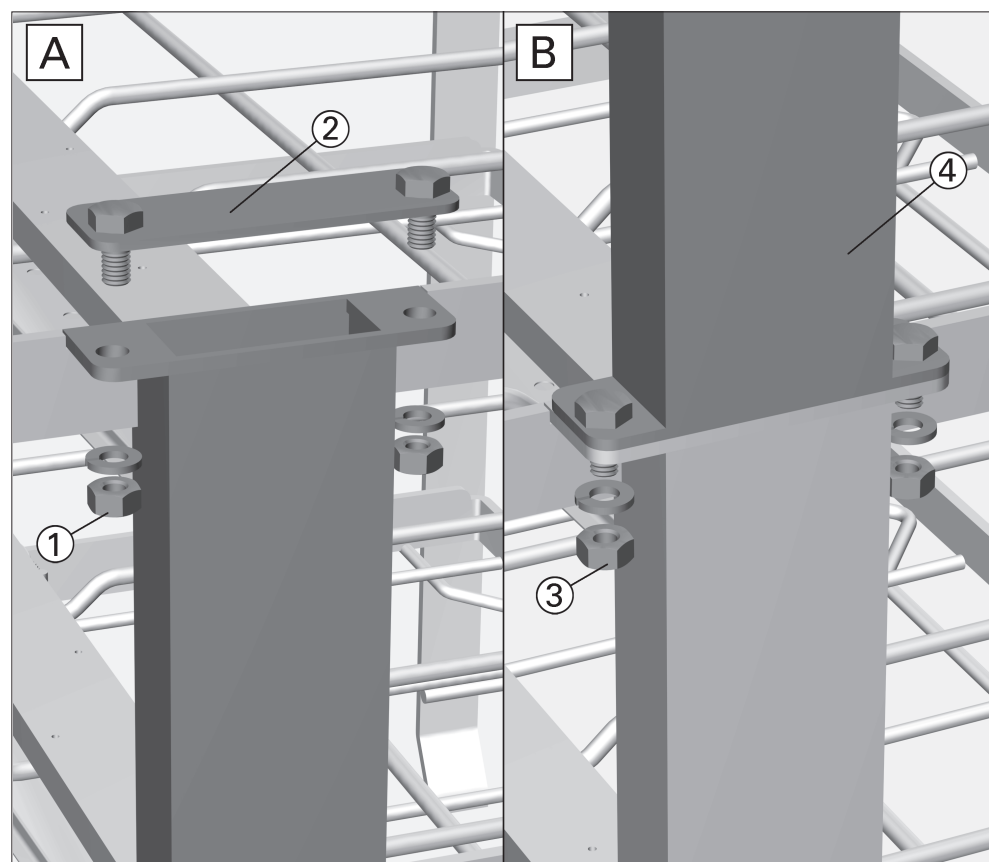


- Unscrew nuts ① on the securing screws on the mobile unit with an SW 10 spanner, and remove the spring washers.
- Remove blanking plate ② from the water intake.

Keep blanking plate ②, nuts ① and the spring washers in case the add-on module is removed at a later date.

- To connect the add-on module, first unscrew nuts ③ from the fixing screws on add-on module ④ with an SW 10 spanner, and remove the spring washers.
- Place add-on module ④ on top of the mobile unit. See illustration above for installation details.

After replacing the fixing screws:

- Replace the spring washers on the fixing screws, and then nuts ③, and tighten with an SW 10 spanner.

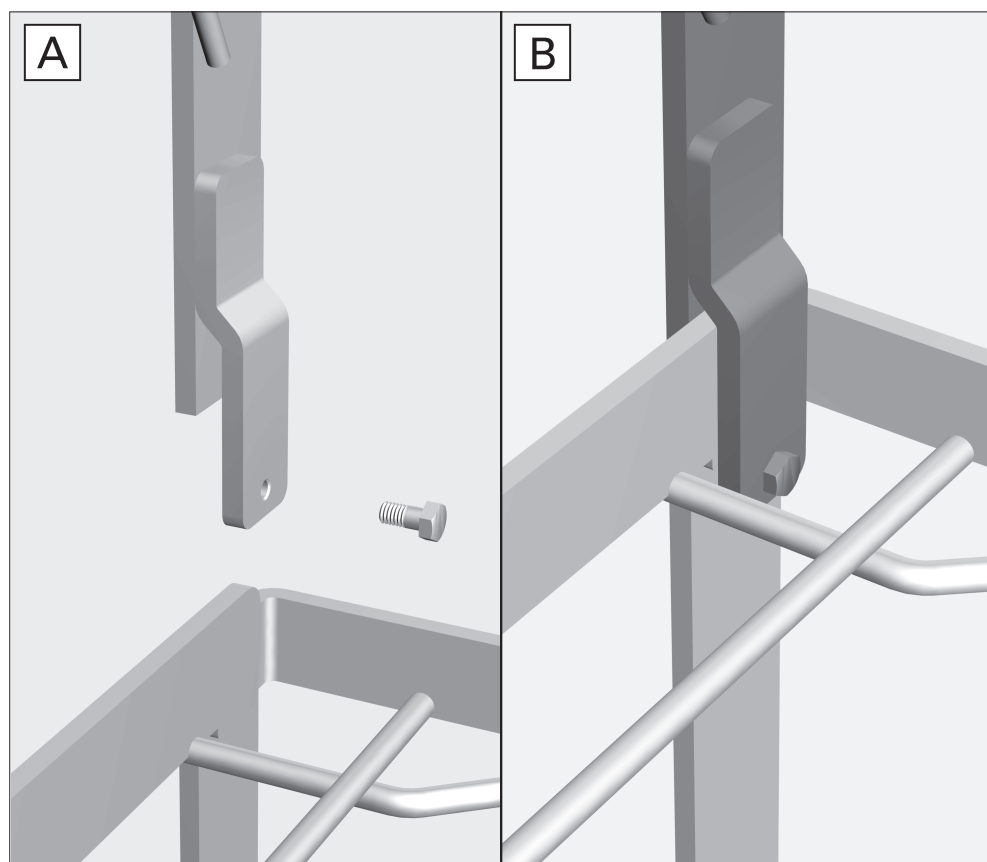To tighten the fixing screws

A:

- Slot the side hooks on the add-on module over the outer frame of the mobile unit for surgical instruments.
- Lower the add-on module onto the mobile unit, and secure to the mobile unit through the drill holes using the fixing screws. See B in illustration above.

B:

- Tighten fixing screws with an SW 7 spanner.

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