⚠ Please read the Warning and Safety instructions in the Operating Instructions for your washer-disinfector carefully. You should only process utensils/medical products in this basket if they have been declared as suitable for machine processing by the manufacturer. Please also observe the manufacturer's care instructions. The programme selected must be suitable for the materials from which the items to be processed are made. Where possible the programme should include thermal disinfection.

Areas of application:


Lower basket U 167 is designed for use in the following washer-disinfectors: G 7882, G 7882 CD, G 7892, G 7835, G 7836, PG 8535 and PG 8536. It is suitable for cleaning, disinfecting and drying soda lime containers and secretion flasks used in anaesthetics and intensive care.

⚠ These must be emptied before machine processing to ensure that any residual respiratory excretions have been completely removed.

The picture above illustrates how to arrange items over the holders and in between them.

- Cylindrical sections of soda lime containers which require dismantling should be secured in position by four holders in the corners of the basket.
- Secretion flasks should be placed over holders in the centre of the basket to ensure that the water jets can reach the bottom of the flasks. Stubborn dried on soiling may require a pre-wash or soak before cleaning, disinfecting and drying.

⚠ Observe permissible drying temperatures to prevent any premature material deterioration. Please be aware that the use of rinsing agent can cause stress cracks and corrosion to plastic vessels.

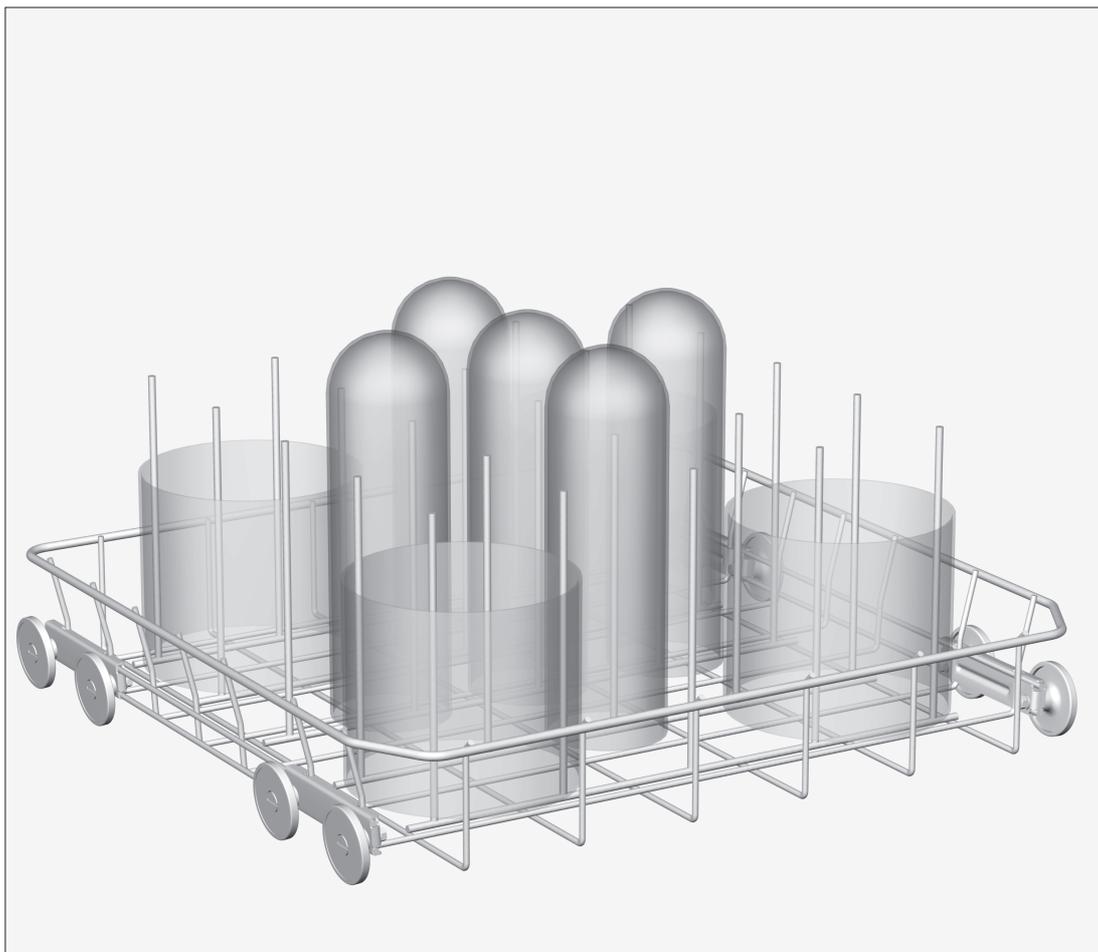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