Mobile unit E 439/3 for surgical instruments

⚠ Please read the Warning and Safety instructions in the Operating Instructions for your G 7836 and PG 8536 Washer-disinfector carefully. You should only process instruments/medical products in this mobile unit if they have been declared as suitable for machine processing by the manufacturer. Please also observe the manufacturer's care instructions. The programme selected must be suitable for the materials from which the medical products to be processed are made. Where possible the programme should include thermal disinfection. The final rinse should, if possible, always be carried out with fully de-mineralised water. If using automatic mobile unit recognition please make sure that the coding is correctly set.

Items supplied:

Mobile unit E 439/3 with four levels, H 503, W 536, D 518 mm, fitted with a magnetic strip for automatic mobile unit recognition and spray arm magnets for spray arm monitoring (PG 8536 only).

Accessories available to order:

- Magnet strip ML/2 for automatic mobile unit recognition
- E 142 DIN mesh tray
- E 492 insert for kidney dishes
- E 476 holders for micro instruments
- E 473 insert for very small items

Areas of application:

Mobile unit E 439/3 enables surgical instruments to be cleaned and disinfected in 7 DIN mesh trays. The lower levels can take 2 DIN mesh trays each. The top level can take 1 mesh tray.


The second level from the bottom can be taken out if necessary.

Loading height (levels numbered from the bottom upwards) [H x W x D in mm]	
Level 1	76 x 488 x 510
Level 1 with level 2 removed	160 x 488 x 510
Level 2	74 x 510 x 510
Level 3	94 x 520 x 475
Level 4	95 x 490 x 460

Before loading the machine and before starting a programme please check:

- that the spray arms rotate freely,
- that the spray arm jets are clean and free of any blockages,
- that the mobile unit is correctly fitted into the cabinet and that it has connected correctly with the water intake.


For validation purposes please make sure that you follow the loading instructions given on the template.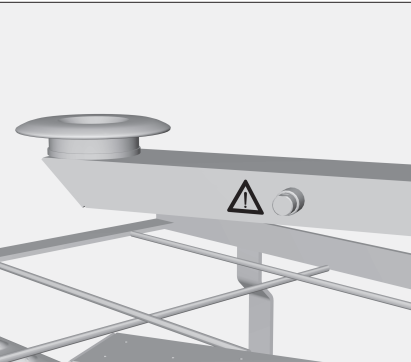

Access port for wash pressure measurements, e.g. within context of performance testing and validation in accordance with DIN ISO 15883.

- To carry out wash pressure measurements, exchange the screw stopper for a Luer lock adapter, e.g. E 447.

⚠ Under no circumstances may the access port be used to connect instruments, spray rails, etc.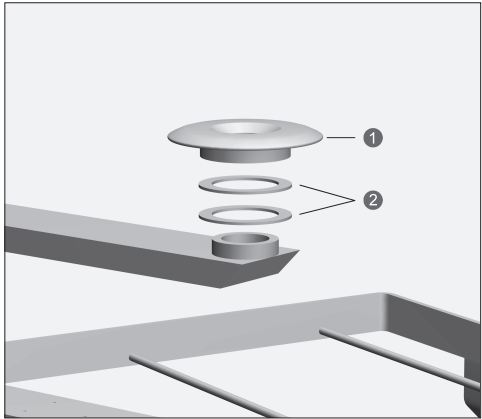


You will feel a resistance when pushing the mobile unit into the cabinet when it is correctly connected to the water intake.


If there is too much resistance you can remove one to two spacers from the water intake point.

To do this:


- Unscrew disc ① from the water intake point.
- Remove spacer ②.
- Screw disc ① back onto the water intake point.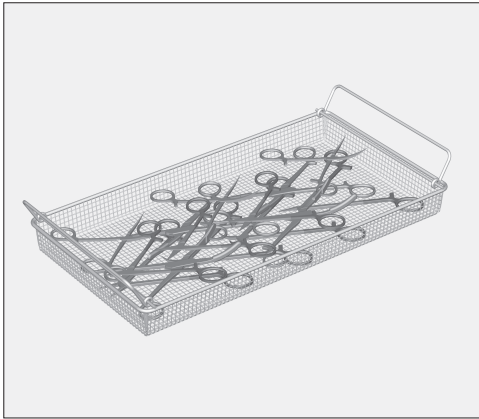

- Hinged instruments must be opened up before being placed in the mesh trays to minimise the risk of surfaces not being accessed during processing.

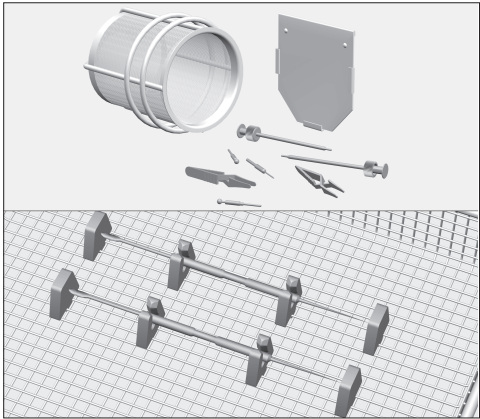
- To prevent the risk of light instruments such as bulldog clips and other small items being moved about during processing, silicone holders e.g. E 473 and closable inserts e.g. E 476 can be used to keep them secure.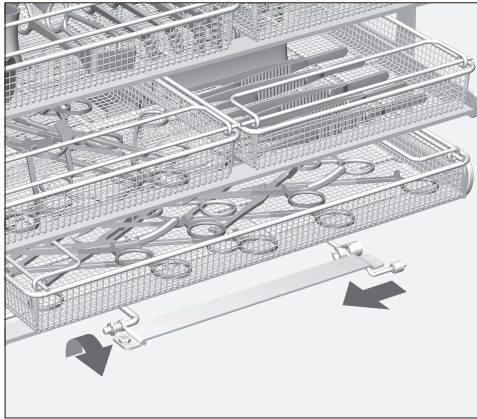


- The holder for the magnet strip must be opened out as shown in order to push mesh trays into the lowest level.

DIN mesh trays have to be arranged across the basket.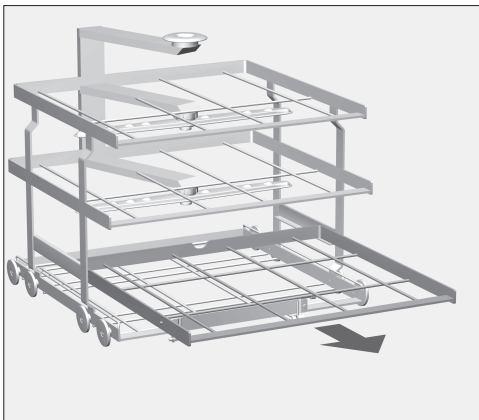

- The second level from the bottom is removable to accommodate tall items.

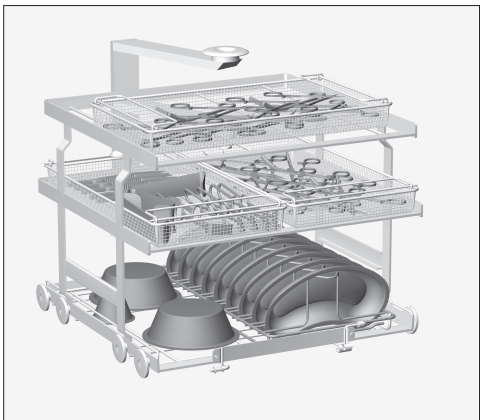- Containers, e.g. beakers and kidney dishes should be placed in the mobile unit with the open side facing downwards.
- Where applicable use an insert, e. g. an E 492.

Containers must be placed such that they do not hinder the processing of instruments. They must also be secure so that they cannot be flipped over by the force of water during processing.

If hollow items are placed such that cleaning solution can collect in recesses etc. they will not get properly clean and rinsing results will be impaired.

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