**de**

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**en**

### **Injector jets for laryngeal masks with two leads**

Laryngeal masks with breathing and drainage tubes can be reprocessed using special jets E 497 and E 498, fitted in the mobile injector unit for anaesthetic instruments. The special jet must replace one of the E 466 jets for breathing bags. Both arms of the jet must be aligned in parallel with the water inlet pipe. The open side of the laryngeal mask must face towards the middle of the wash cabinet.

Please observe the operating instructions for the mobile unit and also the washer-disinfector carefully.

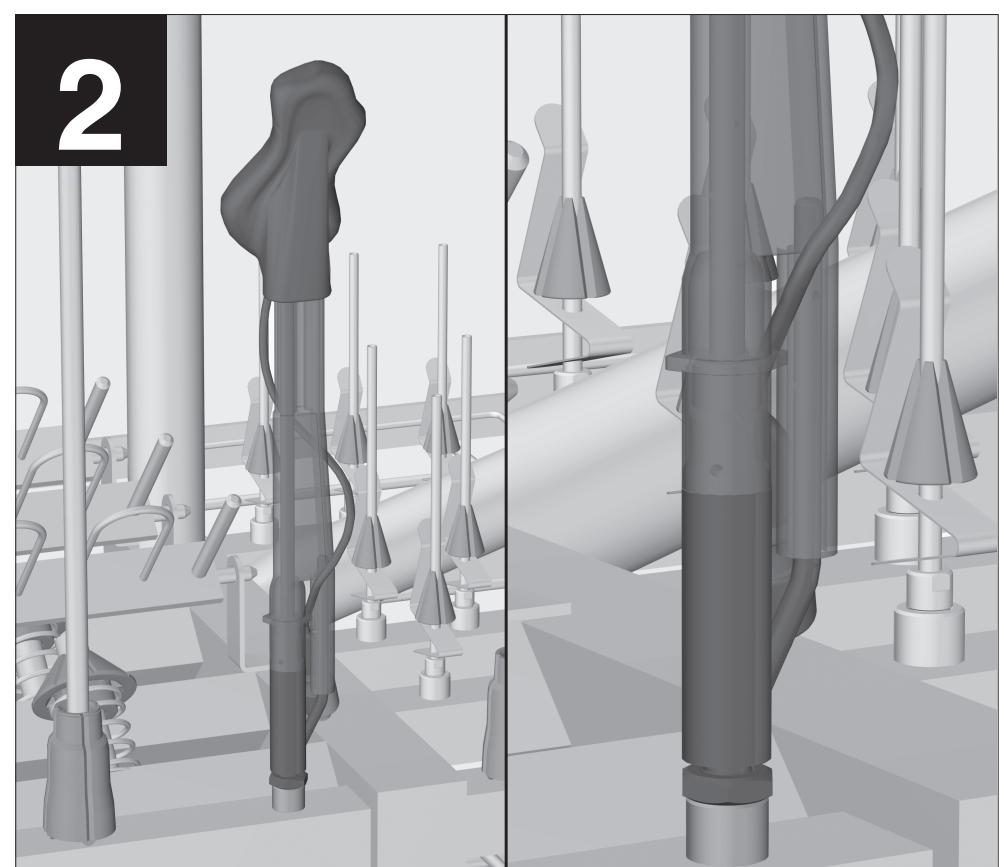
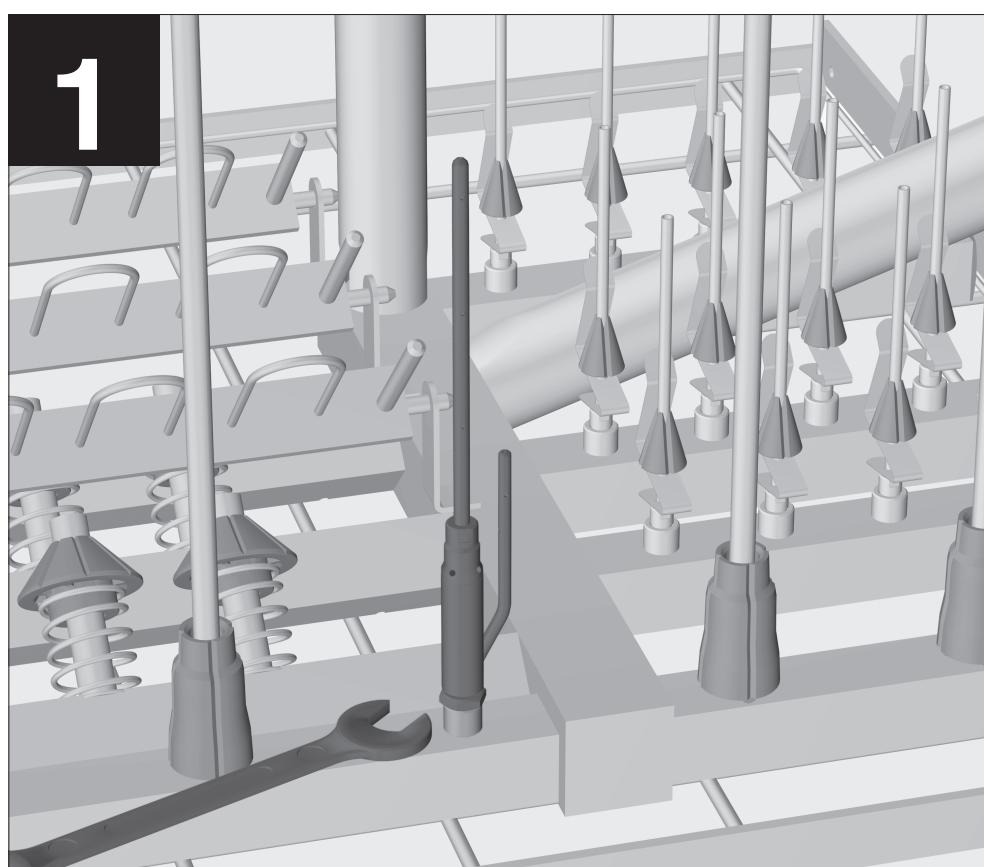
#### **1 Fitting**

- Unscrew injector jet E 466 with an SW 7 spanner.
- Screw special jet E 497 or E 498 into the connection and align it.
- Secure using an SW 13 counter nut.

#### **2 Use**

- Use the balloon lead to deflate the cuff of the laryngeal mask as much as possible and then close the balloon lead valve.
- Place the breathing tube over the long arm of the jet and the drainage tube over the short arm.
- Guide the balloon lead between the tubes to fix it in position.
- Push the breathing tube down over the injector jet until it covers the jet openings.

After reprocessing, inflate the cuff and check that it is intact and working correctly.

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