Inserts E 458 and E 364 for baby bottle teats

 Please read the Warning and Safety instructions in the Operating Instructions for your washer-disinfector carefully. Only process baby bottle teats in your washer-disinfector if they have been declared machine washable by the manufacturer. Follow the manufacturer's care instructions.

Latex teats have a limited lifespan and must be replaced when necessary.

The programme used to process the teats must be suitable for the material from which they are manufactured. We recommend thermal disinfection (e.g. A₀ 600), preferably using a Vario programme.

For improved results the main wash temperature can be set to 65°C. This is because milk protein requires high temperatures for denaturation to take place. Please contact your Miele Professional application specialist if you require advice.

Inserts:


E 458 for screw cap teats. 36 compartments 29 x 29 mm, H 63 mm, W 215 mm, D 445 mm
E 364 for wide necked teats. 36 compartments 41 x 41 mm, H 77 mm, W 215 mm, D 445 mm

Optional accessories:

E 135 with lid, for 19 x 250 ml capacity baby bottles
E 135/1 with lid, for 19 x 110 ml capacity baby bottles
E 135/2 with lid, for 19 x 90 ml capacity baby bottles
E 135/3 with lid, for 19 x 120 ml capacity baby bottles
E 142 mesh tray for screw caps
A 16 lid for E 142 mesh tray


Areas of application:

Inserts E 458 and E 364 are suitable for cleaning and disinfecting 36 baby bottle teats each. They can be used in the upper or lower basket of the washer-disinfector or in empty mobile units e. g. E 935-3, E 735-3 or E 555.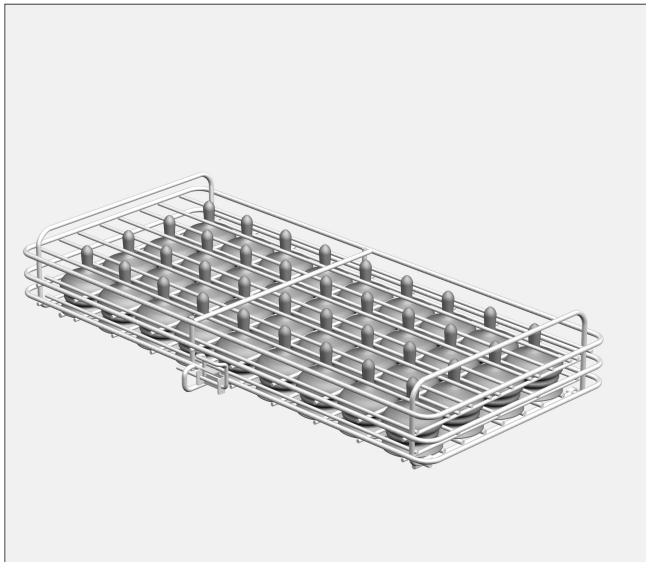


E 364:

- Undo the catch and open the insert out.
- Place used wide necked teats into the insert with the open end facing upwards.
- Close and secure the catch on the insert.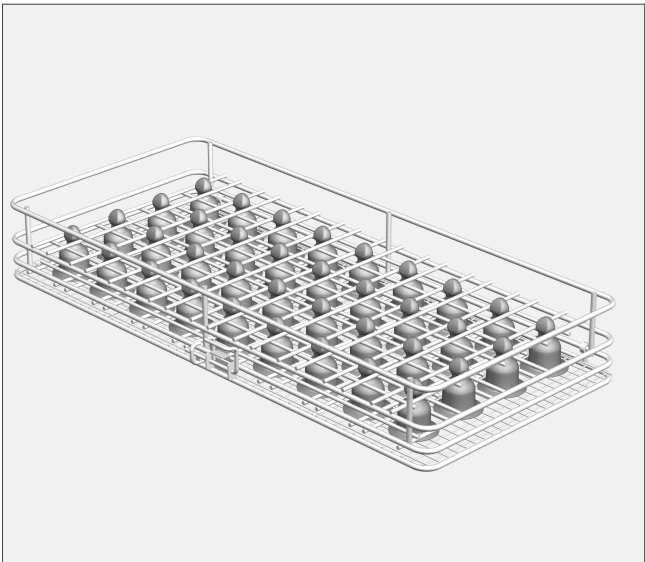


E 458:

- Undo the catch and open the insert out.
- Place used screw cap teats into the insert with the open end facing upwards.
- Close and secure the catch on the insert.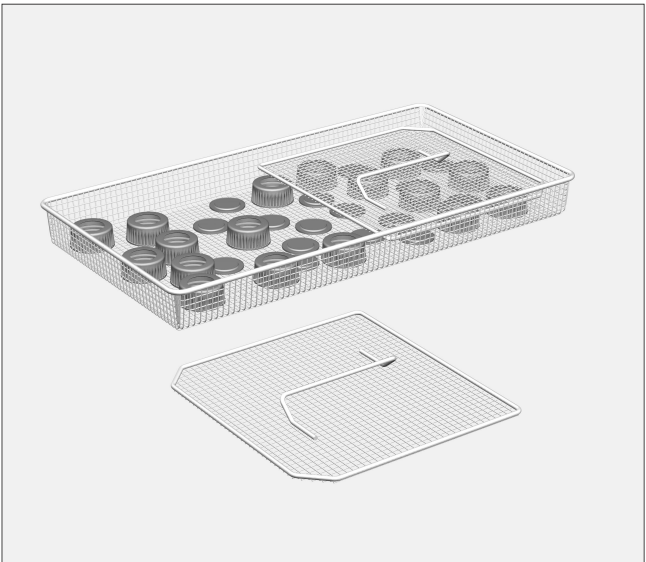

E 142 with A 16:

- Place caps and screw caps with the threaded side facing downwards in the mesh tray.
- Use a cover over the mesh tray to keep secure light weight plastic items.

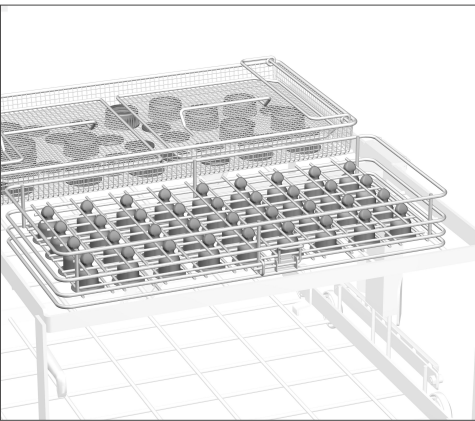If the process is not to be followed by sterilisation, the load should be dried completely to avoid the development of water-borne bacteria. A sufficient drying time is, therefore, absolutely essential.

- Place the inserts into the mobile unit making sure that the open ends of the teats are facing downwards.
- Then place the covered mesh tray E 142 into the mobile unit.

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