E 135 insert for baby bottles


⚠ Please read the Warning and Safety instructions in the Operating Instructions for your washer-disinfector carefully. You should only process baby bottles in this insert if they have been declared as suitable for machine processing by the manufacturer. Please also observe the manufacturer's care instructions. The programme used to process the bottles must be suitable for the material from which they are manufactured. We recommend thermal disinfection (e.g. A₀ 600), preferably using a Vario programme.
For improved results the main wash temperature can be set to 65°C. This is because milk protein requires high temperatures for denaturation to take place. Do not use highly alkaline cleaning agents as these can damage the markings on the bottles. Please contact your Miele Professional application specialist and/or your cleaning agent supplier if you require advice.

Baby bottle inserts:

- E 135 with lid, for 19 x 250 ml capacity baby bottles, H 194, W 192, D 447 mm
- E 135/1 with lid, for 19 x 110 ml capacity baby bottles, H 133, W 192, D 447 mm
- E 135/2 with lid, for 19 x 90 ml capacity baby bottles, H 123, W 192, D 447 mm
- E 135/3 with lid, for 19 x 120 ml capacity baby bottles, H 133, W 192, D 447 mm

Optional accessories:

- E 364 insert for 36 wide necked teats
- E 458 insert for 36 screw cap teats
- E 142 mesh tray for screw caps
- A 16 lid for E 142 mesh tray

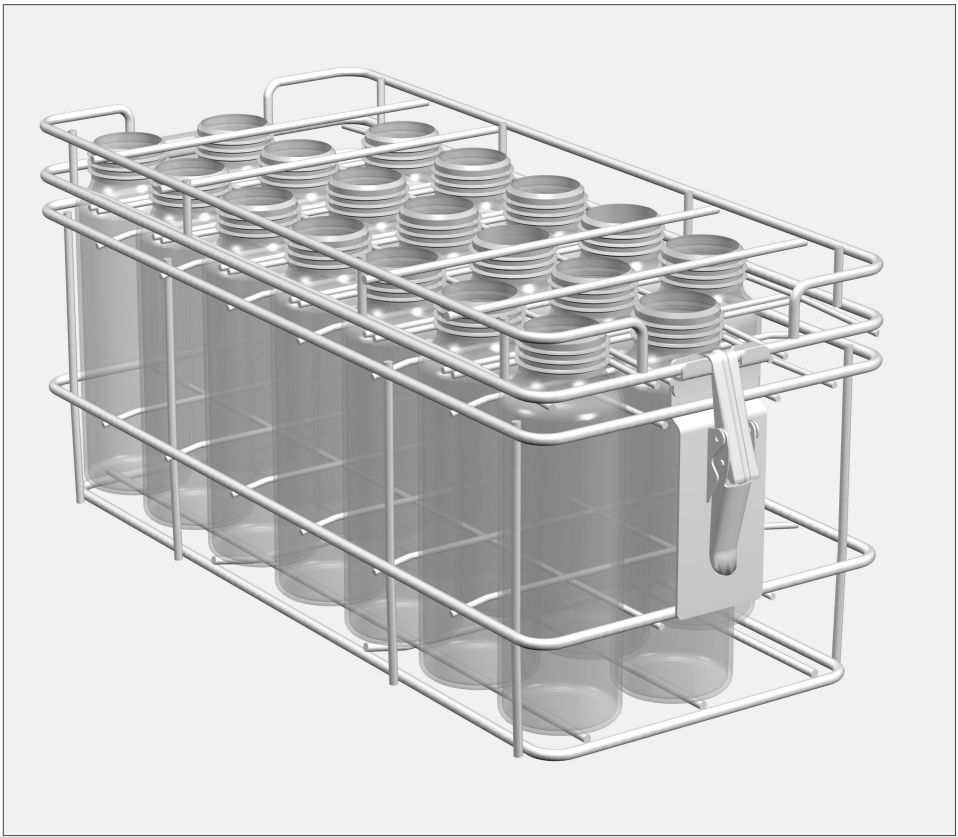
Areas of application:


The E 135 insert is suitable for cleaning and disinfecting 19 baby bottles. It can be used in the upper or lower basket of the washer-disinfector or in empty mobile units e. g. E 935-3, E 735-3 or E 555.

- Open the lid catch and take the lid off the insert.
- Place used baby bottles into the insert with the open end facing upwards.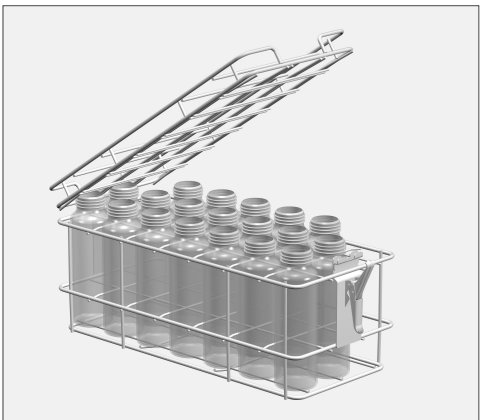


- Replace the lid over the insert. The mesh on the lid will hold the bottles securely in position.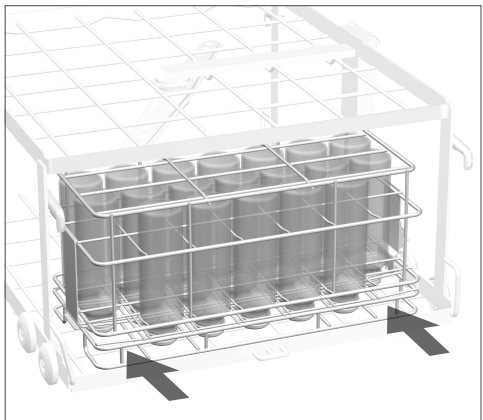

- Place the insert into the mobile unit making sure that the open ends of the bottles are facing downwards.

Both blanked off corners of the insert must be positioned in the corner of the mobile unit, i.e. in the corners of the wash area. This part of the insert is not used for bottles because water from the spray arms cannot get into the bottles sufficiently to clean them properly.

If the process is not to be followed by sterilisation, the load should be dried completely to avoid the development of water-borne bacteria. A sufficient drying time is, therefore, absolutely essential.

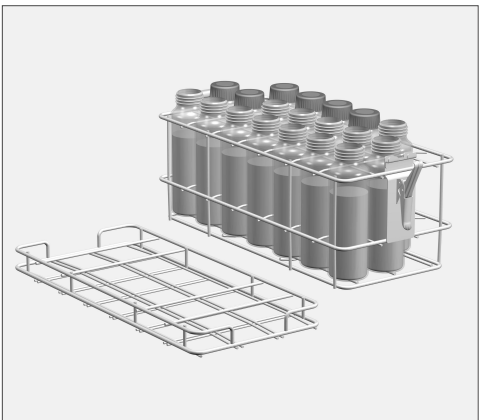- After cleaning, disinfecting and drying bottles can be refilled in the insert.
- First take the lid off the insert.
 - Then fill the bottles, replace their caps and store in a cool place until required.

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