**Miele**

## **ProClean Check**

Routine Checks for ideal process results.

Miele Professional. Immer Besser.

# Check Process Results During Routine Operation.

Reliable reprocessing is a must in dental settings. Successful process validation lays the groundwork for ensuring that reprocessing is effective – but this step is not enough by itself. Processes also need to be monitored during routine operation. Miele Professional’s solution in the form of ProClean Check indicators gives your dental team the tools to assess and document hygiene standards.

## ProClean Check

**Use:** for verifying the washing effectiveness of the automatic machines as part of routine cleaning checks.

**Function:** routine verification of cleaning cycles in automatic washer and washer-disinfectors through indicator strips containing a mixture of specially combined natural-colored components. The indicator colored test soil dissolves with most of the cleaning agent and is swept along with the washing fluid.

**Application:** Place one ProClean Check Holder with one ProClean Check Indicator at different positions on the lower and upper basket to monitor the performance on all areas of the washing machine. After the cleaning cycle, remove each ProClean Check Indicator holder and analyze the ProClean Check Indicators using the reference table and record the results.

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R01

Slide indicators inside ProClean Check Holder (x2)

Place on upper and lower basket inside machine

Program cycle

Visual inspection of indicators

Documentation

**200** indicators per packaging unit

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Miele Ltd./Limitée Canada