DataDiary Android APK Installation Kurzanleitung // Brief instruction

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The DataDiary installation file for Android operating systems is provided via the Miele website (MieleDataDiary1300007.apk, size: 69.6 MB (73,031,879 bytes)).

Customer benefits Product overview System requirements Contact **Downloads**

DataDiary app for Android devices

The app available on this page is for Android tablets. Please refer to the attached documents for details. A version for iPads can still be obtained from the Apple App Store.

App description	↓	Release Notes	↓	EULA	↓
DataDiary privacy policy	↓	Installation file	↓	Installation manual	↓
DataDiary manual	↓				

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By downloading the file, you agree to the terms of use (EULA) and the privacy policy. Both documents are available on the Miele website.

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After downloading the file MieleDataDiary1300007.apk to your Android tablet, the installation can take place. Unlike the installation of apps via the Google Play Store, the installation of apps from unknown sources must be permitted once. This is only necessary for the installation of the apk file and must be switched off again after the installation for security reasons. These instructions describe this procedure.

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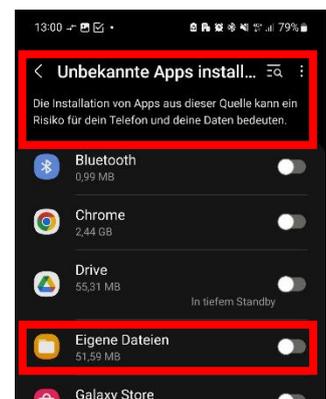
By clicking on the apk file in the file manager (here: "Eigene Dateien" or "My Documents"), the user is automatically redirected to the settings page to allow installation from unknown sources.

To do this, activate the switch behind the app.

Please note that depending on the end device and Android version, this app will be named differently.

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
After activating the switch, you will be asked to install the DataDiary app.

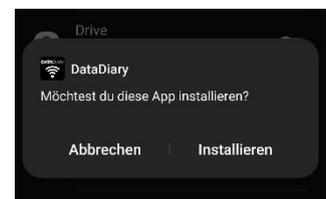Please select "Install" here. The installation takes place automatically in the background.

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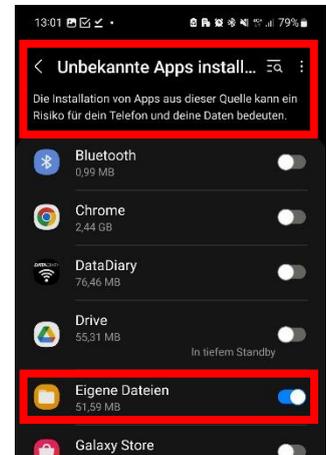
After the installation has been completed, a message is displayed which must be confirmed with "Done". This will take you back to the settings to deactivate further installations from unknown sources.

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
Please now deactivate the switch behind "My Documents" again to prevent further - possibly unwanted - installations of apps from unknown sources.

This step is absolutely necessary to reduce possible security risks!

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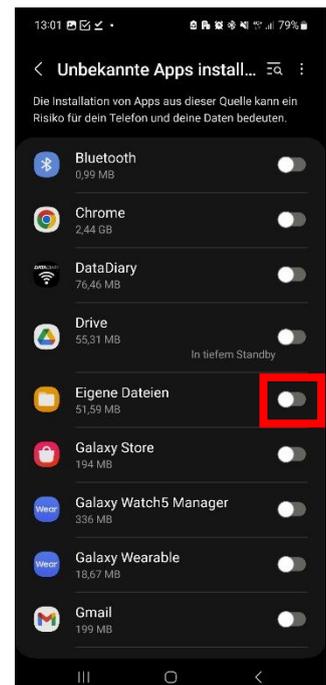The settings can now be exited. You can then start the DataDiary app and begin setting up DataDiary.

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After starting the DataDiary app, please allow access to files on the device to enable the DataDiary app to store report files.